



optimum medical
support™

Training and education to Optimise patient care

Visit www.optimummedicalsupport.co.uk to find out more.



ENVIRONMENT POLICY



As a manufacturer of medical products we recognise that we have a responsibility to reduce our impact on the environment and to conserve natural resource. We endeavour to use the smallest viable packaging size to minimise material usage and therefore landfill footprint.

When transporting products we make sure our transport operations are as efficient and environmentally friendly as possible.

All printed marketing materials are 50% recycled and certified under FSC's Chain-of-Custody programme.



optimummedical

Optimum Medical

First floor, Unit 5, Armley Court, Armley Road,
Leeds, LS12 2LB, United Kingdom

Tel.: +44 (0) 8456 435 479, Fax: +44 (0) 1142 383 826

www.optimummedical.co.uk



Company Registration No: 6779883 OMS-UFCS-102 Sept 15

independence with confidence

www.optimummedical.co.uk

Ugo fix™
catheter strap



designed to support urethral catheters,
suprapubic catheters and sheath systems



optimummedical®

Ugo fix catheter strap


At **Optimum Medical** we understand the need for secure and reliable catheter and leg bag fixation.

We have developed the **Ugo Fix Catheter Strap** to provide comfortable support giving the patient the confidence to move around freely and safely.


The **Ugo Fix Catheter Strap** is designed to support urethral catheters, suprapubic catheters and sheath systems. It can fix in place on the catheter or leg bag tubing as desired.

Key strengths:


- Retains tubing in a fixed position so no tension is placed on urethra or bladder neck yet allows natural movement.
- Effectively assists in supporting weight of leg bag when used with **Ugo Fix Sleeve**.
- Convenient and simple design.
- Clear instructions for use to ensure correct fitting.
- Reusable and washable hypoallergenic fabric strap.
- Silicone technology ensures firm grip to skin without slipping or irritation.

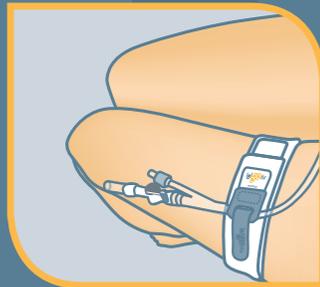

Using the **Ugo Fix Catheter Strap** on upper leg

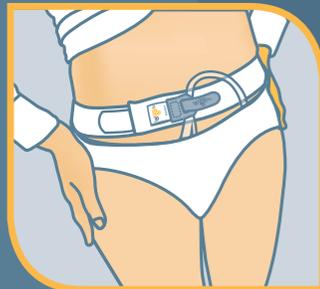

Using the **Ugo Fix Catheter Strap** on abdomen

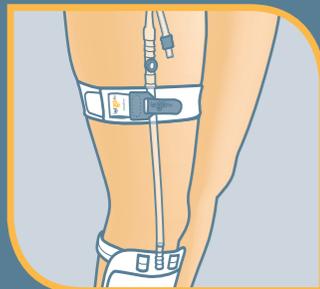

Using the **Ugo Fix Catheter Strap** on the leg bag tubing



Using the **Ugo Fix Catheter Strap** at the catheter bifurcation

LATEX FREE




We recommend that the **Ugo Fix Catheter Strap** is changed daily or sooner if the strap becomes soiled. To make this easy we have packaged the straps in sets of 5 to assist with the daily wear and wash cycle.

To maintain the **Ugo Fix Catheter Strap's** optimum performance, we recommend that the patient washes the strap in a laundry bag. To assist with this we provide a laundry bag in each box of **Ugo Fix Catheter Straps**.

How to use the Ugo Fix Catheter Strap

Prior to fitting it is important that you measure around the leg or abdomen and use the correct length.

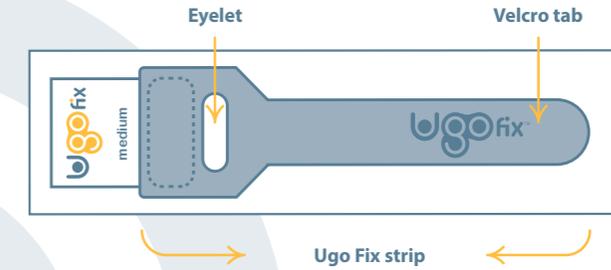

Before securing the tubing/catheter

- 1 Place the strap around either the upper leg or abdomen in the desired position with the two silicone lines facing down on the skin. Ensure the strap is not too tight and the **Ugo Fix** strip is positioned where easily accessible.



Securing the tubing/catheter

- 2 Lift the **Ugo Fix** strip and (a) wrap it over and under the tubing and then (b) through the eyelet.
- 3 Ensure the **Ugo Fix** strip is in the desired position and has a firm hold of the tubing, then secure in place on the strap using the Velcro tab.
- 4 Finally ensure the catheter is not pulling when stood up and that you are able to move around comfortably with it in place.



OM Code	PIP Code	NPC Code	Product Description	Inner Pack	Outer Pack
3001	393-3371	n/a	Ugo Fix Catheter Strap Extra short - up to 35cm	5 pcs.	50 pcs.
3002	393-3389	FSS1068	Ugo Fix Catheter Strap Short - up to 45cm	5 pcs.	50 pcs.
3003	393-3397	FSS1069	Ugo Fix Catheter Strap Medium - up to 80cm	5 pcs.	50 pcs.
3009	393-3405	FSS1070	Ugo Fix Catheter Strap Long - up to 130cm	5 pcs.	50 pcs.
3010	393-3413	n/a	Ugo Fix Catheter Strap Extra Long - up to 180cm	5 pcs.	50 pcs.