



optimum medical
support™

Training and education to Optimise patient care

Visit www.optimummedicalsupport.co.uk to find out more.



ENVIRONMENT POLICY



As a manufacturer of medical products we recognise that we have a responsibility to reduce our impact on the environment and to conserve natural resource. We endeavour to use the smallest viable packaging size to minimise material usage and therefore landfill footprint.



When transporting products we make sure our transport operations are as efficient and environmentally friendly as possible.

All printed marketing materials are 50% recycled and certified under FSC's Chain-of-Custody programme.



optimummedical

Optimum Medical

First floor, Unit 5, Armley Court, Armley Road,
Leeds, LS12 2LB, United Kingdom

Tel.: +44 (0) 8456 435 479, Fax: +44 (0) 1142 383 826

www.optimummedical.co.uk



Company Registration No: 6779883 OMS-UFS-101 Sept 15

independence with confidence

W go fix™
sleeve leg bag holder



designed for supporting
leg bags attached to
indwelling catheters and sheaths



optimummedical®

www.optimummedical.co.uk

Ugo fix

sleeve leg bag holder

At **Optimum Medical** we understand the need for secure and reliable catheter and leg bag fixation.

We have developed the **Ugo Fix Sleeve** to provide comfortable support giving the patient the confidence to move around freely and safely.

The **Ugo Fix Sleeve** is designed to support leg bags attached to urethral catheters, suprapubic catheters and urinary sheath systems.



Key strengths:


- Washable, knitted, fibre blends providing strength and durability
- Contours to leg shape for even distribution of urine and reduced noise
- Graduated lines of concentrated elastane for support and controlled expansion when bag is filling
- External seam to prevent skin damage and provide comfort
- Strengthened outlet hole to maintain continuous support
- Increased width and strength to top and bottom bands for support and fixation
- No unnecessary tube holes in the top seam therefore preventing disconnection of the link drainage system
- Colour coded top seam to assist with correct fitting and re ordering
- Ideally complemented by the **Ugo Fix Catheter Strap**

LATEX FREE

HOW TO APPLY THE UGO FIX SLEEVE LEG BAG HOLDER

Before fitting, it is important that you measure around the widest part of your thigh or calf and use the correct size in the table below.

OM Code	PIP Code	Product Description	Inner Pack	Outer Pack
3005	394-0178	Ugo Fix Sleeve Small - 24cm-39cm	4 pcs.	80 pcs.
3006	394-0160	Ugo Fix Sleeve Medium - 36cm-55cm	4 pcs.	80 pcs.
3007	394-0152	Ugo Fix Sleeve Large - 40cm-70cm	4 pcs.	80 pcs.
3008	394-0145	Ugo Fix Sleeve Extra Large - 65cm-90cm	4 pcs.	80 pcs.

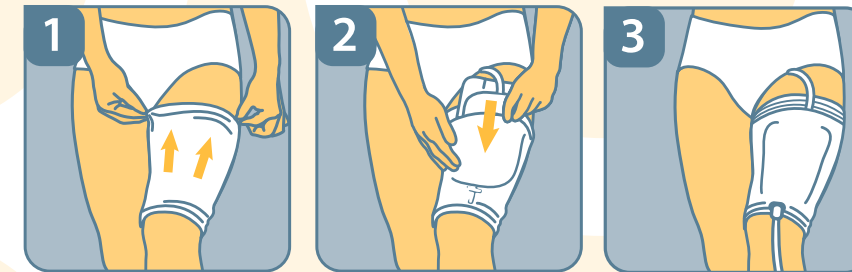

- 1 Pull the **Ugo Fix Sleeve** over your foot and position on your thigh or calf. Ensure the coloured strip is at the top and facing outwards.
 - 2 Slide your leg bag into the front pouch through the opening at the top.
 - 3 Push the drainage tap through the hole in the bottom of the sleeve and adjust to ensure optimum drainage and comfort.
- ➔ You can now empty your leg bag without removing it from the **Ugo Fix Sleeve**.



We recommend that the **Ugo Fix Sleeve** is changed daily or sooner if the sleeve becomes soiled. To make this easy we have packaged the sleeves in sets of 4 to assist with the daily wear and wash cycle.

To maintain the **Ugo Fix Sleeve's** optimum performance, we recommend that the patient washes the sleeve in a laundry bag. To assist with this we provide a laundry bag in each box of **Ugo Fix Sleeve**.

