
readycath flex

Instructions for use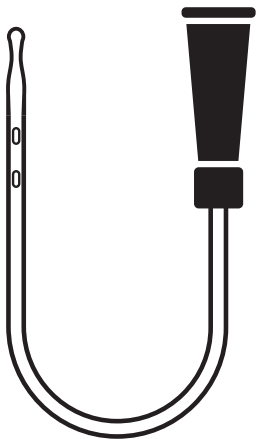

PRODUCT REF / DESCRIPTION

Code	Description	Quantity
RCFM10	10CH urinary intermittent male catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters
RCFM12	12CH urinary intermittent male catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters
RCFM14	14CH urinary intermittent male catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters
RCFM16	16CH urinary intermittent male catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters
RCFF10	10CH urinary intermittent female catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters
RCFF12	12CH urinary intermittent female catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters
RCFF14	14CH urinary intermittent female catheter hydrophilic coated, ready-to-use	30 individually wrapped catheters

This is a product information leaflet for the user.

Readycath flex has been recommended/prescribed for you by a Healthcare Professional.

Do not share it with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

INGREDIENTS/COMPOSITION

A hydrophilic coated TPU catheter.

INTENDED PURPOSE

The **Readycath flex** catheters are indicated for intermittent urinary catheterisation by inserting into the urethra to pass urine from the bladder.

INDICATIONS FOR USE

ISC catheter for patients who are required to use a catheter to empty their bladder. ISC can be used to empty the bladder regularly which will help protect the kidneys and reduce the occurrence of Urinary Tract Infections (UTIs).

CONTRAINDICATIONS

The **Readycath flex** may be contraindicated or not be suitable for individuals with:

- Acute urethritis.
- Acute prostatitis.
- Acute epididymitis.
- Patients with calcareous urolithiasis.

Do not use the **Readycath flex** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.

Do not use the **Readycath flex** if you are sensitive or allergic to any of the device ingredients.

WARNINGS AND PRECAUTIONS**Warnings:**

- After opening the packaging do not touch the catheter to avoid damaging the hydrophilic coating.
- This is a single use device. Do not re-sterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

Precautions:

- It is recommended that the catheter is used in line with local policies and procedures following a patient assessment by a healthcare professional.
- This product is single-use, do not reuse.
- Do not use if the package is damaged, or soiled.
- Insertion and removal should be as gentle as possible to prevent urethral injury.
- Slowly advance the catheter through the urethra into the bladder. If substantial resistance is met, do not force the catheter. Slowly pull out the catheter, check it carefully and speak to a healthcare practitioner if required.
- Do not reinsert a used catheter.

The **Readycath flex** is not made with natural rubber latex. Do not use the **Readycath flex** if it is damaged or soiled. Keep out of reach of children.

UNDESIRABLE SIDE EFFECTS**Adverse reactions:**

- Bleeding in the urethra.
- Catheter-associated urinary tract infection.
- Urethral injuries and inflammation.

Any serious incident or malfunction that has occurred in relation to **Readycath flex** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

1. Gently clean the genital area with soap and water. If you are using public toilets, cleansing wipes or antibacterial hand gel may be useful to enable you to ensure the genital area is clean prior to catheterisation.
2. Wash your hands thoroughly with soap and water. Be careful not to touch anything else other than your catheter once you have washed your hands.
3. Open the package and expose the funnel.
4. Holding the funnel, slide the contactless sleeve along the catheter towards the tip.
5. Carefully insert the catheter into the urethra until urine starts to flow - this occurs after the eyelets of the catheter reach the bladder.
6. The catheter can be attached to a urine bag or drained directly into a toilet or jug.
7. When urine stops flowing, slowly begin to withdraw the catheter, stopping each time urine begins to flow until the bladder is empty.
8. After use, remove the catheter from the urethra and put in a disposal bag and place in the general household rubbish.
9. Wash hands after catheterisation.

STORAGE AND HANDLING

The catheter is supplied sterile. Sterilised using gamma radiation. Store between 5°C and 35°C. Use immediately after the package is opened and dispose of after use. The device should be stored in a cool, dry and dark place. Keep dry. Keep out of direct sunlight. Do not store catheters where they are directly exposed to organic solvents, ionizing radiation, ultraviolet light and high temperatures. Unopened packs have a 3-year shelf life. Do not use after expiry date.

DISPOSAL

Readycath flex must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

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Hangzhou Jimushi Meditech Co., Ltd.
EC REP Shanghai International Holding Corp. GmbH (Europe)
UK REP Etheria Medical Ltd
MedEnvoy Global B.V.,
Hunter Urology Limited
Optimum Medical Solutions Ltd.