EC REP


MT Promedit Consulting GmbH  
Altenhofstrasse 80  
66386 St. Ingbert Germany

REF 3100/3101/3102/3103/3104/3105

REF 3106/3107/3108/3109/3110/3111

3100-3111-IFU-Jan9-V1.0

ugo™  
sheath

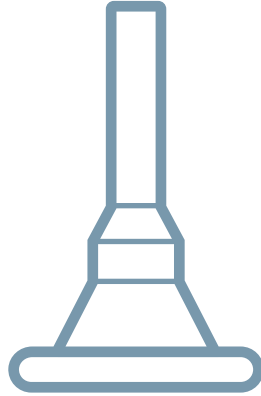## Instructions for use

## Ugo Sheath | soft silicone, self-adhesive penile sheath

For the containment of male urinary incontinence | Up to 24 hour use

### INSTRUCTIONS FOR USE

#### Intended use:

The **Ugo Sheath** is used for the management of male urinary incontinence by external application of the sheath over the penis. It is connected to a urine drainage bag to drain urine.

#### Indications:

For non-invasive management of male urinary incontinence.

#### Contraindications:

The **Ugo Sheath** must be used in line with local policies and procedures following a patient continence assessment by a clinician.

The sheath is contraindicated;

- In patients with urinary retention
- On penile skin which is broken, inflamed or swollen
- In patients who are sensitive or allergic to silicone
- On a retracted penis where there is no accessible penile length to fit the sheath

#### Warnings:

- Do not use moisturisers, moisturising soap, shower gel, wet wipes, baby wipes or talcum powder as they may affect the adhesion of the sheath
- Do not use on skin which is broken or inflamed
- Do not shave pubic area as may cause skin irritation
- Do not use sheath if it is damaged or soiled
- Keep out of reach of children

#### How to select the Ugo Sheath:

- Patient suitability and assessment for a urinary sheath and the sizing and fitting should always be carried out after a full patient assessment and in accordance with local and national best practice policies and procedures by appropriately qualified and trained personnel
- Select the correct size sheath length and then use the **Ugo Sheath** measuring guide to measure the penis where its diameter is largest and select the correct size diameter

**Note:** When sizing for a sheath take into consideration natural changes in penis size.

### Sizing chart

The **Ugo Sheath** is available in the following sizes:

CODE	PR. NAME	LENGTH	DIAMETER
3100	<b>Ugo Sheath</b>	Standard	Ø 25mm
3101	<b>Ugo Sheath</b>	Standard	Ø 28mm
3102	<b>Ugo Sheath</b>	Standard	Ø 30mm
3103	<b>Ugo Sheath</b>	Standard	Ø 32mm
3104	<b>Ugo Sheath</b>	Standard	Ø 36mm
3105	<b>Ugo Sheath</b>	Standard	Ø 40mm
3106	<b>Ugo Sheath</b>	Pop-On	Ø 25mm
3107	<b>Ugo Sheath</b>	Pop-On	Ø 28mm
3108	<b>Ugo Sheath</b>	Pop-On	Ø 30mm
3109	<b>Ugo Sheath</b>	Pop-On	Ø 32mm
3110	<b>Ugo Sheath</b>	Pop-On	Ø 36mm
3111	<b>Ugo Sheath</b>	Pop-On	Ø 40mm

#### How to fit the Ugo Sheath:

The **Ugo Sheath** can be worn up to 24 hours.

- Ensure the penis is clean and dry and inspect the skin to check that it is intact before fitting the sheath
- Wash and dry your hands (if a carer is fitting the sheath for you then they must apply a clean pair of gloves after drying their hands)
- Trim any hair which may get caught in the sheath and if necessary use the enclosed gauze fabric square hair guards
- Position the **Ugo Sheath** over the end of the penis

**Note:** Ensure the foreskin is in place covering the glans penis and not retracted when fitting the sheath.

- Leave a small space between the end of the penis and the base of the sheath spout
- Fully unroll the **Ugo Sheath** along the length of the penis ensuring that it rolls evenly

**Note:** Hold the sheath in place to ensure the adhesive has bonded.

- Fit the urine drainage bag connector into the sheath spout ensuring it is not pushed too far in and that the tubing is not twisted or the sheath spout has not kinked
- Secure the urine drainage bag to the leg using appropriate fixation devices and make sure you are comfortable and can move around freely

#### How to remove the Ugo Sheath:

- Wash and dry your hands (if a carer is fitting the sheath for you then they must apply a clean pair of gloves after drying their hands)
  - Loosen the base of the **Ugo Sheath** all the way around
  - Use warm water and soap or use an adhesive remover to help unroll the sheath and gently remove
- Note:** Always inspect the penile skin after removal of the sheath and prior to fitting a new **Ugo Sheath**.

#### Caring for the Ugo Sheath:

- After the **Ugo Sheath** is fitted, ensure it is connected to an appropriate urine drainage bag
- Secure and support the leg worn urine drainage bag to prevent pulling on the sheath

(if a 2L urine drainage bag is used then this must be attached to a stand or a hanger)

- Maintain a good daily hygiene routine by washing with soap and water between each sheath fitting
- Allow enough time for the penis skin to dry and inspect the penile skin between sheath fittings

### Potential complications and side effects:

There is an increased risk of complications from wearing a sheath if you have decreased sensation and if the **Ugo Sheath** is not sized and fitted correctly.

Remove the **Ugo Sheath** immediately if you experience any of the following and seek medical help;

- Swelling, inflammation or discomfort of the penis
- Lesions or broken skin on the penis
- Compression or discolouration of the penis
- Cloudy or foul-smelling urine
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort
- Urine not draining into the urine drainage bag
- Urine leaking from the sheath or sheath falling off

### Disposal of the Ugo Sheath:

Dispose of all sheath equipment in line with your local waste disposal policy.

### Packaging:

The **Ugo Sheath** is supplied in protective boxes of 7.

It is important that the lid is kept closed to protect the sheaths between use.

Once the box has been opened then use within 3 months.

This packaging has been designed with minimal plastic content.

Each box and wrapping is fully recyclable.

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