SWE Ugo Catheter Valve Kateterventil

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GBR Ugo Catheter Valve

Instructions for use

A discreet, comfortable, easy to operate catheter valve with a safety lever feature to prevent accidental opening and a universal stepped connector which can be inserted into all catheter materials.

Introduction

At Optimum Medical, we understand the importance of your independence and the Ugo Catheter Valve is especially designed to ensure you feel comfortable and confident every day.

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The Ugo Catheter Valve has been prescribed by you a health care professional following a bladder and catheter assessment. It is important that you use the valve as instructed by your doctor or nurse as part of your ongoing catheter management plan.

The Ugo Catheter Valve is a medical device designed to be used with an indwelling urethral or suprapubic catheter. It is suitable for men and women.

Only use the Ugo Catheter Valve if you (or your carer) have the ability to operate it at regular intervals and can remember to do so.

Benefits of using a valve

The Ugo Catheter Valve is a discreet alternative to urine drainage bags, however unlike bags it enables intermittent bladder filling and drainage, therefore maintaining normal bladder function.

Contra-indications

The Ugo Catheter Valve may not be suitable for individuals who:

- Are unable to operate the valve
- Are unable to remember to operate the valve
- Have a small bladder capacity
- Have reduced bladder sensation
- Have renal impairment or ureteric reflux
- Have uncontrolled overactive bladder

How to drain your bladder using your Ugo Catheter Valve

Drain your bladder using the valve as frequently as recommended by your doctor or nurse

1. Wash and dry your hands

2. Hold the valve over a toilet or suitable receptacle

3. Open the valve by pushing the lever tap fully into the downward position (you may feel some mild tension in the lever, this is normal – it is the safety feature working to prevent accidental opening)

4. Allow the valve to drain the bladder completely (the flow may be slow – this is due to the narrow lumen inside the catheter)

5. Once urine has stopped draining, close the valve by lifting the lever tap fully into the upright position

6. Dry the end of the valve with a clean wipe if necessary

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• If you are concerned that your catheter is not draining urine and you are in discomfort then contact your doctor or nurse immediately.

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