Pantone 5425

SWE Ugo Fix Bag Belt Benpåse remmar

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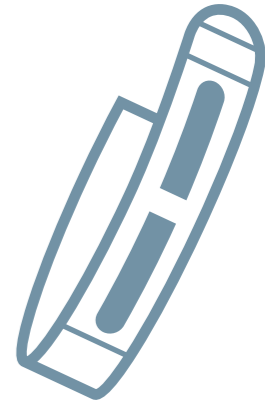
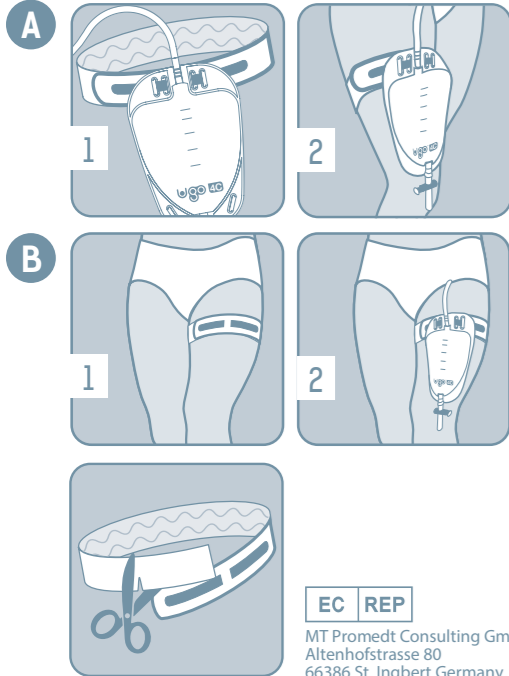
MT Promed Consulting GmbH
Altenhofstrasse 80
66386 St. Ingbert Germany

GBR **Ugo Fix Bag Belt**

At Optimum Medical we understand the need for secure and reliable fixation and support of your catheter and leg bag. The **Ugo Fix Bag Belt** provides enhanced support and comfort for your **Ugo Leg Bag** enabling you to feel confident when increasing your level of activity. It is suitable for attaching any size **Ugo Leg Bag** to your thigh, upper knee or shin.

The **Ugo Fix Bag Belt** is designed to hold the top of your **Ugo Leg Bag**. It is fully adjustable and can be cut to fit if necessary. The **Ugo Fix Bag Belt** is washable and reusable, please follow the washing instructions provided.

Instructions for use

It is important that you do not let your catheter or leg bag hang down unsupported as this will cause discomfort. Always support the weight of your leg bag when fitting the **Ugo Fix Bag Belt**.

Fitting options

You may choose to attach your **Ugo Leg Bag** to the **Ugo Fix Bag Belt** first and then to your leg afterwards (**Method A**) or you may find it easier to fit the **Ugo Fix Bag Belt** to your leg first and then attach your **Ugo Leg Bag** afterwards (**Method B**).

Method A

1. Feed the grey Velcro tabs through the four holes at the top of your **Ugo Leg Bag** and secure
2. Place the **Ugo Fix Bag Belt** around the leg in the desired position with the silicone line facing down on the skin and fix in place
 - Ensure the **Ugo Fix Bag Belt** is not too tight, your catheter is not pulling and that you are able to move around comfortably with it in place.

Method B

1. Place the **Ugo Fix Bag Belt** around the leg in the desired position with the silicone line facing down on the skin and fix in place
2. Feed the grey Velcro tabs through the four holes at the top of your **Ugo Leg Bag** and secure
 - Ensure the **Ugo Fix Bag Belt** is not too tight, your catheter is not pulling and that you are able to move around comfortably with it in place.

WARNING: Keep out of reach of children!

To find out more about the **Ugo Fix** please contact us by email ugo@optimummedical.co.uk or call +44 (0) 845 643 5479.

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