\* Dansu-China Health Care Co., Ltd, No. 366, Pengjialing, Yongfeng Street, Hanyang District, 430050 Wuhan, People's Republic of China

Shanghai International Holding Corp. GmbH, Eiffestraße 80, 20537 Hamburg, Germany

\*\* Manufactured by: Changzhou Holinx Industries Co., Ltd, No. 1 Dontang Road, Zhenglu Town, Tianning District, Changzhou, 213115 Jiangsu, China

Euroscene Business Solutions Limited, Flat 3, The Picture House, 195 Darkes Lane, Potters Bar, Herts, EN6 1BP

\*\*\* Nantong Strip Medical Supply Co., Ltd., A Building, 182 Yue Long Nan Road, 226001, Nantong, China

Helen Trading Company, 1303 Sofia, Bulgaria

REF **3400**

STERILE EO

EC REP

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
66386 St. Ingbert Germany

3400-IFU-Mar19-V1.0

optimummedical®

Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom  
T: +44(0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

ugopro.™  
catheterisation procedure pack

Instructions for use

GBR

## Ugo Pro

catheterisation procedure pack

Series of horizontal dotted lines for notes or additional information.

### INSTRUCTIONS FOR USE

#### Intended use:

Ugo Pro is designed to provide a pre wrapped aseptic field and essential routine components for the safe and effective aseptic removal and /or insertion of a urinary catheter.

#### Indications:

Removal and /or insertion of a urinary catheter.

#### Warnings:

- The Ugo Pro should be used by a healthcare professional in accordance with local policies and procedures
- Single use; sterile until open
- Keep out of reach of children
- Ensure that any additional medical devices placed on the aseptic field are sterile

#### Saline cleansing pod warnings:

- For external use only • Do not re-use • Do not use for injection

#### Instructions:

Ugo Pro contains equipment for the aseptic removal and/or insertion of a urinary catheter.

- Ugo Pro must be used in accordance with local policies and procedures and best practice guidance by suitably qualified and competent healthcare practitioners
- Open the Ugo Pro pouch packaging and place pack on a clean flat work surface
- Open the Ugo Pro pack outer wrapping touching only the outside corners and spread out to form an aseptic field. Arrange the contents onto the aseptic field (the contents may be used in the order presented to assist with technique)
- Add any additional sterile medical devices required for the procedure to the aseptic field
- Carry out the catheterisation procedure using the Ugo Pro essential routine components (tabled below) in line with your local policies and procedures following a full patient assessment and patient consent
- Document clearly any relevant information following the procedure including reason for catheterisation
- Dispose of all waste following the procedure in line with your local waste disposal policy

Ugo Pro Component	Instructions for use/Rationale
Outer wrapper *	<ul style="list-style-type: none"> <li>To wrap the sterile products</li> <li>Create aseptic field for the components to be placed upon</li> </ul>
Apron *	<ul style="list-style-type: none"> <li>For personal protection</li> <li>Assists with aseptic technique</li> </ul>
Grey waterproof drape *	<ul style="list-style-type: none"> <li>To be placed under the patient with grey side facing down and absorbent side up</li> <li>Protects the bed from spilt fluids during procedure</li> <li>Assists with aseptic technique</li> </ul>
Blue waterproof "0" drape *	<ul style="list-style-type: none"> <li>To be placed over the patient with blue side facing down and absorbent side up</li> <li>Assists with dignity during the procedure</li> <li>Assists with aseptic technique</li> </ul>
Waste bag *	<ul style="list-style-type: none"> <li>For all waste to be collected throughout the procedure</li> </ul>
10ml empty syringe **	<ul style="list-style-type: none"> <li>To remove fluid from pre filled Foley catheter balloon prior to catheter removal</li> </ul>
6 non-woven swabs *	<ul style="list-style-type: none"> <li>To assist with the removal of Foley catheter</li> <li>To assist with general catheterisation procedural technique</li> </ul>
Clear tray with 2 compartments *	<ul style="list-style-type: none"> <li>Holds components for procedure</li> <li>Holds cotton balls and saline cleansing solution in small compartment during procedure</li> <li>Large compartment can be used as a urine receiver during catheterisation</li> </ul>
20ml saline cleansing pod (0.9% w/v sodium chloride)***	<ul style="list-style-type: none"> <li>Empty contents over non-woven balls in tray compartment</li> <li>Cleanse catheter insertion site during procedure</li> </ul>
5 non-woven balls *	<ul style="list-style-type: none"> <li>For cleansing the catheter insertion site during procedure once coated in saline cleansing solution</li> </ul>
3 pairs of medium nitrile Ugo Pro gloves *	<ul style="list-style-type: none"> <li>3 pairs of gloves to support an aseptic technique - to be used for the following aspects of the procedure:               <ol style="list-style-type: none"> <li>Removal of catheter</li> <li>Cleaning of the catheter insertion site</li> <li>Insertion of a new catheter</li> </ol> </li> </ul>

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