


constantly creating better

UROLOGY Range

from **Optimum Medical**

specialists in healthcare
lubrication, urology and
infection prevention products

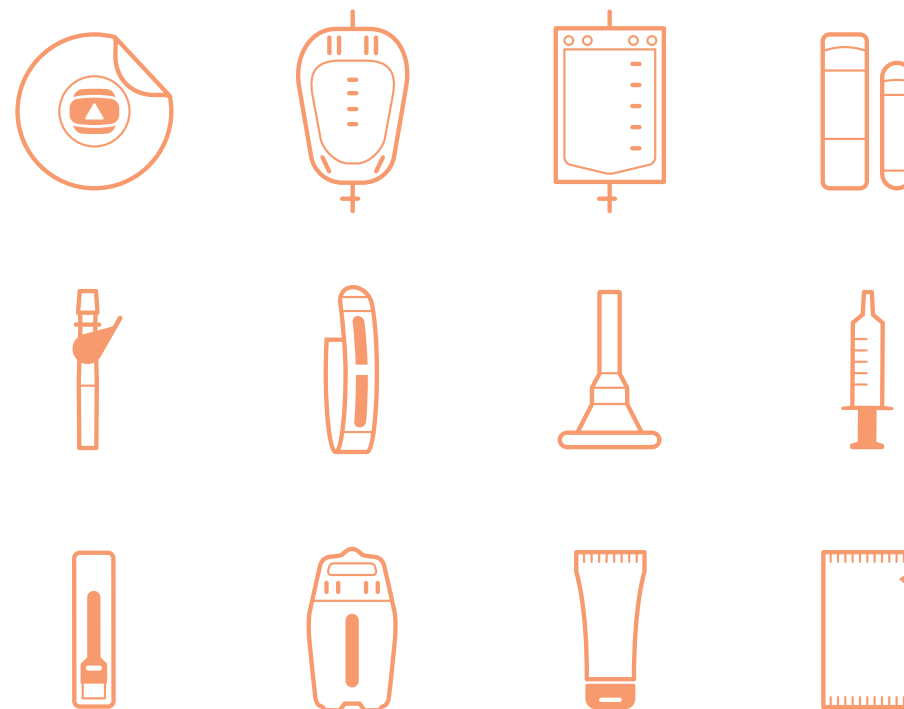

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
66386 St. Ingbert Germany



OMS-MP-3000-Apr21



optimummedical®

Over the last decade we've dedicated a lot of time to perfecting our Urology product portfolio. We've listened to clinicians and patients alike about what's important to them, and developed quality products which reflect our passion for healthcare. Our products offer the performance, precision and comfort needed to support those with intimate healthcare needs.



optimummedical®

Contents

Product Codes	4
Cost Effective Prescribing	11
Vyne Prescription Service	18
Procedure Packs	13
Ugo Pro (catheterisation procedure pack)	14
The Ugo Box	16
Drainage	19
Ugo Foley Catheter	20
Ugo Leg Bags	22
Ugo 2L Drainage Bags	24
Ugo Stand with Dignity	26
Ugo Sheath	28
Ugo Catheter Valve	30
Fixation Devices	31
Ugo Fix Gentle	32
Ugo Fix Catheter Strap	34
Ugo Fix Sleeve (leg bag holder)	36
Multi Fix Gentle	38
Catheter Drainage and Management Pack	39
Ugo 4 Weeks	40
Ugo Home	42
Catheterisation Jelly	43
OptiLube Syringes	44
OptiLube Active	44
OptiLube Active CHG Free	44
Clinician Support	46
Patient Support	47

Each product in the **Ugo Urology** range is available on prescription through “**Part IXA Appliances**” or “**Part IXB Incontinence Appliances**” on the Drug Tariff.

To find out more about the range, visit www.ugourology.co.uk, email ugo@optimummedical.co.uk or call **0113 263 3849**.

Urine Drainage

Ugo Pro (catheterisation procedure pack)

PR CODE	PR NAME	PIP CODE	OUTER CASE	DT PRICE /PER UNIT
3400	Ugo Pro Catheterisation procedure pack	414-6494	Case of 80	£2.23/per unit

Ugo Box - a fully equipped catheterisation kit

PR CODE	PIP CODE	UGO FOLEY CATHETER FR SIZE	UGO LEG BAG	11ML CATHETERISATION JELLY SYRINGE	DT PRICE
3450	415-0660	12FR	Ugo 1C	OLA CHG Free*	£16.22/per unit
3451	415-0678	14FR	Ugo 1C	OLA CHG Free	£16.22/per unit
3452	415-0686	16FR	Ugo 1C	OLA CHG Free	£16.22/per unit
3453	415-0694	12FR	Ugo 1C	OptiLube	£16.22/per unit
3454	415-0702	14FR	Ugo 1C	OptiLube	£16.22/per unit
3455	415-0710	16FR	Ugo 1C	OptiLube	£16.22/per unit
3456	415-0728	12FR	Ugo 2C	OLA CHG Free	£16.22/per unit
3457	415-0736	14FR	Ugo 2C	OLA CHG Free	£16.22/per unit
3458	415-0744	16FR	Ugo 2C	OLA CHG Free	£16.22/per unit
3459	415-0769	12FR	Ugo 2C	OptiLube	£16.22/per unit
3460	415-0751	14FR	Ugo 2C	OptiLube	£16.22/per unit
3461	415-0777	16FR	Ugo 2C	OptiLube	£16.22/per unit

Please note: The **Ugo Foley Catheter** balloon size is 10ml.

*OptiLube Active CHG Free.

Ugo Foley Catheter

PR CODE	PR NAME	PIP CODE	SIZE	DT PRICE
UFC12	Ugo Foley Catheter	414-3558	12FR	£6.00/per unit
UFC14	Ugo Foley Catheter	414-3541	14FR	£6.00/per unit
UFC16	Ugo Foley Catheter	414-3533	16FR	£6.00/per unit
UFC18	Ugo Foley Catheter	414-3525	18FR	£6.00/per unit

Ugo Leg Bags (Community) - Fabric backing – 1 pair of Ugo Fix Leg Bag Straps in every box

PR CODE	PR NAME	PIP CODE	BAG VOL	TUBE LENGTH	TAP DESIGN	INNER PACK	DT PRICE
1C/500/ST/L	Ugo 1C	400-8546	500ml	Short Tube	Lever Tap	Box of 10	£24.84
2C/500/LT/L	Ugo 2C	400-8603	500ml	Long Tube	Lever Tap	Box of 10	£24.84
3C/500/ST/T	Ugo 3C	400-8611	500ml	Short Tube	T Tap	Box of 10	£24.84
4C/500/LT/T	Ugo 4C	400-8629	500ml	Long Tube	T Tap	Box of 10	£24.84
5C/350/DI/L	Ugo 5C	400-8637	350ml	Direct Inlet	Lever Tap	Box of 10	£24.84
6C/350/ST/L	Ugo 6C	400-8645	350ml	Short Tube	Lever Tap	Box of 10	£24.84
7C/750/ST/L	Ugo 7C	400-8652	750ml	Short Tube	Lever Tap	Box of 10	£24.84
8C/750/LT/L	Ugo 8C	400-8660	750ml	Long Tube	Lever Tap	Box of 10	£24.84

Ugo Leg Bags (Hospital) - No fabric backing – 1 pair of Ugo Fix Leg Bag Straps in every pouch

PR CODE	PR NAME	PIP CODE	BAG VOL	TUBE LENGTH	TAP DESIGN	INNER PACK	DT PRICE
1H/500/ST/L	Ugo 1H	410-0517	500ml	Short Tube	Lever Tap	Box of 10	£24.53
2H/500/LT/L	Ugo 2H	410-0657	500ml	Long Tube	Lever Tap	Box of 10	£24.53
3H/500/ST/T	Ugo 3H	410-0657	500ml	Short Tube	T Tap	Box of 10	£24.53
4H/500/LT/T	Ugo 4H	410-0673	500ml	Long Tube	T Tap	Box of 10	£24.53

Ugo Pro Leg Bags – Fabric backing – includes disposable gloves and apron, and 1 pair of Ugo Fix Leg Bag Straps in every box

PR CODE	PR NAME	PIP CODE	BAG VOL	TUBE LENGTH	TAP DESIGN	INNER PACK	DT PRICE
3410	Ugo Pro 1C	415-9554	500ml	Short Tube	Lever Tap	Box of 10	£26.10
3411	Ugo Pro 2C	415-9547	500ml	Long Tube	Lever Tap	Box of 10	£26.10
3412	Ugo Pro 3C	415-9570	500ml	Short Tube	T tap	Box of 10	£26.10
3413	Ugo Pro 4C	415-9562	500ml	Long Tube	T tap	Box of 10	£26.10



The healthcare service grown for you & your patients.

Our dedicated customer care agents and our specialist nurse team are on hand to take the stress out of ordering and receiving healthcare appliances and medication.

Register your patients at:

☎ **0344 225 1518** ✉ register@myvyne.co.uk 🌐 www.myvyne.co.uk

Product Codes

Each product in the **Ugo Urology** range is available on prescription through “**Part IXA Appliances**” or “**Part IXB Incontinence Appliances**” on the Drug Tariff.

To find out more about the range, visit www.ugourology.co.uk, email ugo@optimummedical.co.uk or call 0113 263 3849.

Ugo 2L Drainage Bags

PR CODE	PR NAME	PIP CODE	TAP DESIGN	CLINICAL USE	INNER PACK	DT PRICE
9/NS/SU/DO	Ugo 9	400-8488	Non Drainable (tear corner to drain) Non sterile	Single use link drainage only	Bag of 10	£2.15
10/NS/SU/DO	Ugo 10	400-8496	Single Use T Tap (can only drain once) Non sterile	Single use link drainage only	Bag of 10	£3.16
11/S/SU/DO	Ugo 11	400-8504	Single Use T Tap (can only drain once) Sterile pouch packaged	Attaching directly to catheter or single use link drainage	Box of 10	£5.12
12/S/D/L	Ugo 12	400-8512	Lever Tap (continuous drainage) Sterile pouch packaged	Attaching directly to catheter for up to 7 days with needle free sample port	Box of 10	£12.42
13/S/D/T	Ugo 13	400-8520	T Tap (continuous drainage) Sterile pouch packaged	Attaching directly to catheter for up to 7 days with needle free sample port	Box of 10	£12.42

Did you know?

The **Ugo Stand with Dignity** is available for all community patients using **Ugo 2L Drainage Bags**. Find out more on page 26.

Ugo Sheath

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3100	413-4110	Ugo Sheath Standard - 25mm	Box of 28	£47.14
3101	413-4102	Ugo Sheath Standard - 28mm	Box of 28	£47.14
3102	413-4128	Ugo Sheath Standard - 30mm	Box of 28	£47.14
3103	413-4136	Ugo Sheath Standard - 32mm	Box of 28	£47.14
3104	413-4144	Ugo Sheath Standard - 36mm	Box of 28	£47.14
3105	413-4151	Ugo Sheath Standard - 40mm	Box of 28	£47.14
3106	413-4169	Ugo Sheath Pop-on - 25mm	Box of 28	£47.14
3107	413-4177	Ugo Sheath Pop-on - 28mm	Box of 28	£47.14
3108	413-4185	Ugo Sheath Pop-on - 30mm	Box of 28	£47.14
3109	413-4193	Ugo Sheath Pop-on - 32mm	Box of 28	£47.14
3110	413-4201	Ugo Sheath Pop-on - 36mm	Box of 28	£47.14
3111	413-4219	Ugo Sheath Pop-on - 40mm	Box of 28	£47.14

Ugo Catheter Valve

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3000	388-2388	Ugo Catheter Valve	Box of 5	£11.23

Fixation Devices

Ugo Fix Gentle (catheter clip)

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3004	394-9054	Ugo Fix Gentle (catheter clip)	Box of 5	£13.00

Ugo Fix Catheter Strap

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3001	393-3371	Ugo Fix Catheter Strap Extra short – up to 35cm	Box of 5	£13.16
3002	393-3389	Ugo Fix Catheter Strap Short – up to 45cm	Box of 5	£13.16
3003	393-3397	Ugo Fix Catheter Strap Medium – up to 80cm	Box of 5	£14.53
3009	393-3405	Ugo Fix Catheter Strap Long – up to 130cm	Box of 5	£17.69
3010	393-3413	Ugo Fix Catheter Strap Extra Long – up to 180cm	Box of 5	£19.79

Ugo Fix Sleeve (leg bag holder)

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3005	394-0178	Ugo Fix Sleeve Small – 24cm-39cm	Box of 4	£7.66
3006	394-0160	Ugo Fix Sleeve Medium – 36cm-55cm	Box of 4	£7.66
3007	394-0152	Ugo Fix Sleeve Large – 40cm-70cm	Box of 4	£7.66
3008	394-0145	Ugo Fix Sleeve Extra Large – 65cm-90cm	Box of 4	£7.66
3027	409-6699	Ugo Fix Sleeve XXL - 75cm-105cm	Box of 4	£8.54

Ugo Fix Leg Bag Straps

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3011	400-8538	Ugo Fix Leg Bag Straps	10 pairs (1 x 75cm & 1 x 45cm)	£13.79

Multi Fix Gentle - universal tubing securement device

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3030	405-9200	Multi Fix Gentle (large)	Box of 10	£24.38
3031	405-9192	Multi Fix Gentle (small)	Box of 10	£23.86



The healthcare service grown
for you & your patients.

Our dedicated customer care agents and our specialist nurse team are on hand
to take the stress out of ordering and receiving healthcare appliances and medication.

Register your patients at:

☎ 0344 225 1518 ✉ register@myvyne.co.uk 🌐 www.myvyne.co.uk

Catheter Drainage and Management Pack

Ugo 4 Weeks - a 4-week supply of catheter drainage and fixation products.
Available through "Part IXB Incontinence Appliances"
on the Drug Tariff.

PR CODE	PIP CODE	PRODUCT NAME	LEG BAG DESCRIPTION	DT PRICE
3014	403-8824	Ugo 4 Weeks 1C	(500ml, short tube, lever tap)	£30.52
3015	403-8840	Ugo 4 Weeks 2C	(500ml, long tube, lever tap)	£30.52
3016	403-8865	Ugo 4 Weeks 3C	(500ml, short tube, T tap)	£30.52
3017	403-8873	Ugo 4 Weeks 4C	(500ml, long tube, T tap)	£30.52

Ugo Home - 7-day discharge pack

PR CODE	DESCRIPTION	INNER PACK	OUTER CASE
3018	Ugo Home 1C	See below	8 Bags
3019	Ugo Home 2C	See below	8 Bags
3020	Ugo Home 3C	See below	8 Bags
3021	Ugo Home 4C	See below	8 Bags

Training and education

We offer complementary product training for healthcare
professionals via video call or face to face in your workplace.*


In addition to this, a male and female catheterisation DVD and
supporting booklet is also available.

To find out more, email training@optimummedical.co.uk



*Face to face training may be subject to change dependent on government safety recommendations.
Where face to face meetings are not possible we can offer virtual training or support via video call.

Engaging product videos

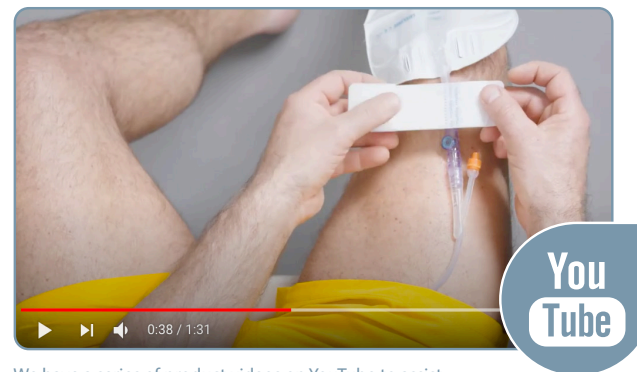

We have a series of product videos on YouTube to assist
patients and carers with how to use their products
with confidence. Follow our channel by searching for
'Optimum Medical'.

Did you know?

For added value, each **Ugo Urology** product is
provided with a comprehensive patient and carer
user guide. To request a user guide for any product
in the **Ugo Urology** range, please email
ugo@optimummedical.co.uk





The healthcare service grown
for you & your patients.

Our dedicated customer care agents and our specialist nurse team are on hand to take the stress out of ordering and receiving healthcare appliances and medication.

Register your patients at:

☎ 0344 225 1518 ✉ register@myvyne.co.uk 🌐 www.myvyne.co.uk

Catheterisation Jelly

| Available through “**Part IXA Appliances**” on the Drug Tariff.

OptiLube Active with Lidocaine 2%, and Chlorhexidine 0.05%

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
1160	366-2061	OptiLube Active 6ml syringe	Box of 10	£1.05/Syringe
1161	366-2079	OptiLube Active 11ml syringe	Box of 10	£1.10/ Syringe

OptiLube Active CHG Free with Lidocaine 2%

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
1167	404-0663	OptiLube Active CHG Free 6ml syringe	Box of 10	£1.05 / Syringe
1168	404-0671	OptiLube Active CHG Free 11ml syringe	Box of 10	£1.10 / Syringe

OptiLube Syringes without active ingredients

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
1125	373-4357	OptiLube 6ml syringe	Box of 25	£0.95/ Syringe
1126	373-4365	OptiLube 11ml syringe	Box of 25	£1.00/ Syringe

Cost-effective prescribing

Each **Optimum Medical** product is purpose-designed to reduce the risk of pain, trauma and infection, improving patient outcomes and reducing associated costs. According to EPIC 3¹, catheter associated urinary tract infections (CAUTI) cost the NHS £99 million per year. That’s £1968 per episode.

As well as saving money through reducing the risk of infection, all of our products are competitively priced. Competitor pricing* has been provided on the following pages to demonstrate cost savings available when switching to **Optimum Medical** products.

*accurate at the time of publication

Find out how much you could save

To demonstrate the cost savings that can be generated by switching to our products, we can provide you with a tailored cost savings analysis.

To receive your cost savings analysis,
email enquiries@optimummedical.co.uk.

Procedure Packs

Our procedure pack range has been designed in line with EPIC¹ guidelines and clinician input to support best practice aseptic catheterisation, promote efficiency, and provide a cost-effective solution in one handy pack.

ugo.TM13
drainable 2L bag

optilube.TM

optilube.TM
active pH5 free

500ml
400ml
300ml
200ml
100ml
approx. ml

In this range

Ugo Pro (catheterisation procedure pack)	14
The Ugo Box	16

ugopro.™

catheterisation procedure pack

Ugo Pro provides a pre-wrapped aseptic field and essential routine components for the safe and effective removal and/or insertion of a urinary catheter.

Ugo Pro should be used by a healthcare professional in accordance with local policies and procedures.

Ugo Pro Catheterisation procedure pack

PR CODE	PR NAME	PIP CODE	OUTER CASE	DT PRICE /PER UNIT
3400	Ugo Pro (catheterisation procedure pack)	414-6494	Case of 80	£2.23/per unit

UGO PRO CONTAINS:

- 1 Clear instructions for use.
- 2 1x outer wrapper – creates an aseptic field for the components to be placed upon.
- 3 1x disposable apron.
- 4 1x waste bag.
- 5 1x grey waterproof drape – protects the bed from spilt fluids during procedure. Assists with aseptic technique.
- 6 1x blue waterproof "O" drape – provides dignity during the procedure. Assists with aseptic technique.
- 7 1x 10ml empty syringe – to remove fluid from Foley catheter balloon prior to catheter removal.
- 8 6x non-woven swabs – to assist with removal and general catheterisation procedure technique.
- 9 5x non-woven balls – for cleansing catheter insertion site during procedure (once coated in saline).
- 10 1x 20ml saline cleansing pod (0.9% w/v sodium chloride) – to empty over non-woven balls for cleansing catheter insertion site.
- 11 1x clear tray with 2 compartments.
- 12 3x pairs of medium nitrile **Ugo Pro** gloves – assists with aseptic technique.

Key features:

- The unique design of the **Ugo Pro** allows freedom to choose which size/type of catheter, leg bag or catheter valve, and lubricant to use depending on patient or clinician preference.
- The **Ugo Pro** includes saline, which is recommended by EPIC3 as cleaning solution.
- Inside each pack you'll find detailed instructions for use, which are compatible with all local guidelines and policies/procedures.
- The **Ugo Pro** comes with 3 pairs of gloves, which is the required amount for best practice catheter removal and insertion.
- The drapes included in the **Ugo Pro** are different colours for easy differentiation.



THE ugo box.™

The **Ugo Box** is a fully equipped catheterisation kit for the insertion and removal of urethral catheters. Each box comes equipped with a choice of **Ugo Foley Catheter**, **Ugo Leg Bag** and **OptiLube** catheterisation jelly syringe.

The **Ugo Box** has been designed to support best practice aseptic catheterisation, promote efficiency, and provide a cost-effective solution in one handy pack.

The contents of the **Ugo Box** must be used in accordance with local policies and procedures and best practice guidance by suitably qualified and competent healthcare practitioners.

The Ugo Box

PR CODE	PIP CODE	UGO FOLEY CATHETER SIZE	UGO LEG BAG	11ML CATHETERISATION JELLY SYRINGE	DT PRICE / PER UNIT
3450	415-0660	12FR	Ugo 1C	OLA CHG Free*	£16.22
3451	415-0678	14FR	Ugo 1C	OLA CHG Free	£16.22
3452	415-0686	16FR	Ugo 1C	OLA CHG Free	£16.22
3453	415-0694	12FR	Ugo 1C	OptiLube	£16.22
3454	415-0702	14FR	Ugo 1C	OptiLube	£16.22
3455	415-0710	16FR	Ugo 1C	OptiLube	£16.22
3456	415-0728	12FR	Ugo 2C	OLA CHG Free	£16.22
3457	415-0736	14FR	Ugo 2C	OLA CHG Free	£16.22
3458	415-0744	16FR	Ugo 2C	OLA CHG Free	£16.22
3459	415-0769	12FR	Ugo 2C	OptiLube	£16.22
3460	415-0751	14FR	Ugo 2C	OptiLube	£16.22
3461	415-0777	16FR	Ugo 2C	OptiLube	£16.22

Please note: The **Ugo Foley Catheter** balloon size is 10ml.

*OptiLube Active CHG Free.

THE UGO BOX CONTAINS:

- 1 1X **Ugo Pro** (catheterisation procedure pack with 20ml saline and 10ml empty syringe)
- 2 1X **Ugo Foley Catheter** (all silicone)
- 3 1X **OptiPure** (10ml sterile water syringe)
- 4 1X 11ml catheterisation jelly syringe (OptiLube or OptiLube Active CHG Free)
- 5 1X **Ugo Leg Bag** (1C – short tube, 500ml, lever tap or 2C – long tube, 500ml, lever tap)
- 6 1X **Ugo Fix Gentle** (catheter clip)
- 7 1X pair **Ugo Fix Leg Bag Straps**



Key features:

- Cost effective.
- Recyclable packaging.
- Reduces waste - by providing all the items required in one box, this reduces stockpiling, over-prescribing and excessive packaging.
- Promotes efficiency - reduces time spent setting up the procedure, resulting in more time to spend with the patient.
- The fully comprehensive IFU supports both HCP and patient education of the procedure and the components included.
- Compliant with National Guidelines and evidence-based practices.
- Adheres to all local policies and procedures or techniques.
- 100% latex-free.





The healthcare service grown for you & your patients.

Our reliable, eco-friendly home delivery service is the most convenient way to get your patients' essential continence supplies and prescription medication delivered direct to their door.



Trusted & reliable



Stress-free service



With a personal touch



Discreet, eco-friendly delivery

Register your patients at:

www.myvyne.co.uk

email:

register@myvyne.co.uk

or call:

0344 225 1518




www.myvyne.co.uk

Drainage

A thoughtful urine drainage range based on patient user evidence.

Each is designed with a range of features to meet individual patient and clinical requirements in line with NICE² and EPIC¹ best practice guidelines.

ugo.TM13
drainable 2L bag

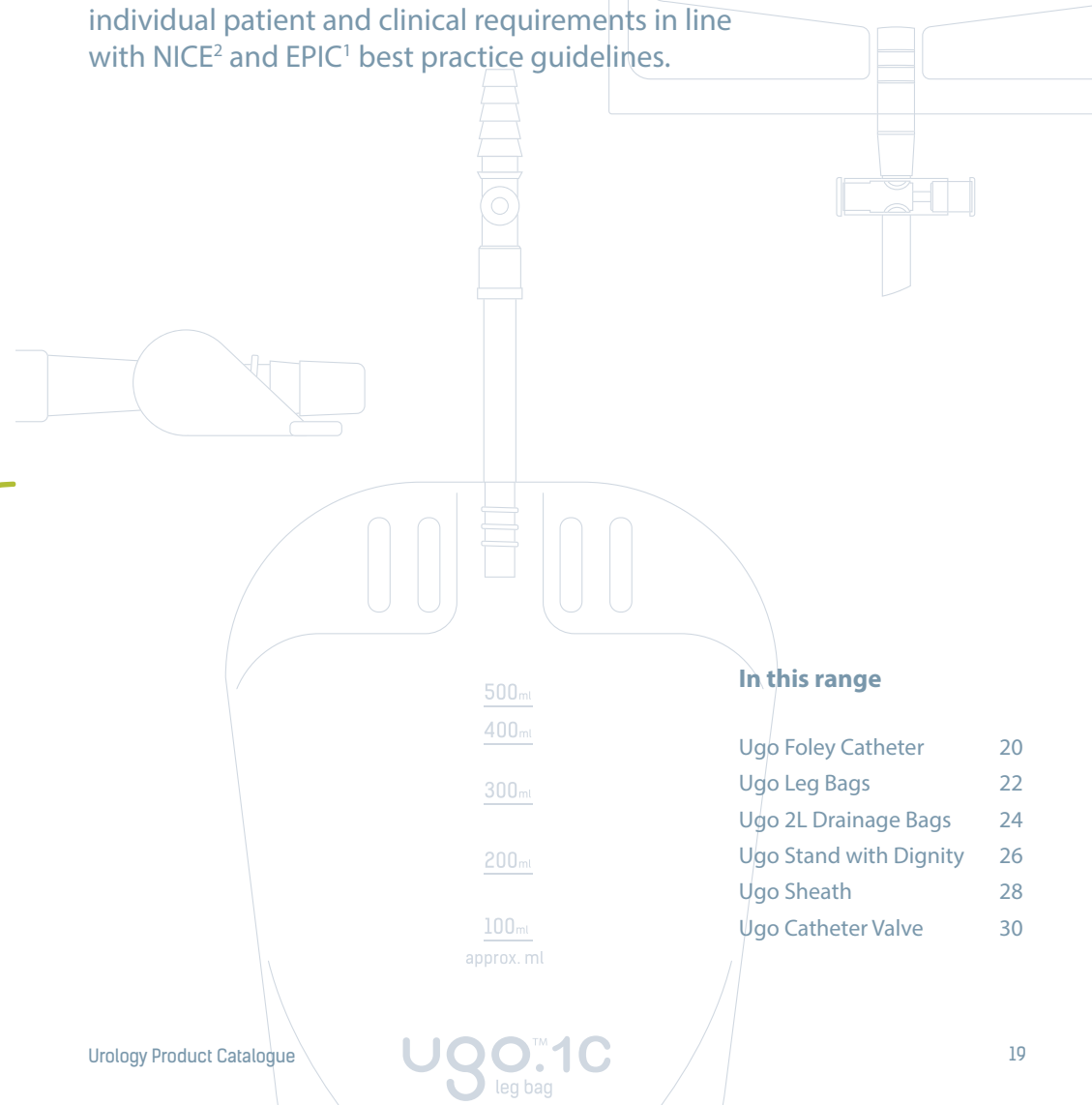

In this range

Ugo Foley Catheter	20
Ugo Leg Bags	22
Ugo 2L Drainage Bags	24
Ugo Stand with Dignity	26
Ugo Sheath	28
Ugo Catheter Valve	30

ugo.TM1C
leg bag

UgoTM Foley catheter

The **Ugo Foley Catheter** is designed for urethral catheterisation of the urinary bladder. When fitted by a qualified healthcare professional following a clinical assessment of a patient, it may be used for continuous bladder drainage, instillation of appropriate therapeutic agents into the bladder, or bladder irrigation.

Patients requiring long-term catheters should be continuously assessed and have them removed when not required.

Ugo Foley Catheter

PR CODE	PR NAME	PIP CODE	SIZE	DT PRICE
UFC12	Ugo Foley Catheter	414-3558	12FR	£6.00/per unit
UFC14	Ugo Foley Catheter	414-3541	14FR	£6.00/per unit
UFC16	Ugo Foley Catheter	414-3533	16FR	£6.00/per unit
UFC18	Ugo Foley Catheter	414-3525	18FR	£6.00/per unit

IMPORTANT:

- Catheters should always be inserted using an aseptic technique.
- Urinary catheters should NOT be left unsupported, as this can increase the risk of trauma and infection. It's advisable to use a fixation device, such as the **Ugo Fix Gentle** (catheter clip) or **Ugo Fix Catheter Strap**.

Find out more at
www.optimummedical.co.uk.

Key features:

- Made from 100% silicone.
- Latex-free.
- Integrated balloon.
- Standard length (420mm).
- Up to 12 weeks use.
- X-ray detectable line.
- Includes 2 catheter product record stickers to support best practice.





Supplied with comprehensive information and instructions, **Ugo Leg Bags** are safe, effective and simple to use. So patients in their own home, and carers in care homes, can use them with complete confidence when there is no healthcare professional on hand.

Ugo Leg Bag - Fabric backing – 1 pair of Ugo Fix Leg Bag Straps in every box

PR CODE	PR NAME	PIP CODE	BAG VOL	TUBE LENGTH	TAP DESIGN	INNER PACK	DT PRICE	SIMPLA/DT PRICE
1C/500/ST/L	Ugo 1C	400-8546	500ml	Short Tube	Lever Tap	Box of 10	£24.84	£31.19
2C/500/LT/L	Ugo 2C	400-8603	500ml	Long Tube	Lever Tap	Box of 10	£24.84	£31.19
3C/500/ST/T	Ugo 3C	400-8611	500ml	Short Tube	T Tap	Box of 10	£24.84	£31.19
4C/500/LT/T	Ugo 4C	400-8629	500ml	Long Tube	T Tap	Box of 10	£24.84	£31.19
5C/350/DI/L	Ugo 5C	400-8637	350ml	Direct Inlet	Lever Tap	Box of 10	£24.84	£30.67
6C/350/ST/L	Ugo 6C	400-8645	350ml	Short Tube	Lever Tap	Box of 10	£24.84	£30.67
7C/750/ST/L	Ugo 7C	400-8652	750ml	Short Tube	Lever Tap	Box of 10	£24.84	£31.65
8C/750/LT/L	Ugo 8C	400-8660	750ml	Long Tube	Lever Tap	Box of 10	£24.84	£31.65

Ugo Leg Bag - No fabric backing – 1 pair of Ugo Fix Leg Bag Straps in every pouch

PR CODE	PR NAME	PIP CODE	BAG VOL	TUBE LENGTH	TAP DESIGN	INNER PACK	DT PRICE
1H/500/ST/L	Ugo 1H	410-0517	500ml	Short Tube	Lever Tap	Box of 10	£24.53
2H/500/LT/L	Ugo 2H	410-0657	500ml	Long Tube	Lever Tap	Box of 10	£24.53
3H/500/ST/T	Ugo 3H	410-0665	500ml	Short Tube	T tap	Box of 10	£24.53
4H/500/LT/T	Ugo 4H	410-0673	500ml	Long Tube	T tap	Box of 10	£24.53

Ugo Pro Leg Bags – Fabric backing – includes disposable gloves and apron, and 1 pair of Ugo Fix Leg Bag Straps in every box

PR CODE	PR NAME	PIP CODE	BAG VOL	TUBE LENGTH	TAP DESIGN	INNER PACK	DT PRICE
3410	Ugo Pro 1C	415-9554	500ml	Short Tube	Lever Tap	Box of 10	£26.10
3411	Ugo Pro 2C	415-9547	500ml	Long Tube	Lever Tap	Box of 10	£26.10
3412	Ugo Pro 3C	415-9570	500ml	Short Tube	T tap	Box of 10	£26.10
3413	Ugo Pro 4C	415-9562	500ml	Long Tube	T tap	Box of 10	£26.10

Ugo Fix Leg Bag Straps

Not only are **Ugo Fix Leg Bag Straps** packaged with each box of leg bags, they're also available to order as a box of 10 pairs.

PR CODE	DESCRIPTION	INNER PACK	DT PRICE
3011	Ugo Fix Leg Bag Straps	10 pairs (1 x 75cm & 1 x 45cm)	£13.79



Key features of the range:

- Short, easy-to-remember names for quick and easy re-ordering
- Detailed user information printed on pouch in a clear and simple format
- Purpose measured soft silicone outlet tubing for kink-free link drainage ensuring patients can move around in bed without blocking or damaging the tubing
- Available in Lever Tap or T Tap drainage option, and with or without fabric backing to suit patient choice and clinical environment

For more information about **Ugo Leg Bags** you can email ugo@optimummedical.co.uk



Lever Tap



T Tap

Designed to be used with either an indwelling urethral or suprapubic catheter.

UgoTM 2L drainage bag

For anyone using overnight drainage bags, confidence in the right products is vital. Understanding this, we provide our **Ugo 2L Drainage Bags** with plenty of detailed information – to help patients or carers use them with complete confidence when there is no healthcare professional on hand.

Ugo 2L Drainage Bag

PR CODE	PR NAME	PIP CODE	TAP DESIGN	CLINICAL USE	INNER PACK	DT PRICE
9/NS/SU/DO	Ugo 9	400-8488	Non Drainable (tear corner to drain) Non sterile	Single use link drainage only	Bag of 10	£2.15
10/NS/SU/DO	Ugo 10	400-8496	Single Use T Tap (can only drain once) Non sterile	Single use link drainage only	Bag of 10	£3.16
11/S/SU/DO	Ugo 11	400-8504	Single Use T Tap (can only drain once) Sterile pouch packaged	Attaching directly to catheter or single use link drainage	Box of 10	£5.12
12/S/D/L	Ugo 12	400-8512	Lever Tap (continuous drainage) Sterile pouch packaged	Attaching directly to catheter for up to 7 days with needle free sample port	Box of 10	£12.42
13/S/D/T	Ugo 13	400-8520	T Tap (continuous drainage) Sterile pouch packaged	Attaching directly to catheter for up to 7 days with needle free sample port	Box of 10	£12.42

Ugo Stand with Dignity

The **Ugo Stand with Dignity** is available for all community patients using our **Ugo 2L Drainage Bags**.



Key features of the range:

- Thoughtful range designed to meet all patient needs in all clinical environments
- Short, easy-to-remember names for quick and easy re-ordering
- Non sterile 2L bag options provided in tamper-resistant, protective resealable bags

For more information about **Ugo 2L Drainage Bags** you can email ugo@optimummedical.co.uk

Designed to be used for link drainage and/ or continuous free drainage.



Thoughtful and creative, the **Ugo Stand with Dignity** is available with a dignity cover to hide the bag from view.

It's ideally suited to patients who want additional discretion when using their **Ugo 2L Drainage Bag**.

Receive your Ugo Stand with Dignity through Vyne!

All community patients using our **Ugo 2L Drainage Bags** automatically receive a **Ugo Stand with Dignity**.

All of the products in our **Ugo Urology** range are available through our home delivery service - **Vyne**.

To find out more, visit www.myvyne.co.uk or call **0344 225 1518**.



Ugo Stand with Dignity

PR CODE	DESCRIPTION	INNER PACK
3024	Ugo Stand with Dignity	Box of 1

*"I'm so grateful that Optimum Medical have created the **Ugo Stand with Dignity**. The removable dignity cover features a 'viewing strip' so you can see how full your bag is getting without having to have anything personal on show. Thank you for giving me back my privacy and dignity – this is so important to me."*

Margaret Balch, Birmingham, UK

Key features:

- Dignity cover provides discreet urine drainage
- Purpose designed to reduce the risk of infection by ensuring the tap doesn't touch the ground
- Variety of usage options





The **Ugo Sheath** has been designed to transform the management of male urinary incontinence, ideal for use as an alternative to incontinence pads. Each box contains 4 weekly boxes of 7 sheaths - giving you a convenient monthly supply.

Ugo Sheath

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3100	413-4110	Ugo Sheath Standard - 25mm	Box of 28	£47.14
3101	413-4102	Ugo Sheath Standard - 28mm	Box of 28	£47.14
3102	413-4128	Ugo Sheath Standard - 30mm	Box of 28	£47.14
3103	413-4136	Ugo Sheath Standard - 32mm	Box of 28	£47.14
3104	413-4144	Ugo Sheath Standard - 36mm	Box of 28	£47.14
3105	413-4151	Ugo Sheath Standard - 40mm	Box of 28	£47.14
3106	413-4169	Ugo Sheath Pop-on - 25mm	Box of 28	£47.14
3107	413-4177	Ugo Sheath Pop-on - 28mm	Box of 28	£47.14
3108	413-4185	Ugo Sheath Pop-on - 30mm	Box of 28	£47.14
3109	413-4193	Ugo Sheath Pop-on - 32mm	Box of 28	£47.14
3110	413-4201	Ugo Sheath Pop-on - 36mm	Box of 28	£47.14
3111	413-4219	Ugo Sheath Pop-on - 40mm	Box of 28	£47.14

Achieving the perfect fit is key for a good experience using urinary sheaths.

For assistance with sizing, download the **Ugo Sheath** Measuring Guide at www.optimummedical.co.uk

Patients should only use the **Ugo Sheath** if it has been recommended to do so by a healthcare professional and the patient (or their carer) have the ability to fit and change it at regular intervals.



Key features:

- Made of silicone
- 100% latex-free
- Easy to apply
- Discreetly packaged
- Each box is fully recyclable
- Environmentally friendly with reduced plastic
- Pubic hair protector included for more comfortable application
- Resealable travel pouch provided with every box





Easy to use, discreet and comfortable to wear, and designed in detail for safety and security. The **Ugo Catheter Valve** gives patients the confidence they need to carry on living independent lives. Using a catheter valve as an alternative to urine drainage bags may improve experience and outcomes for suitable patients.

The **Ugo Catheter Valve** is designed to be used with either an indwelling urethral or suprapubic catheter.


Ugo Catheter Valve

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE	FLIP FLO/DT PRICE
3000	388-2388	Ugo Catheter Valve	Box of 5	£11.23	£13.48



Key features:

- Safety lever feature to prevent accidental opening
- Universal stepped connector for secure fitting to all catheter materials
- Lever handle designed to ensure easy single handed operation

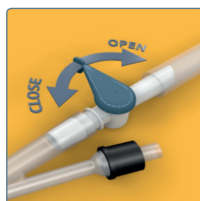

*"I use the **Ugo Catheter Valve** with my suprapubic catheter. It grips extremely well, and doesn't slip like some other valves which I have tried."*

Christine P, London, UK

Fixation Devices

Purpose designed fixation devices for safety and effectiveness to reduce the risk of trauma, pain and infection.

NICE² guidelines state that 'best practice suggests catheters are secured to avoid trauma'.



In this range

Ugo Fix Gentle (catheter clip)	32
Ugo Fix Catheter Strap	34
Ugo Fix Sleeve (leg bag holder)	36
Multi Fix Gentle	38



Secure and reliable, the **Ugo Fix Gentle** re-adhesive gel pad uses soft silicone technology to ensure minimal trauma when it's removed. It's also easy to reposition and leaves no sticky residue.

Fixing in place comfortably, it lets patients move around freely and safely, with reduced pulling and rubbing on the bladder neck and urethra.

The **Ugo Fix Gentle** provides safe, effective positioning of two-way indwelling urinary catheters at the bifurcation of the catheter. It helps secure urethral or suprapubic catheters that are attached to catheter valves or drainage bags.

Ugo Fix Gentle (catheter clip)

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE	STATLOCK/DT PRICE
3004	394-9054	Ugo Fix Gentle	Box of 5	£13.00	£13.02

*"My wife has been using the **Ugo Fix Gentle** catheter clip for approximately two weeks. We have found that there is no skin damage which she was getting with other products. She loves the rotating clip feature – it ensures the leg bag is always in the right position when it's attached to her leg or when she's laying on the bed."*

Dave Wickenden, Kent, UK

Key features:

- Gentle re-adhesive pad with silicone technology
- Easy to apply, remove and reposition
- No skin preparation required before application
- No sticky residue upon removal
- Suitable for re-application over the same skin area
- Revolving clip to allow natural movement
- Ideally complemented by the **Ugo Fix Catheter Strap** and **Ugo Fix Sleeve**



With silicone technology for atraumatic patient experience



Secure, reliable and simple to use. The **Ugo Fix Catheter Strap** provides comfortable support, giving your patients the freedom to move around freely and safely.

Ugo Fix Catheter Strap

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE	SIMPLA GSTRAP/ DT PRICE
3001	393-3371	Ugo Fix Catheter Strap Extra short – up to 35cm	Box of 5	£13.16	-
3002	393-3389	Ugo Fix Catheter Strap Short – up to 45cm	Box of 5	£13.16	Adult - £14.84
3003	393-3397	Ugo Fix Catheter Strap Medium – up to 80cm	Box of 5	£14.53	Abdominal £16.37
3009	393-3405	Ugo Fix Catheter Strap Long – up to 130cm	Box of 5	£17.69	-
3010	393-3413	Ugo Fix Catheter Strap Extra Long – up to 180cm	Box of 5	£19.79	-



To assist with patient's daily wash and wear cycle, we provide a laundry bag in each box of **Ugo Fix Catheter Straps**.

*"The **Ugo Fix Catheter Strap** made life more comfortable for my husband while he had a catheter. The previous straps he used cut into his leg whereas the **Ugo Fix Catheter Strap** was wider and longer, so we could easily cut to size, and we found the fastening efficient and secure. They washed well and dried quickly".*

Nicola Bradbury, Reading, UK



Key features:

- All five lengths are fully adjustable and can be cut to fit
- Silicone technology for extra firm grip to skin
- Reusable and washable (provided with laundry bag)
- Comprehensive patient and carer user guide includes range of fixation options
- Ideally complemented by the **Ugo Fix Sleeve** and **Ugo Fix Gentle**

Designed to support urethral catheters, suprapubic catheters and sheath systems. Fixes on either the catheter or leg bag tubing.



Durable, comfortable and extra supportive.
The **Ugo Fix Sleeve** is designed to support leg bags attached to indwelling catheters and sheaths.

Ugo Fix Sleeve (leg bag holder)

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE	AQUASLEEVE / DT PRICE
3005	394-0178	Ugo Fix Sleeve Small – 24cm-39cm	Box of 4	£7.66	£8.73
3006	394-0160	Ugo Fix Sleeve Medium – 36cm-55cm	Box of 4	£7.66	£8.73
3007	394-0152	Ugo Fix Sleeve Large – 40cm-70cm	Box of 4	£7.66	£8.73
3008	394-0145	Ugo Fix Sleeve Extra Large – 65cm-90cm	Box of 4	£7.66	£8.55
3027	409-6699	Ugo Fix Sleeve XXL – 75cm-105cm	Box of 4	£8.54	-



To assist with patient's daily wash and wear cycle, we provide a laundry bag in each box of **Ugo Fix Catheter Straps**.

*"The **Ugo Fix Sleeve** leg bag holder has given my partner enough confidence to wear shorts this summer - something he usually does but was feeling he might not be able to do with a leg bag. The **Ugo Fix Sleeve** has made this option available to him and given him the greatest confidence boost!"*

Martin Otter, Devon.



Key features:

- No need to disconnect leg bag from catheter when changing sleeve
- External seam to protect skin and increase comfort
- Colour coded for easy reordering
- Reusable and washable (provided with a laundry bag)
- Ideally complemented by the **Ugo Fix Gentle** and **Ugo Fix Catheter Strap**

Designed for supporting leg bags attached to indwelling catheters and sheaths



Secure, reliable and gentle. The **Multi Fix Gentle** is a universal securement device for medical tubings and urinary catheters. It has been developed to provide comfortable fixation, giving the patient the confidence to move around freely.

The **Multi Fix Gentle** benefits from re-adhesive soft silicone technology to allow easy application, removal and repositioning with minimal trauma and no sticky residue.

The **Multi Fix Gentle** is available in 2 sizes to suit a range of applications and tubing diameters, and each one can be used for up to 7 days.


Multi Fix Gentle - universal tubing securement device

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
3030	405-9200	Multi Fix Gentle (large)	Box of 10	£24.38
3031	405-9192	Multi Fix Gentle (small)	Box of 10	£23.86



Catheter Drainage and Management Packs

Smart management systems created to reduce product shortages and improve efficiency.

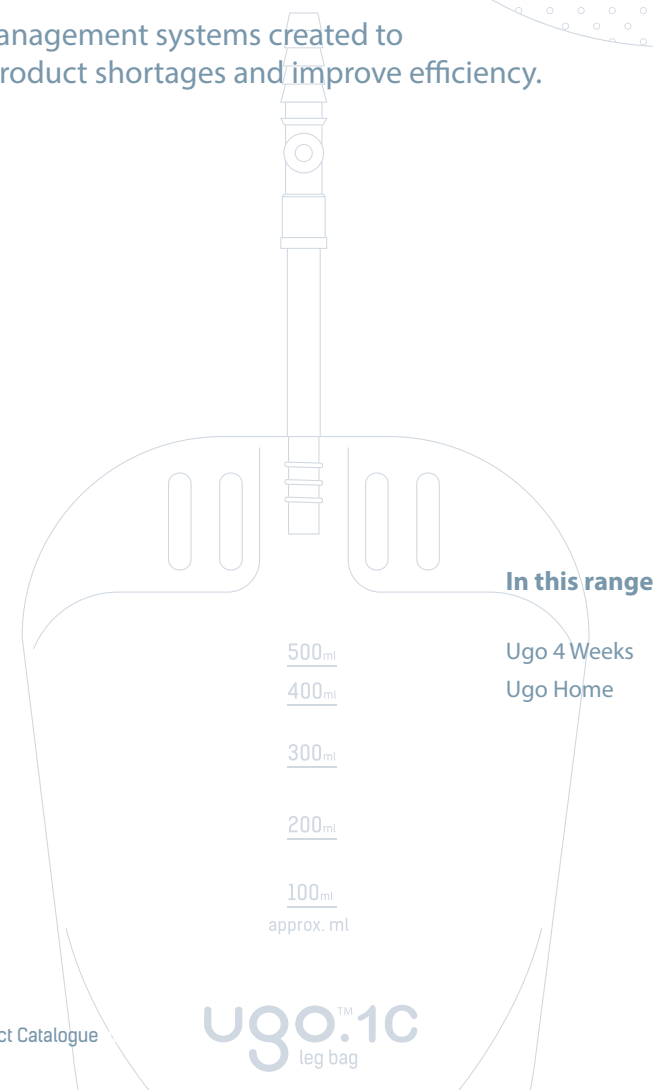

In this range

500ml

Ugo 4 Weeks

40

400ml

Ugo Home

42

300ml

200ml

100ml

approx. ml

ugo.1c
leg bag



The **Ugo 4 Weeks** provides a 4 week supply of essential catheter drainage and fixation products, all contained in a compact, discreet box - perfect for storage in patient's bedrooms or bathrooms.

The **Ugo 4 Weeks** ensures an organised supply with just one code per box, making re-ordering simple. It's easy and efficient to prescribe, saving time for everyone involved. Helping clinicians and patients to manage their care, the **Ugo 4 Weeks** effectively prevents product shortages and improves efficiency. It also improves clarity on products required and managing a link drainage system effectively.

Ugo 4 Weeks

PR CODE	PIP CODE	DESCRIPTION	DT PRICE
3014	403-8824	Ugo 4 Weeks 1C*	£30.52
3015	403-8840	Ugo 4 Weeks 2C*	£30.52
3016	403-8865	Ugo 4 Weeks 3C*	£30.52
3017	403-8873	Ugo 4 Weeks 4C*	£30.52

* **Ugo Leg Bag 1C/ 2C/ 3C/ 4C** depending on patient preference.
For further information, please refer to **Ugo Leg Bag** table on page 22.

Find out more: www.ugo4weeks.co.uk

• **ONE BOX** • **ONE CODE** • **ONE PRESCRIPTION** • **ONE MONTH**

Each box contains:

- 4 x fabric backed **Ugo Leg Bags** (1C/2C/3C/4C)*
- 28 x **Ugo 10 2L Bags** with single use T Tap
- 1 x **Ugo Hanger**
- 4 x pairs of **Ugo Fix Leg Bag Straps**
- 4 x **Ugo Fix Gentle** catheter clips
- Patient and Carer User Guides





Convenient and compact catheter drainage and management system, containing one week's supply of products. **Ugo Home** is ideal for use when patients are discharged from hospital.

Each product inside the **Ugo Home** is available to order separately.

Ugo Home

PR CODE	DESCRIPTION	INNER PACK	OUTER CASE
3018	Ugo Home 1C	See below	8 Bags
3019	Ugo Home 2C	See below	8 Bags
3020	Ugo Home 3C	See below	8 Bags
3021	Ugo Home 4C	See below	8 Bags

Contains:

- 1 x fabric backed **Ugo Leg Bag** (Ugo 1C / Ugo 2C / Ugo 3C / Ugo 4C)
- 7 x **Ugo 10 2L Bags** with single use T Tap
- 1 x **Ugo Hanger**
- 1 x pair of **Ugo Fix Leg Bag Straps**
- 1 x **Ugo Fix Gentle** (catheter clip)
- Patient and carer user guides




www.optimummedical.co.uk

Catheterisation Jelly

A range of catheterisation jelly, supporting patient-centred care by offering a choice.

NICE² guidelines state that 'urethral trauma and discomfort will be minimised by using an appropriate sterile, single-use lubricant or anaesthetic gel'.

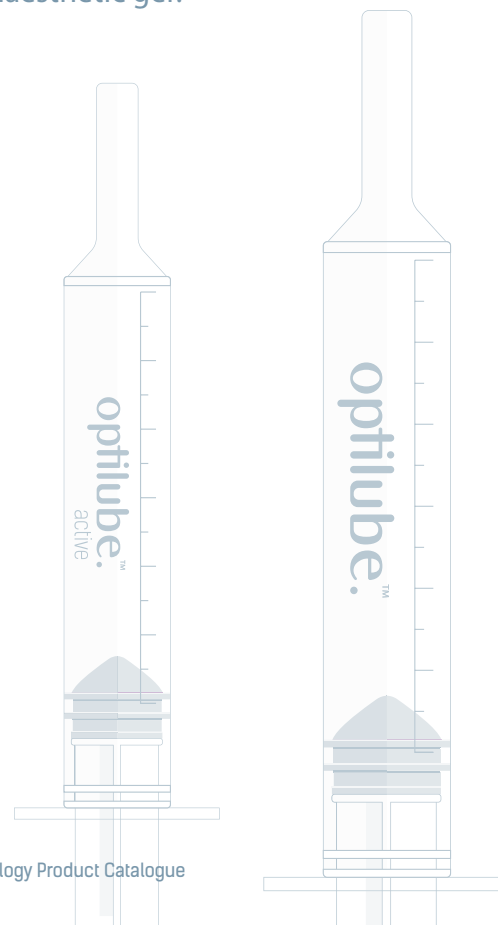

In this range

OptiLube Syringes	44
OptiLube Active	44
OptiLube Active CHG Free	44

Catheterisation Jelly

We understand the importance of having a choice.

This is why we offer a range of catheterisation jelly, supporting clinicians in practicing patient-centred care by selecting the gel best suited to each individual patient.

Available in 6ml (female) and 11ml (male) volumes, all of our pre-filled, sterile pouch packaged syringes help ensure the full length of the urethra is coated in lubricant and an aseptic procedure is achieved.

optilube.TM
active

OptiLube Active with Lidocaine 2%, and Chlorhexidine 0.05%

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
1160	366-2061	OptiLube Active 6ml syringe	Box of 10	£1.05/Syringe
1161	366-2079	OptiLube Active 11ml syringe	Box of 10	£1.10/ Syringe

optilube.TM
active CHG free

OptiLube Active CHG Free with Lidocaine 2%

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
1167	404-0663	OptiLube Active CHG Free 6ml syringe	Box of 10	£1.05 / Syringe
1168	404-0671	OptiLube Active CHG Free 11ml syringe	Box of 10	£1.10 / Syringe

optilube.TM

OptiLube Syringes without active ingredients

PR CODE	PIP CODE	DESCRIPTION	INNER PACK	DT PRICE
1125	373-4357	OptiLube 6ml syringe	Box of 25	£0.95/ Syringe
1126	373-4365	OptiLube 11ml syringe	Box of 25	£1.00/ Syringe

To conduct an evaluation of our catheterisation jelly range, please email enquiries@optimummedical.co.uk.



Key features of the range:

- Intuitively designed syringe format enables controlled application of the gel
- Tip design helps to protect the delicate tissue and reduce the risk of suction when inserted into the urethral opening
- Latex-free

Clinician Support

We offer complementary product training for healthcare professionals via video call or face to face in your workplace.*

In addition to this, a male and female catheterisation DVD and supporting booklet is also available.

Find out more:

Email: training@optimummedical.co.uk

Call: 0113 263 3849



*Face to face training may be subject to change dependent on government safety recommendations. Where face to face meetings are not possible we can offer virtual training or support via video call.

*"Our staff found your training very beneficial. It was delivered in an engaging way which assisted our staff in their learning. The staff were really taken with the **Ugo Urology** products and can see how these will be of a clear benefit to our catheter users."*

Lisa Parry, Park View Care Home



Keep up to date with the latest news at **Optimum Medical** by following us on Twitter or LinkedIn [@OptimumMedical](#)

Patient Support

At **Optimum Medical**, we pride ourselves on providing patients and carers with the highest levels of support. Whether patients like having access to printed literature, or prefer to go online, we have an option to suit everyone!

User guides


Each **Ugo Urology** product is provided with its own comprehensive patient and carer user guide. This can help to provide additional reassurance when using our products, particularly between nurse visits. Digital versions are also available to view and download on our website www.optimummedical.co.uk.



Visit our YouTube channel

Our YouTube channel features a series of short yet informative videos, showing how to use the products in our **Ugo Urology** range.

Find us on YouTube by searching for '[Optimum Medical](#)'.

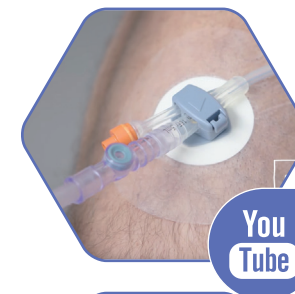

Find us on social media

For patients who are active on social media, they can keep up to date with all of the latest news from **Optimum Medical** by following us on Facebook.

Find us by searching [@OptimumMedicalSolutions](#)



References:

1. Loveday HP, Wilson JA, Pratt RJ et al. (2013) Epic3: National Evidence-Based guidelines for Preventing Healthcare-Associated Infections in NHS Hospitals in England. *Journal of Hospital Infection*; 86S1: S1-S70.2
2. National Institute for Health and Care Excellence (2012) Guidance 139. *Infection: Prevention and Control of Healthcare-associated Infections in Primary and Community Care*. London: National Institute for Health and Care Excellence; available at www.nice.org.uk

NOTES

