# optipre.™

## alcohol wipes

REF 6100

REF 6103

REF 61004

- ◆ Wipes for disinfecting hard surfaces.
- ◆ 75% alcohol.
- ◆ Removes potentially harmful bacteria, viruses and fungi.

### | GBR | OptiPre Alcohol wipes

Our disposable OptiPre Alcohol Wipes, disinfect hard surfaces in a (s)wipe, removing potentially harmful bacteria, viruses and fungi. Use alcohol wipes safely. Always read the label and product information before use.

**DANGER:** • Highly flammable liquid and vapour. • Causes serious eye irritation.

**PRECAUTION:** • If contact with eyes occurs, rinse cautiously with water for several minutes. Remove contact lenses, if present, and easy to do. Continue rinsing. • If medical advice is needed, have product container or label at hand. • Wash hands and exposed skin thoroughly after handling. • Keep away from heat, hot surfaces, sparks, open flames, and other sources of ignition. No smoking. • Keep out of reach of children.

**STORAGE AND DISPOSAL:** • Use within 60 days of opening (record open date in space provided). • Do not use if the wipe appears dry. • Store in a cool, dry place, away from direct sunlight and ignition sources. • Keep packaging tightly closed when stored to avoid drying out. • Do not flush. • Dispose of wipes and container in an appropriate bin.

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