

vue.TM

ultrasound gel



LATEX FREE




EU Importer:
Optimum Medical Solutions GmbH,
c/o Warth & Klein Grant Thornton AG,
Ulmenstraße 37-39,
60325 Frankfurt am Main, Germany
Email: enquiries@optimummedical.de

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
66386 St. Ingbert Germany

1157-1159-FU-Sept21-V1.0

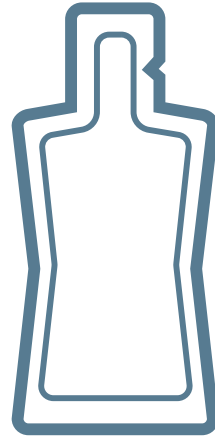

Instructions for use



optimummedical®

 **Optimum Medical Solutions Ltd**
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

BRAND: Vue Ultrasound Gel**PRODUCT DESCRIPTION:**

Vue Ultrasound Gel is a non-sterile, water-based gel.

INGREDIENTS/COMPOSITION:

- Deionised water - Carbomer - Propylene glycol
- Triethanolamine - Benzyl alcohol - Methylchloroisothiazolinone - Methylisothiazolinone

INTENDED PURPOSE:

Vue Ultrasound Gel is a medical device intended to be used for the transmission of ultrasonic waves during low risk, general examinations on intact skin, not relating to a procedure or immediately prior to a procedure.

INDICATIONS FOR USE / INTENDED USER:

Vue Ultrasound Gel is a medical device intended to be used by Healthcare Professionals when performing ultrasound.

CONTRAINDICATIONS:

Do not use for invasive procedures or prior to invasive procedures on the same site (for example needle insertions, aseptic technique).

Do not use for contact with mucous membrane (for example for transrectal or transvaginal procedures).

Do not use on immunocompromised, neonatal, or critically ill hospitalised patients.

Do not use if sensitive or allergic to any of the ingredients. →

Do not use on broken or damaged skin.

Not suitable for use with a defibrillator, ECG or laser treatment.

WARNINGS AND PRECAUTIONS:

It is recommended that **Vue Ultrasound Gel** is used in line with local healthcare policies and procedures.

Ensure that **Vue Ultrasound Gel** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.

Do not use if the bottle is damaged, soiled or the seal is broken prior to first use.

Do not refill. Reuse or refilling of the bottle may result in infection and/or cross-contamination.

Do not decant into smaller bottles as there is a risk of contamination.

Discard bottles containing **Vue Ultrasound Gel** in any of the following circumstances:

- if it has been in use for one week or more;
- if the bottle or contents appear dirty, old, or are otherwise soiled or damaged;
- if the nozzle comes into contact with the patient, person administering the gel or instruments.

For external use only.

Not to be ingested.

Avoid contact with the eyes.

Keep out of reach of children.

Vue Ultrasound Gel is Latex Free.

UNDESIRABLE SIDE EFFECTS:

There are no known side effects associated with using **Vue Ultrasound Gel**, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include

redness, itchiness, or blistering. If this occurs, stop using **Vue Ultrasound Gel** and ensure patient records are updated.

Any serious incident or malfunction that has occurred in relation to the **Vue Ultrasound Gel** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is benefited.

HOW TO USE:

Prior to use, label the bottle with the date first opened in the space provided.

Close cap between uses.

Once opened, use within seven days (see warnings and precautions).

Always wash and dry your hands before and after using **Vue ultrasound Gel** and apply a clean pair of gloves on every occasion to avoid cross infection.

1. Clean the patient's skin prior to applying **Vue Ultrasound Gel**.
2. Follow the instructions supplied by the manufacturer of the equipment being used to perform the medical ultrasound procedure.
3. Apply the required amount of **Vue Ultrasound Gel** to the area of skin/equipment.
4. Following the procedure, remove any remaining ultrasound gel from the patient's skin using personal hygiene wipes or wash off using patient dry wipes and soap and water.
5. Dry the patient's skin thoroughly afterwards with paper towels.

STORAGE AND HANDLING:

Keep dry and out of direct sunlight.

Unopened packs have a 3-year shelf life.

Do not use after expiry date.

DISPOSAL:

Vue Ultrasound Gel must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

250ml bottle - clear gel - 1157

250ml bottle - blue gel - 1157B

1l bottle - clear gel - 1159

1l bottle - blue gel - 1159B

MANUFACTURER INFORMATION:

Optimum Medical Solutions Ltd, Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.

Tel: +44 (0) 845 643 5479

Email: enquires@optimummedical.co.uk

Website: www.optimummedical.co.uk

AUTHORISED REPRESENTATIVE:

MT PromedT Consulting GmbH, Altenhofstrasse 80, 66386 St. Ingbert, Germany.

Tel: +49 (0) 689 458 1020

Email: info@mt-procons.com

EU IMPORTER INFORMATION:

Optimum Medical Solutions GmbH, c/o Warth & Klein Grant Thornton AG, Ulmenstraße 37-39, 60325

Frankfurt am Main, Germany.

Email: enquiries@optimummedical.de