This IFU can be recycled.

MT Promedt Consulting GmbH,  
Altenhofstrasse 80,  
66386 St. Ingbert, Germany

Decomplex AG,  
Freiburgstrasse 3,  
3010 Bern, Switzerland


EU Importer:  
Optimum Medical Solutions GmbH,  
c/o Grant Thornton AG,  
Senckenberganlage 19,  
60325 Frankfurt am Main, Germany.  
Email: [enquiries@optimummedical.de](mailto:enquiries@optimummedical.de)

OL-IFU-Feb22-V16.0

# optilube.™

sterile lubricating jelly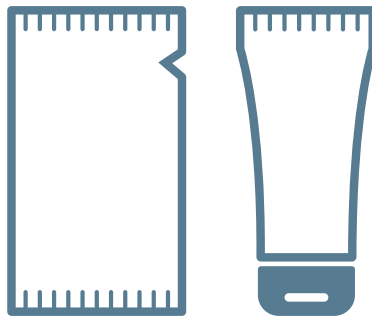

## Instructions for use

optimummedical®

 **Optimum Medical Solutions Ltd**  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom  
T: +44(0) 845 643 5479  
[enquiries@optimummedical.co.uk](mailto:enquiries@optimummedical.co.uk)  
[www.optimummedical.co.uk](http://www.optimummedical.co.uk)

**BRAND: OptiLube****COMPOSITION:**

**OptiLube** is a sterile, transparent, water-soluble lubricating jelly with a pH range of 5.0 – 7.0.

- Purified water • Polyethylene Glycol (PEG) • Glycerine
- Carbomer • Sodium hydroxide • 4-Hydroxybenzoic acid esters

**INTENDED PURPOSE:**

**OptiLube** is a medical device that is a water-soluble, sterile lubricating jelly intended for the smooth insertion of medical devices, and examinations, reducing the risk of trauma during procedures, including:

- Insertion of catheters • Endoscopy
- Gynaecological examinations • Rectal examinations • Cystoscopy

**INTENDED USER:**

The intended users of **OptiLube** are patients, their Carers and/or Healthcare Professionals.

**CONTRAINDICATIONS**

Do not use **OptiLube** if you are sensitive or allergic to any of the device ingredients.

**WARNINGS AND PRECAUTIONS:** It is recommended that **OptiLube** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.

Ensure that **OptiLube** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it. **OptiLube** is Latex Free. Do not use an **OptiLube** sachet or tube if it is damaged, or seal is broken. Keep out of reach of children. Apply the required amount of lubrication necessary for the procedure. Not for use in the eye.

Re-use of this medical device may result in infection and/or cross-contamination. Resterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.

**UNDESIRABLE SIDE EFFECTS:**

There are no known side effects associated with using **OptiLube**, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itching, or blistering. If this occurs, stop using **OptiLube** and consult your Healthcare Professional.

Any serious incident or malfunction that has occurred in relation to **OptiLube** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

**HOW TO USE:**

Consult your Healthcare Professional for advice before using **OptiLube**.  
Do not use if packaging is damaged or unintentionally opened prior to use.

**SACHETS:** Sterile and single-use. Tear sachet at one end and apply as required.  
**Note:** **OptiLube** 20g sachet (1119) is supplied double wrapped, therefore it is advised that you use an aseptic technique to open.

**TUBES:** Sterile until opened. Remove cap and peel foil from end of tube. Apply as necessary. Close cap after use. Use within 3 months of opening. Discard sooner if the tube becomes contaminated.

**REACH:** Sterile and single-use. Snap nozzle tip to open and apply as necessary. Dispose of after use.

**STERILE DEVICES:**

**OptiLube** is supplied sterile.

**OptiLube** is sterilised by gamma after the packaging process.  
Do not re-sterilise.

**STORAGE AND HANDLING:**

Keep dry and out of direct sunlight.  
Reapply cap in between use.  
Unopened packs have a 5 year shelf life.  
Do not use after expiry date.

**DISPOSAL:**

**OptiLube** must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

**PRESENTATION:**

<b>Tubes:</b>	5g, 10g, 42g, 56.5g, 82g, 113g
<b>Sachets:</b>	2.7g, 3g, 5g, 20g
<b>Reach:</b>	5g, 15g

**MANUFACTURER INFORMATION:**

Optimum Medical Solutions Ltd,  
Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.  
Tel: +44 (0) 845 643 5479  
Email: enquires@optimummedical.co.uk  
Website: www.optimummedical.co.uk

**AUTHORISED REPRESENTATIVE:**

MT Promedit Consulting GmbH,  
Altenhofstrasse 80, 66386 St. Ingbert, Germany.

**EU IMPORTER INFORMATION:**

Optimum Medical Solutions GmbH,  
c/o Grant Thornton AG, Senckenberganlage 19,  
60325 Frankfurt am Main, Germany. Email: enquires@optimummedical.de

**SWITZERLAND AUTHORISED REPRESENTATIVE:**

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