REF 1165A

REF 1165C

LATEX FREE

MT Promedt Consulting GmbH,  
Altenhofstrasse 80,  
66386 St. Ingbert, Germany

EU Importer:  
Optimum Medical Solutions GmbH,  
c/o Grant Thornton AG,  
Senckenberganlage 19,  
60325 Frankfurt am Main, Germany.  
Email: [enquiries@optimummedical.de](mailto:enquiries@optimummedical.de)

optimummedical®

**Optimum Medical Solutions Ltd**

Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom


T: +44(0) 845 643 5479

[enquiries@optimummedical.co.uk](mailto:enquiries@optimummedical.co.uk)

[www.optimummedical.co.uk](http://www.optimummedical.co.uk)

1165A-1165C-IFU-Mar22-V1.0

vue.<sup>TM</sup>  
ultrasound gel

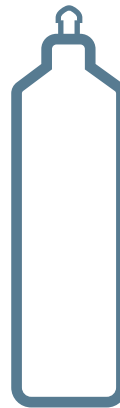Instructions for use

**PRODUCT DESCRIPTION:**

Vue Ultrasound Gel is a non-sterile, water-based gel.

**INTENDED PURPOSE:**

Vue Ultrasound Gel is a medical device intended to be used for the transmission of ultrasonic waves during low risk, general examinations on intact skin, not relating to a procedure or immediately prior to a procedure.

**INDICATIONS FOR USE / INTENDED USER:**

Vue Ultrasound Gel is a medical device intended to be used by Healthcare Professionals when performing ultrasound.

**CONTRAINDICATIONS:**

Do not use for invasive procedures or prior to invasive procedures on the same site (for example needle insertions, aseptic technique).

Do not use for contact with mucous membrane (for example for transrectal or transvaginal procedures).

Do not use on immunocompromised, neonatal, or critically ill hospitalised patients.

Do not use if sensitive or allergic to any of the ingredients.

Do not use on broken or damaged skin.

Not suitable for use with a defibrillator, ECG or laser treatment.

**WARNINGS AND PRECAUTIONS:**

It is recommended that Vue Ultrasound Gel is used in line with local healthcare policies and procedures. Ensure that Vue Ultrasound Gel is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.

Do not use if the bottle is damaged, soiled or the seal is broken prior to first use.

Discard reusable bottles in any of the following circumstances:

- If has been in use for one week or more.
- When the bulk container used to refill the bottle is replaced.
- If the bottle or contents appear dirty, old, or are otherwise soiled or damaged.
- When the expiry date of the bulk container is reached.
- If the nozzle comes into contact with the patient, person administering the gel or instruments.

Do not attempt to clean the inside of bottles, for example to extend their use.

Avoid touching the openings of the dispensing container and the bottle.

Fill reusable bottles with a dispensing device, such as a pump.

Do not refill bottles until they are empty (or as near to empty as practical).

Discard gel pumps that show any sign of damage or if visibly soiled.

Do not decant Vue Ultrasound Gel into bottles or containers of a different product.

Do not refill pre-filled bottles.

For external use only.

Not to be ingested.

Avoid contact with the eyes.

Keep out of reach of children.

Vue Ultrasound Gel is Latex Free.

**UNDESIRABLE SIDE EFFECTS:**

There are no known side effects associated with using Vue Ultrasound Gel, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity

include redness, itchiness, or blistering. If this occurs, stop using Vue Ultrasound Gel and ensure patient records are updated. Any serious incident or malfunction that has occurred in relation to the Vue Ultrasound Gel should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

**HOW TO USE:**

Prior to use, label the 5l<sup>st</sup> soft container with the date first opened on the space provided. Label the 250ml<sup>th</sup> bottle with the date that it was refilled. Once opened, use within seven days (see warnings and precautions). Close cap between uses.

Always wash and dry your hands before and after using Vue Ultrasound Gel and apply a clean pair of gloves on every occasion to avoid cross infection.

1. Clean the patient's skin prior to applying Vue Ultrasound Gel.
2. Follow the instructions supplied by the manufacturer of the equipment being used to perform the medical ultrasound procedure.
3. Apply the required amount of Vue Ultrasound Gel to the area of skin/equipment.
4. Following the procedure, remove any remaining ultrasound gel from the patient's skin using personal hygiene wipes or wash off using patient dry wipes and soap and water.
5. Dry the patient's skin thoroughly afterwards with paper towels.

**STORAGE AND HANDLING:**

Store between 5°C and 30°C.

Keep dry and out of direct sunlight.

Unopened packs have a 3-year shelf life.

Do not use after expiry date.

**DISPOSAL:**

Vue Ultrasound Gel must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

**MANUFACTURER INFORMATION:**

Optimum Medical Solutions Ltd,  
Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.

Tel: +44 (0) 845 643 5479,

Email: enquires@optimummedical.co.uk,

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Optimal Medical Solutions Ltd
MT Promed Consulting GmbH,
c/o Grant Thornton AG, Senckenberganlage 19,