
ugo.TM
hanger

Instructions for use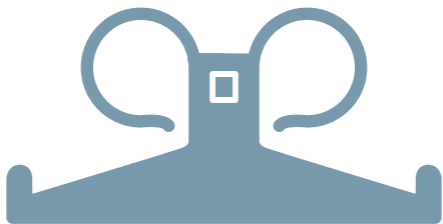

optimummedical®

 Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
LS2 9ET, Leeds, UK
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

PRODUCT REF / CONTENTS

Product Code	Description
3013	Ugo Hanger

This is a **Ugo Hanger** product information leaflet for the user. Do not share the **Ugo Hanger** with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

DESCRIPTION | The **Ugo Hanger** is a single patient, multiple use, non-sterile accessory for supporting the weight of 2L drainage bags. The 2L drainage bag is attached to the hooks of the **Ugo Hanger** through the 2 outer eyelets in the top band of the bag.

COMPOSITION | Polypropylene

INTENDED PURPOSE | The **Ugo Hanger** supports the weight of your 2L drainage bag allowing it to fill whilst in bed or overnight.

INTENDED USER | The intended users of the **Ugo Hanger** are patients with 2L drainage bag attached to urinary drainage systems who are in bed or immobile, and their Carers and/or Healthcare Professionals.

CONTRAINDICATIONS | Do not use the **Ugo Hanger** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.

WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Hanger** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional;
- Ensure that the **Ugo Hanger** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it.
- Always support the weight of your catheter and 2L drainage bag when attaching or removing from your **Ugo Hanger**.
- In addition to supporting your 2L drainage bag using the **Ugo Hanger**, it is important that your catheter and drainage system are also secured with appropriate devices.
- It is important to check that your catheter is not twisted prior to attaching onto the **Ugo Hanger** (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely.
- The **Ugo Hanger** is a reusable device. Please refer to 'How to care for your device' to ensure the product performance is maintained.
- The **Ugo Hanger** is not suitable for using when you are walking around.
- Do not use the **Ugo Hanger** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Hanger** is not made with natural rubber latex.

UNDESIRABLE SIDE EFFECTS | There may be an increased risk of complications from using the **Ugo Hanger** if you have decreased sensation and if your 2L drainage bag has not been fitted correctly onto the **Ugo Hanger**. If you experience any of the following whilst using the hanger in conjunction with a 2L drainage bag, remove the urine bag from the **Ugo Hanger** and seek medical advice:

- Urine not draining into urine bag.
- Urine leaking outside the catheter.
- Cloudy or foul-smelling urine.
- Blood in urine.
- Changes to urine colour or volume.
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort

Any serious incident or malfunction that has occurred in relation to the **Ugo Hanger** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE | Consult your Healthcare Professional for advice before using the **Ugo Hanger**.

Always wash and dry your hands before and after handling your 2L drainage bag and hanger. If someone else is

handling your 2L drainage bag and **Ugo Hanger**, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

Positioning of the Ugo Hanger:

The handles of the **Ugo Hanger** are designed to stretch and hook over bed or chair frames.

How to attach your 2L drainage bag to the **Ugo Hanger**:

- Line up and hook the 2 outer eyelets in the top band of the 2L drainage bag onto the **Ugo Hanger's** 2 hooks.
- Position and press the drainage tubing into the clip for optimal drainage.

Make sure your catheter or drainage tubing is not twisted before securing your 2L drainage bag onto the **Ugo Hanger**.

Removal and Repositioning of the Ugo Hanger:

- Your Healthcare Professional can advise on repositioning of yourself and the **Ugo Hanger** when in bed.
- It is recommended that the **Ugo Hanger** is removed and not used, along with your 2L drainage bag if you are walking around and not in bed.

Using your 2L drainage bag in conjunction with the Ugo Hanger:

- Do not disconnect your urine 2L drainage bag when attaching to your **Ugo Hanger**. It is designed specifically to allow attaching without disconnecting the system. Disconnecting the system unnecessarily increases your risk of introducing infection.
- Empty your 2L drainage bag before it reaches 2/3 full.
- It is important that your 2L drainage bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are free from obstruction and urine is draining freely.
- Ensure that the tap at the bottom of your 2L drainage bag does not touch the floor.

CARING FOR YOUR DEVICE

- It is recommended that you wash your **Ugo Hanger** daily along with your usual catheter hygiene regime.
- The **Ugo Hanger** should no longer be used if there are any signs of damage or degradation.

STORAGE AND HANDLING | Keep dry and out of direct sunlight.

DISPOSAL | The **Ugo Hanger** must be disposed of or recycled according to local policies and waste disposal procedures, including any accessories or consumables used with the device.

MANUFACTURER INFORMATION

Optimum Medical Solutions Ltd, Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.

Tel: +44 (0) 845 643 5479

Email: enquires@optimummedical.co.uk

Website: www.optimummedical.co.uk

AUTHORISED REPRESENTATIVE

MT Promedt Consulting GmbH, Altenhofstrasse 80, 66386 St. Ingbert, Germany.

Tel: +49 6894 581020

Email: info@mt-procons.com

IMPORTER INFORMATION

Optimum Medical Solutions Ltd GmbH, c/o Grant Thornton AG, Senckenberganlage 19, 60325 Frankfurt am Main, Germany.

Email: enquiries@optimummedical.de

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Product Code	Description
3013	Ugo Hanger

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Optimum Medical Solutions Ltd, Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, Wielka Brytania.
MT Promedt Consulting GmbH, Altenhofstrasse 80, 66386 St. Ingbert, Niemcy.
Optimum Medical Solutions Ltd GmbH, c/o Grant Thornton AG, Senckenberganlage 19, 60325 Frankfurt nad Menem, Niemcy.
EC REP MT Promedt Consulting GmbH