

# Product Specification Sheet

OptiPre Alcohol Wipes  
6100, 6103, 61004



optimummedical®

## ● General information

**General description:** Our disposable OptiPre Alcohol Wipes, disinfect hard surfaces in a (s)wipe, removing potentially harmful bacteria, viruses and fungi.

**Intended use:** Disposable, Alcohol Wipes for disinfecting hard surfaces.

**Lifetime of device:** Use within 60 days of opening.

## ● Physical properties:

- Non-Sterile.
- 75% Alcohol.
- Do not use if wipe appears dry.
- Do not flush.
- Highly flammable.
- Can cause serious eye irritation.

## ● Storage Conditions

- Store in original packaging in a cool, dry place away from direct sunlight.
- Keep packaging tightly closed when stored to avoid drying out.
- Dispose of wipes and container in general waste.
- Shelf Life – 2 Years if unopened.

## ● Packaging configuration:

6100	100 Wipes per Pack	260 x 290 mm
	36 Packs per Outer Case	455 x 350 x 255 mm
6103	200 wipe per Tub	100 x 210 mm
	12 Tubs per Outer Case	425 x 320 x 225 mm
61004	4 Packs per Multipack	260 x 290 mm (x4)
	8 Multipacks per Outer Case	460 x 450 x 180 mm

Product Code	Format	Wipe Thickness
6100	100 wipe pack	25gsm
6103	200 wipe Tub	25gsm
61004 (Bulk)	4 x 6100	25gsm

## ● Contains:

Wipe (substrate)

- ES (45% Polyethylene, 45% Polypropylene)
- Viscose

Liquid

- Purified Water
- Ethanol



Company Registration No: 6779883  
OMS-OM-101-Apr18

# Product Specification Sheet

OptiPre Alcohol Wipes  
6100, 6103, 61004



optimummedical®

## Efficacy Data:

			Clean conditions	Dirty conditions
Standard	Organism	Contact	Result	Result
EN 13727 Bactericidal activity	E faecium	15 secs	5.40	5.46
		30 secs	5.40	5.46
	E hirae	15 secs	5.33	5.33
		30 secs	5.33	5.33
	E coli	15 secs	5.43	5.40
		30 secs	5.43	5.40
	P aeruginosa	15 secs	5.51	5.52
		30 secs	5.51	5.52
	S aureus	15 secs	5.37	5.37
		30 secs	5.37	5.37
EN 13624 Fungus and Yeast	C albicans (Yeast)	15 secs	4.30	4.32
		30 secs	4.30	4.32
EN 14476 Viruses	Vaccinia virus	30 secs	>4.82	4.82
		1 min	>4.82	4.82
	Adenovirus	30 secs	4.00	3.44
		1 min	>4.63	4.63
	Murine norovirus	30 secs	>6.19	6.19
		1 min	>6.19	6.19
Standard	Organism	Contact	Results	CLAIMS
EN 16615	E hirae	15 secs	>5.04	99.999% effective
		30 secs	>5.04	99.999% effective
	P aeruginosa	15 secs	5.07	99.999% effective
		30 secs	>5.91	99.999% effective
	S aureus	15 secs	>5.76	99.999% effective
		30 secs	>6.13	99.999% effective
	C albicans	15 secs	>4.20	99.99% effective
		30 secs	>4.20	99.99% effective



Company Registration No: 6779883  
OMS-OM-101-Apr18