ugo.™
stand with dignity

REF 3023
REF 3024

ugo.™
stand with dignity

EU Importer:
Optimum Medical Solutions GmbH,
c/o Grant Thornton AG,
Senckenberganlage 19,
60325 Frankfurt am Main, Germany.
Email: enquiries@optimummedical.de

EC REP

MT Promedt Consulting GmbH,
Ernst-Heckel-Straße 7
66386 St. Ingbert Germany
Tel: +49 6894 581020
Email: info@mt-procons.com

optimummedical®

 Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, UK
T: +44 (0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

3023-3024-FU-OC12-16.0

Instructions for use

Figure 1. | Figura 1. | Abbildung 1.

A

Stand

Designed to support 2-litre urinary drainage bags.

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B

Dignity Cover

To be used in conjunction with your stand for discreet drainage.

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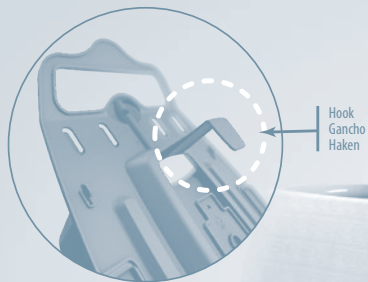
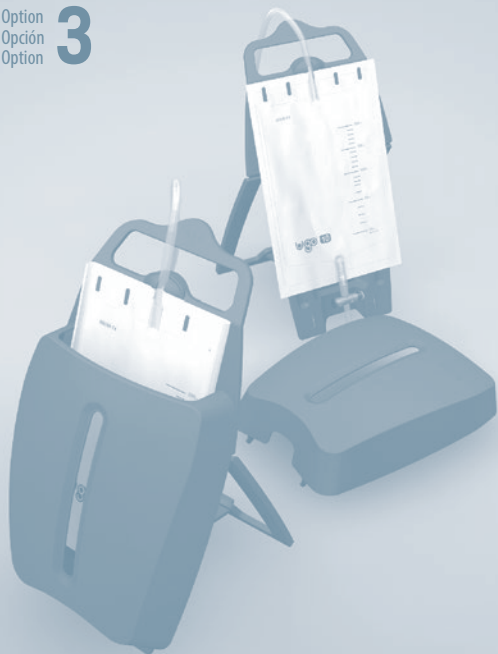
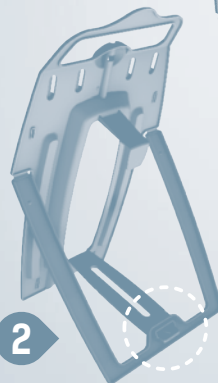
6

Figure 6. | Figura 6. | Abbildung 6.

Attaching your
2L urinary drainage bag.

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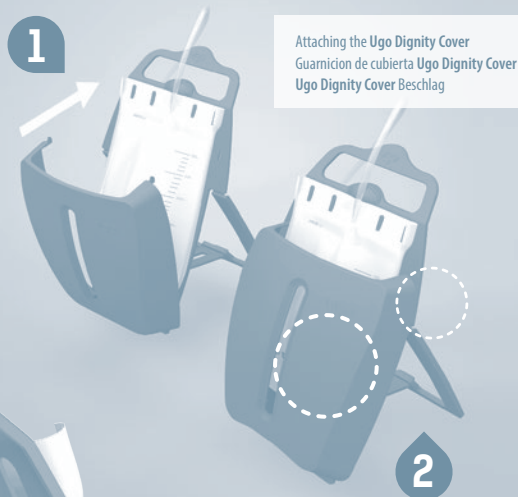
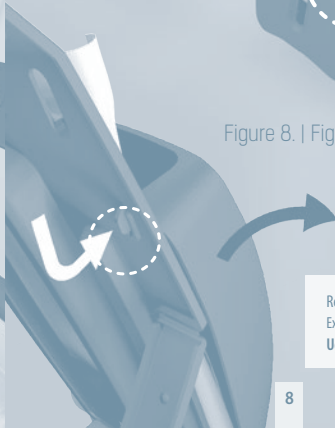
8

3023 Ugo Stand
3024 Ugo Stand with Dignity

This is a **Ugo Stand** and **Ugo Stand with Dignity** product information leaflet for the user. Do not share the **Ugo Stand** or **Ugo Stand with Dignity** with anyone else. Read all of the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

PRODUCT DESCRIPTION

The **Ugo Stand** and **Ugo Stand with Dignity** is a single patient, multiple use, non-sterile accessory for supporting the weight of 2-litre urinary drainage bags that are attached to a urinary drainage system.

COMPONENTS (See Figure 1.)

A

Stand - designed to support 2-litre urinary drainage bags

B

Dignity Cover - To be used in conjunction with your stand for discreet drainage

The 2-litre urinary drainage bag is attached to the hooks of the **Ugo Stand** through the 4 eyelets in the top band of the urinary drainage bag.

INGREDIENTS/COMPOSITION

Polypropylene.

INTENDED PURPOSE

The **Ugo Stand** supports the weight of your 2-litre urinary drainage bag as it fills. The **Ugo Stand with Dignity** cover provides optional screening for discreet drainage and protects the urinary drainage bag. This support and screening is intended to provide comfort and reduce the risk of injury from the weight of your 2-litre urinary drainage bag pulling on your drainage system while in use. It is purposely designed to reduce the risk of infection by keeping the drainage tap off the floor.

INTENDED USER

The intended users of the **Ugo Stand** or **Ugo Stand with Dignity** are patients with 2-litre urinary drainage bags, attached to a urinary drainage system, who are in bed or immobile, and their Carers and/or Healthcare Professionals.

CONTRAINDICATIONS

Do not use the **Ugo Stand** or **Ugo Stand with Dignity** if you are sensitive or allergic to the device material.

WARNINGS AND PRECAUTIONS

It is recommended that the **Ugo Stand** and **Ugo Stand with Dignity** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional. Ensure that the **Ugo Stand** and **Ugo Stand with Dignity** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it;

The dignity cover must be removed for observation and to drain urine from the 2-litre urinary drainage bag; The **Ugo Stand** and **Ugo Stand with Dignity** is a reusable device. Please refer to 'How to care for your device' to ensure the product performance is maintained;

The **Ugo Stand** and **Ugo Stand with Dignity** is not suitable for using when you are walking around; In addition to supporting your 2-litre urinary drainage bag using the stand, it is important that your catheter and drainage system are also secured with appropriate devices; It is important to check that your catheter is not twisted prior to attaching onto the stand (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely;

Do not use the **Ugo Stand** or **Ugo Stand with Dignity** if it is damaged or soiled; Do not use if the packaging is damaged or unintentionally opened prior to use. Keep out of reach of children; The **Ugo Stand** and **Ugo Stand with Dignity** are not made with natural rubber latex.

UNDESIRABLE SIDE EFFECTS

There may be an increased risk of complications from using the **Ugo Stand** and **Ugo Stand with Dignity** if you have decreased sensation and/or if your 2-litre urinary drainage bag has not been fitted correctly onto the **Ugo Stand** or **Ugo Stand with Dignity**.

If you experience any of the following whilst using the stand, with or without the dignity cover, in conjunction with a 2-litre urinary drainage bag, remove the drainage bag from the hooks at the top of the stand and seek medical help:

- Urine not draining into urinary drainage bag.
- Urine leaking outside the catheter.
- Cloudy or foul-smelling urine.
- Blood in urine.
- Changes to urine colour or volume.
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.

Any serious incident or malfunction that has occurred in relation to the **Ugo Stand** or **Ugo Stand with Dignity** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare Professional for advice before using the **Ugo Stand** or **Ugo Stand with Dignity**. Always wash and dry your hands before and after handling your urinary drainage bag and stand. If someone else is handling your drainage bag and/or stand, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

ASSEMBLING AND POSITIONING THE STAND

There are 3 options to use the **Ugo Stand** or **Ugo Stand with Dignity** to support your 2-litre urinary drainage bag. **Warning!** The **Ugo Stand** or **Ugo Stand with Dignity** is not suitable for using when you are walking around.

Option 1. (See Figure 2.) Slots between mattress and divan.

Slide the longer panel on the back of the **Ugo Stand** and **Ugo Stand with Dignity**, in between the mattress and the divan/bed.

Position the stand as close to the bed as possible to ensure a secure fit.

Option 2. (See Figure 3.) Hooks onto bed frame or chair arm.

Using the hook on the back of the stand, carefully hook the stand into a position that is comfortable for you.

Option 3. (See Figure 4.) Free standing on the floor.

ASSEMBLING THE STAND FOR USE IN THE FREE-STANDING POSITION

(See Figure 5.)

1. Pull out the longer panel on the back of the stand.
2. Pull down the shorter arm on the back of the stand.
3. Click the longer panel and short arm together to create the free-standing stand.

HOW TO ATTACH YOUR 2-LITRE URINARY DRAINAGE BAG TO THE STAND

(See Figure 6.)

Line up and hook the 4 eyelets in the top band of the 2-litre urinary drainage bags onto the stand's 4 hooks.

Warning! - Make sure your catheter or drainage tubing is not twisted before securing your 2-litre urinary drainage bag onto the **Ugo Stand** or **Ugo Stand with Dignity**.

ATTACHING AND REMOVING THE DIGNITY COVER

ATTACHING (See Figure 7.)

1. Line up the tabs on the bottom of the dignity cover with corresponding holes on the stand (as a hinge).
2. Once hinged at the bottom, raise the dignity cover and click in place at the top.

REMOVING (See Figure 8.)

1. Remove the dignity cover by simultaneously pressing the two tabs at the top/back of the stand to release.
2. The cover can then be hinged down and removed.

Removal and Repositioning of the Ugo Stand and Ugo Stand with Dignity

- Your Healthcare Professional can advise on repositioning of yourself and the **Ugo Stand** or **Ugo Stand with Dignity** when in use.
- It is recommended that the **Ugo Stand** and **Ugo Stand with Dignity** is removed and not used, along with your 2-litre urinary drainage bag if you are walking around and not in bed.

Using your urine drainage bag in conjunction with the Ugo Stand and Ugo Stand with Dignity

- Always remove the dignity cover for observation of urine and to drain urine from the 2-litre urinary drainage bag.
- Do not disconnect your urine drainage bag when attaching to your stand. It is designed specifically to allow attaching without disconnecting the system from the catheter. Disconnecting the system unnecessarily increases your risk of introducing infection.
- Empty your 2-litre urinary drainage bag before it reaches 2/3 full or sooner if advised by your Healthcare Professional.
- It is important that your 2-litre urinary drainage bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are free from obstruction and urine is draining freely.
- Ensure that the drainage tap at the bottom of your 2-litre urinary drainage bag does not touch the floor.
- Always support the weight of your catheter and 2 litre urinary drainage bag when attaching or removing from your stand.

CARING FOR YOUR DEVICE

- It is recommended that you wash your **Ugo Stand** or **Ugo Stand with Dignity** daily along with your usual catheter hygiene regime.
- The **Ugo Stand** or **Ugo Stand with Dignity** should no longer be used if there are any signs of damage or degradation.

STORAGE AND HANDLING

Keep dry and out of direct sunlight.

DISPOSAL

The **Ugo Stand** or **Ugo Stand with Dignity** must be disposed of or recycled according to local policies and waste disposal procedures, including any accessories or consumables used with the device.

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MT Promed Consulting GmbH,