This IFU can be recycled.

MT Promedt Consulting GmbH,
Ernst-Heckel-Straße 7,
66386 St. Ingbert, Germany

Decomplex AG,
Freiburgstrasse 3,
3010 Bern, Switzerland


Optimum Medical Solutions GmbH,
c/o Grant Thornton AG,
Senckenberganlage 19,
60325 Frankfurt am Main, Germany.
Email: enquiries@optimummedical.de

OL-IFU-Jan23-V17.0

optilube.™

sterile lubricating jelly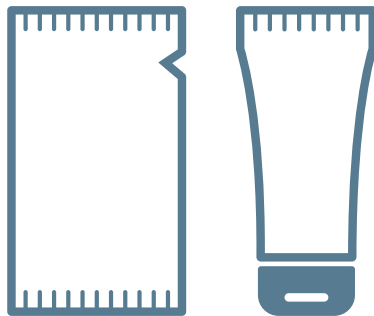

Instructions for use

optimummedical®

 **Optimum Medical Solutions Ltd**
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

BRAND: OptiLube**PRODUCT REF / CONTENTS**

Sachets	Tubes	Reach Tubes
11195 20g – single wrapped	11195 10g	1140 5g
1119 20g – double wrapped	1121 42g	1141 15g
1120 5g	1122 82g	
1123 2.7g	1124 5g	
1129 3g	1127 113g	

This is a product information leaflet for the user.

OptiLube has been recommended/prescribed for you by a Healthcare Professional. Do not share it with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek medical advice.

PRODUCT DESCRIPTION:

OptiLube is a sterile, transparent, water-soluble lubricating jelly with a pH range of 5.0 – 7.0.

INGREDIENTS:

- Purified water • Polyethylene Glycol (PEG) • Glycerine
- Carbomer • Sodium hydroxide • 4-Hydroxybenzoic acid esters

INTENDED PURPOSE:

OptiLube is a medical device that is a water-soluble, sterile lubricating jelly intended for the smooth insertion of medical devices, and examinations, reducing the risk of trauma during procedures, including:

- Insertion of catheters • Endoscopy
- Gynaecological examinations • Rectal examinations • Cystoscopy

INTENDED USER:

The intended users of **OptiLube** are patients, their Carers and/or Healthcare Professionals.

CONTRAINDICATIONS:

Do not use **OptiLube** if you are sensitive or allergic to any of the device ingredients.

WARNINGS AND PRECAUTIONS:

It is recommended that **OptiLube** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.

Ensure that **OptiLube** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.

OptiLube is not made with natural rubber latex.

Do not use **OptiLube** if it is damaged or seal is broken. Keep out of reach of children. Apply the required amount of lubrication necessary for the procedure. Not for use in the eye.

Re-use of this medical device may result in infection and/or cross-contamination. Re-sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.

UNDESIRABLE SIDE EFFECTS:

There are no known side effects associated with using **OptiLube**, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering.

If this occurs, stop using **OptiLube** and consult your Healthcare Professional. If symptoms suddenly progress or become worse, which may include feeling lightheaded or faint, breathing difficulties, a fast heartbeat, confusion and anxiety, seek medical help immediately.

Any serious incident or malfunction that has occurred in relation to **OptiLube** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE:

Consult your Healthcare Professional for advice before using **OptiLube**.

Do not use if packaging is damaged or unintentionally opened prior to use.

SACHETS: Sterile and single-use. Tear sachet at one end and apply as required.

Note: **OptiLube** 20g sachet (1119) is supplied double wrapped, therefore it is advised that you use an aseptic technique to open.

TUBES: Sterile until opened. Remove cap and peel foil from end of tube. Apply as necessary. Close cap after use. Use within 3 months of opening. Discard sooner if the tube becomes contaminated.

REACH: Sterile and single-use. Snap nozzle tip to open and apply as necessary. Dispose of after use.

STERILE DEVICES:

OptiLube is supplied sterile.

OptiLube is sterilised by gamma after the packaging process.

Do not re-sterilise.

STORAGE AND HANDLING:

Keep dry.

Keep away from sunlight.

Reapply cap in between use.

Unopened packs have a 5 year shelf life.

Do not use after expiry date.

DISPOSAL:

OptiLube must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

MANUFACTURER INFORMATION:

Optimum Medical Solutions Ltd,
Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.

Tel: +44 (0) 845 643 5479

Email: enquiries@optimummedical.co.uk

Website: www.optimummedical.co.uk

AUTHORISED REPRESENTATIVE:

MT Promedt Consulting GmbH,
Ernst-Heckel-Straße 7, 66386 St. Ingbert, Germany.

Email: ear@mt-procons.com

EU IMPORTER INFORMATION:

Optimum Medical Solutions GmbH,
c/o Grant Thornton AG, Senckenberganlage 19,
60325 Frankfurt am Main, Germany. Email: enquiries@optimummedical.de

SWITZERLAND AUTHORISED REPRESENTATIVE:

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