optipure.TM

Instructions for use

PRODUCT REF / CONTENTS

1128 **OptiPure** 10ml Sterile water syringe

1131 **OptiPure** 10ml Sterile water syringe with 10% Glycerine

This is a product information leaflet for the user.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice.

PRODUCT DESCRIPTION

OptiPure is a sterile, single use syringe, prefilled with 10ml of sterile water (1128) or sterile water with 10% glycerine (1131) for the inflation of indwelling urinary Foley catheter balloons. **OptiPure** is a clear liquid.

INGREDIENTS/COMPOSITION

1128 100% Sterile water

1131 90% Sterile water and 10% Glycerine

INTENDED PURPOSE

OptiPure is for the inflation of indwelling urinary Foley catheter balloons, to retain the catheter in the bladder.

INDICATIONS FOR USE / INTENDED USER

OptiPure is a medical device intended to be used by Healthcare Professionals when performing an indwelling catheterisation procedure.

CONTRAINDICATIONS

Do not use **OptiPure** if the intended user or the patient are sensitive or allergic to the ingredients.

WARNINGS AND PRECAUTIONS

- It is recommended that **OptiPure** is used by a Healthcare Professional in line with local healthcare policies and procedures.
- Ensure that **OptiPure** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.
- Always ensure the Foley catheter is fully inserted into the bladder and urine is draining before inflating the balloon using **OptiPure**.
- Always use the volume advised in the Foley catheter instructions for use for inflating the balloon.
- This is a single-use device. Re-use of this device may result in patient infection / cross-contamination.
- Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.
- Keep out of reach of children.
- **OptiPure** is not made from natural rubber latex.
- Do not use the **OptiPure** syringe if it is damaged or soiled.
- Do not use for injection, reconstitution of drugs, medication dilution or patient rehydration.

UNDESIRABLE SIDE EFFECTS

Any serious incident or malfunction that has occurred in relation to the **OptiPure** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Do not use if packaging is damaged or unintentionally opened prior to use.

OptiPure does not have a measuring function, do not use for accurate recording.

OptiPure must be used as part of an aseptic technique and in accordance with local healthcare policies and procedures and best practice guidance by suitably qualified and competent Healthcare Professionals.

- Wash hands and open **OptiPure** pouch.
- Visually inspect the contents of each prefilled syringe for clarity, discoloration or leakage prior to use and do not use if any of the above is observed within the syringe.
- Remove the cap from the **OptiPure** syringe and hold the tip of syringe upward and gently depress the plunger to expel any air.
- Attach the syringe tip to the inflation lumen of the Foley catheter.
- Inflate the Foley catheter balloon using the correct volume.

STERILE DEVICES

- **OptiPure** is supplied sterile.
- **OptiPure** is sterilised by gamma radiation after the packaging process.
- Do not re-sterilise.

STORAGE AND HANDLING

- Store between 5-30°C (41-86°F) until expiry date.
- Keep dry and out of direct sunlight.
- Unopened packs have a 3-year shelf life.
- Do not use after expiry date.

DISPOSAL

OptiPure must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

MANUFACTURER INFORMATION

Optimum Medical Solutions Ltd, Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.

Tel: +44 (0) 845 643 5479

Email: enquiries@optimummedical.co.uk

Website: www.optimummedical.co.uk

EU AUTHORISED REPRESENTATIVE

MT Promedt Consulting GmbH, Ernst-Heckel-Straße 7, 66386 St. Ingbert, Germany.

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MedEnvoy Global B.V.