
# HunterUrology

The Self Catheterisation Specialists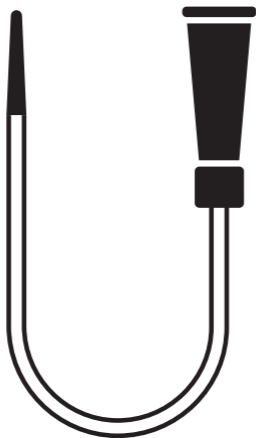

## Instructions for use

# Hunter Urology Intermittent Self Catheterisation (ISC)

## PRODUCT INFORMATION

This is a product information leaflet for the user of the following Intermittent Self Catheterisation (ISC):

SNF/SNM/SNP	<b>Smooth Nelaton</b> Non-coated urethral catheter with Nelaton tip, for intermittent self-catheterisation
SCF/SCM/SCP	<b>HunterCath</b> Non-coated urethral catheter with soft tip, for intermittent self-catheterisation
ABF/ABM/ABP	<b>Aloe Original</b> Hydrophilic coated urethral catheter with Nelaton tip, for intermittent self-catheterisation
AHF/AHM/AHP	<b>Aloe Hydro</b> Hydrophilic coated urethral catheter with Nelaton tip, for intermittent self-catheterisation
PSAHF/PSAHM/PSAHP	<b>Aloe Hydro+</b> Hydrophilic coated urethral catheter with soft tip, for intermittent self-catheterisation
ALOEMPS	<b>Aloe Meatal+</b> Hydrophilic coated dilation catheter

This catheter has been prescribed for you by a healthcare professional. Do not share it with anyone else. Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a healthcare professional.

## INGREDIENTS/COMPOSITION

This catheter is made from medical grade polyvinylchloride (PVC), which does not contain phthalates (DEHP free). The hydrophilic coating contains polyvinylpyrrolidone (PVP).

## INTENDED PURPOSE

ISC catheter for patients who are required to use a catheter to empty their bladder. ISC can be used to empty the bladder regularly which will help protect the kidneys and reduce the occurrence of Urinary Tract Infections (UTIs). The **Aloe Meatal+** catheter has no drainage holes and is used for dilation of the urethra only.

## INDICATIONS FOR USE / INTENDED USER

The ISC catheter is a medical device indicated for transient bladder catheterisation by introduction through the urethra for urine drainage or therapeutic purposes.

The intended users of the medical device are male or female patients (who have been assessed as suitable by a healthcare professional), their carers and/or healthcare professionals.

### **CONTRAINDICATIONS**

Do not use the catheter unless you have been assessed as suitable by a healthcare professional and it has been recommended for you.

Use of the catheter is not advised on patients with known or suspected hypersensitivity to any of the ingredients.

### **WARNINGS AND PRECAUTIONS**

- It is recommended that the catheter is used in line with local policies and procedures following a patient assessment by a healthcare professional.
- Ensure that the catheter is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.
- Do not use the catheter if the catheter or the packaging is damaged or soiled.
- For catheters provided with water sachets, do not use the catheter where full emptying of water sachet is not achieved, or the water sachet is already empty or damaged.
- For soft tip catheters, do not use the catheter if the tip is damaged. After catheter use, check if the catheter tip is still intact. In the case of a missing part of the tip, consult with a healthcare professional.
- Do not puncture the catheter to obtain a urine sample as this may result in trauma or infection to the patient.
- This is a single-use medical device. Re-use of the catheter may result in patient infection / cross contamination. Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.
- This catheter is not made with natural rubber latex.
- If the product is not used in accordance with the instructions provided by the manufacturer, the company does not take responsibility of any injuries or other damage that may arise.

### **UNDESIRABLE SIDE EFFECTS**

- Although the procedure of self-catheterisation itself is safe, complications can occur, especially for patients performing self-catheterisation for longer periods. If any complications occur, consult a healthcare professional.
- To avoid damage to the urethra and other complications, consult a healthcare professional on choosing the suitable size and shape of the catheter or any other accessories.
- Some pain and blood may occur especially in the beginning due to irritation of the mucous membrane of the urethra. If symptoms persist, consult a healthcare professional.
- Symptoms of a urinary tract infection include blood in urine, pain or discomfort when urinating, increased need to urinate, cloudy or foul-smelling urine, fever and/or inflammation. If any of these symptoms occur, stop using the catheter and consult a healthcare professional.
- In rare cases, local irritation/hypersensitivity reactions may occur such as redness, itching or blistering. If any of these symptoms occur, stop using the catheter and consult a healthcare professional.

- Any serious incident or malfunction that has occurred in relation to the catheter should be reported to the manufacturer and the Competent Authority of the Member State in which the patient is established.

## **HOW TO USE**

Consult your healthcare professional for advice before using this catheter. Do not use if packaging is damaged or unintentionally opened prior to use.

## **HOW TO CATHETERISE**

1. Try to pass urine naturally before catheterisation.
2. Wash your hands thoroughly with soap and water.
3. Gently clean the genital area with soap and water. Be careful not to touch anything else other than your catheter once you have washed your hands. If you are using public toilets, cleansing wipes or antibacterial hand gel may be useful to enable you to ensure the area is clean prior to catheterisation.
4. **Choice of catheter:**

### **Coated catheters without water sachet:**

Discard the product if the packaging is not intact.

- Peel the catheter packaging 2-3cm. If your catheter touches anything or you drop it, throw it away and use a new one.
- Put water into the pouch up to the top of the coating or coloured funnel. If cold water is used, the catheter will be stiffer; if hot water is used the catheter will be softer.
- Using the sticker on the back or the hole in the packaging, hang the catheter on a clean surface in a position of easy access near the toilet or a jug or catheter bag to drain the urine into.
- The catheter should be left for at least 30 seconds to activate the coating.

### **Coated catheters with water sachet:**

Discard the product if the packaging is not intact.

- Before activation of the water sachet, check its integrity.
  - Squeeze the water sachet in closed packaging together with the catheter. For easier opening, it is recommended to bend the sachet together with the packaging before squeezing.
  - All water has to be squeezed out of the sachet.
  - Using the sticker on the back or the hole in the packaging, hang the catheter on a clean surface in a position of easy access near the toilet or a jug or catheter bag to drain the urine into.
  - Catheter should be left for at least 30 seconds to activate the coating.
  - Peel the catheter packaging 2-3cm using aseptic technique and do not touch this area.
5. After taking the catheter out of the packaging, visually inspect the catheter for any damage and use it immediately. Do not place the catheter without its packaging on any surface and do not touch the end of the catheter to be inserted into the urethra.
  6. **CATHETER INSERTION:**  
Adopt the position that you find most comfortable in order to catheterise.

**Instructions for females:** Part the labia with one hand and using your

dominant hand, insert the catheter into your urethra until it enters the bladder. You may find that using a mirror to help locate your urethra is helpful while you are getting used to this procedure. Be careful not to touch any part of the catheter that is to enter the bladder.

**Instructions for males:** Pull back your foreskin (if you have one) and hold your penis firmly up towards your stomach with one hand. Insert your catheter with your dominant hand and be careful not to touch any part of the catheter that is to enter the bladder.

7. When urine starts to flow, the catheter can be attached to a urine bag or drained directly into a toilet or jug. When the urine flow stops, retract the catheter by 2-3cm. If urine flow starts again wait for it to stop, retract the catheter for another 1-2cm to drain the rest of the urine. When the bladder is completely empty, slowly and gently remove the catheter.

If using the **Aloe Meatal+** catheter, remove the catheter after performing dilation of the urethra.

The **Aloe Meatal+** catheter has no drainage holes and must be removed to allow urine to flow.

8. Dispose of the used catheter and any other used items in a disposal bag and place in the general household rubbish.
9. After the procedure is finished, wash your hands with soap and water. Dry your hands afterwards.

### **STORAGE AND HANDLING**

- The catheter is supplied sterile. Sterilised using ethylene oxide. Do not re-sterilise.
- Store between 5-40°C.
- Keep dry.
- Keep away from sunlight.
- Unopened packs have a 5-year shelf life.
- Do not use after expiry date.

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