REF 3100/3101/3102/3103/3104/3105

REF 3106/3107/3108/3109/3110/3111

LATEX FREE

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33-Suite 123
2595 AM The Hague, The Netherlands

EC REP

MT Promedt Consulting GmbH,
Ernst-Heckel-Straße 7,
66386 St. Ingbert, Germany
Tel: +49 6894 581020, Email: ear@mt-procons.com

Sponsor:
Optimum Medical Australasia Pty Limited,
Level 4, 470 Church Street, North Parramatta,
New South Wales, 2151, Australia


optimummedical®

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, UK
T: +44 (0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

ugo.™
sheath

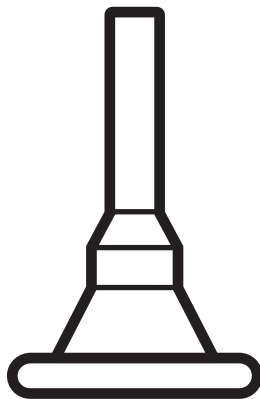Instructions for use

optimummedical®

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
LS2 9ET, Leeds, UK
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

3100-3111-IFU-Apr23-V5.0

PRODUCT REF / CONTENTS

CODE	PRODUCT NAME	LENGTH	DIAMETER	QUANTITY
3100	Ugo Sheath	Standard	25mm	28 units - 4 boxes of 7 Ugo Sheaths
3101	Ugo Sheath	Standard	28mm	28 units - 4 boxes of 7 Ugo Sheaths
3102	Ugo Sheath	Standard	30mm	28 units - 4 boxes of 7 Ugo Sheaths
3103	Ugo Sheath	Standard	32mm	28 units - 4 boxes of 7 Ugo Sheaths
3104	Ugo Sheath	Standard	36mm	28 units - 4 boxes of 7 Ugo Sheaths
3105	Ugo Sheath	Standard	40mm	28 units - 4 boxes of 7 Ugo Sheaths
3106	Ugo Sheath	Pop-On	25mm	28 units - 4 boxes of 7 Ugo Sheaths
3107	Ugo Sheath	Pop-On	28mm	28 units - 4 boxes of 7 Ugo Sheaths
3108	Ugo Sheath	Pop-On	30mm	28 units - 4 boxes of 7 Ugo Sheaths
3109	Ugo Sheath	Pop-On	32mm	28 units - 4 boxes of 7 Ugo Sheaths
3110	Ugo Sheath	Pop-On	36mm	28 units - 4 boxes of 7 Ugo Sheaths
3111	Ugo Sheath	Pop-On	40mm	28 units - 4 boxes of 7 Ugo Sheaths

Each inner box contains:

28 Sheaths -
4 boxes of 7 Ugo Sheaths
28 disposable hair guards.
1 reusable travel pouch.

This is a product information leaflet for the user.

The **Ugo Sheath** has been recommended/prescribed for you by a Healthcare Professional. Do not share it with anyone else.

Read all of the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek medical advice.

PRODUCT DESCRIPTION

The **Ugo Sheath** is a self-adhesive soft silicone penile sheath. It is used for the management of male urinary incontinence, by external application of the sheath over the penis. It is then connected to a urine drainage bag with a universal connector to drain away urine.

COMPOSITION

Silicone sheath, Acrylic adhesive

INTENDED PURPOSE

The **Ugo Sheath** is a medical device indicated for the non-invasive management and containment of male urinary incontinence.

INTENDED USER

The intended users of the **Ugo Sheath** are men who require a non-invasive urinary containment device (who have been assessed as suitable by a Healthcare Professional). It is applied by the user, their carers and/or Healthcare Professionals.

CONTRAINDICATIONS

Do not use the **Ugo Sheath** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.

The **Ugo Sheath** is contraindicated:

- In patients with urinary retention;
- On penile skin which is broken, inflamed or swollen;
- In patients who are sensitive or allergic to any of the device materials;
- On a retracted penis where there is no accessible penile length to fit the sheath.

WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Sheath** is used in line with healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Sheath** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.
- Only use the **Ugo Sheath** if you (or your carer) are willing to use it and are able to care for it and you can remember to do so.
- The **Ugo Sheath** measuring guide should only be used when measuring for the **Ugo Sheath**. It is not suitable for use with any other sheath.
- It is important that your urine drainage bag is secured or supported with appropriate devices to prevent pulling on your sheath.
- It is important to check your **Ugo Sheath** is not twisted prior to attaching to your urine drainage bag (see how to use) and the urine drainage bag connector is not pushed too far into the sheath.
- Regularly check that your **Ugo Sheath** and all urine draining tubing are free from obstruction and urine is draining freely.
- Do not modify or adapt the **Ugo Sheath** to fit.
- Do not use barrier creams and moisturisers including moisturising soaps or shower gels, wet wipes, baby wipes or talcum powder prior to, or while using the **Ugo Sheath**, as they may affect the adhesion of the sheath.
- Do not use on skin which is broken or inflamed.
- Do not shave the pubic area as it may cause skin irritation.
- Do not use the **Ugo Sheath** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Sheath** is not made of natural rubber latex.

This is a single-use device. Re-use of this device may result in patient infection / cross-contamination.

Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.

UNDESIRABLE SIDE EFFECTS

There may be an increased risk of complications from using the **Ugo Sheath** if you have decreased sensation, if you have not been assessed by a Healthcare Professional and you are not

suitable to use the **Ugo Sheath** (see contraindications), or you have not been measured and fitted correctly.

Remove the **Ugo Sheath** and seek help immediately from a Healthcare Professional if you experience any of the following:

- Swelling, inflammation or discomfort of the penis;
- Lesions or broken skin on the penis;
- Compression or discolouration of the penis;
- Feeling unwell, high temperature, nausea/vomiting;
- Feeling of any pain or discomfort;
- Urine not draining into the urine drainage bag;
- Urine leaking from the sheath or sheath falling off;
- Cloudy or foul-smelling urine;
- Blood seen in urine;
- Changes to urine colour and volume.

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Sheath** and consult your Healthcare Professional.

Any serious incident or malfunction that has occurred in relation to the **Ugo Sheath** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare Professional for advice before using the **Ugo Sheath**.

Always wash and dry your hands before and after handling your **Ugo Sheath**. If someone else is handling your **Ugo Sheath**, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

Selecting the correct size of your Ugo Sheath:

It is important to measure for the correct size. If the **Ugo Sheath** is too tight it can restrict circulation and if too loose will not give an adequate seal. Measure the penis when seated in a chair or on the edge of a bed.

Warning! Only use the **Ugo Sheath** measuring guide to measure for a **Ugo Sheath**.

Sheath length:

If the penis length is 5cm or less in the seated position, select a pop-on length sheath.

Sheath diameter:

Using the **Ugo Sheath** measuring guide, measure the penis where its diameter is widest and select the correct size. When sizing for a sheath take into consideration natural changes in penis size.

If you measure between sizes, try both sizes for best fit.

HOW TO APPLY THE UGO SHEATH:

The **Ugo Sheath** can be worn for up to 24 hours.

Always inspect the penile skin after removal of the sheath and prior to fitting a new **Ugo Sheath**. Maintain a good daily hygiene routine by washing with soap and water between each sheath fitting and ensure the penis is clean and dry before fitting the sheath.

- Trim any hair which may get caught in the sheath and if necessary, use the enclosed gauze fabric square hair guards;

- Position the **Ugo Sheath** over the end of the penis. If a foreskin is present, ensure it is in place covering the glans penis and not retracted when fitting the sheath;
- Leave a 2cm – 3cm space between the end of the penis and the base of the sheath spout;
- Fully unroll the **Ugo Sheath** along the length of the penis, ensuring that it rolls evenly. Hold the sheath in place to allow time for the adhesive to bond;
- Fit the urine drainage bag connector into the sheath spout.

Warning! - Make sure your **Ugo Sheath** or drainage tubing are not twisted and the urine drainage bag connector is not pushed too far into the sheath.

- Secure the urine drainage bag to the leg using appropriate fixation devices and make sure you are comfortable and can move around freely and urine is draining into the bag. If a 2L drainage bag is used, then this must be attached to a stand or a hanger.

HOW TO REMOVE THE UGO SHEATH:

- Loosen the base of the **Ugo Sheath** from the penis, all the way around;
- Use warm water and soap or use an adhesive remover to help unroll the sheath down the length of the penis and gently remove.

Always inspect the penile skin after removal of the sheath and prior to fitting a new **Ugo Sheath**.

Using your urine drainage bag in conjunction with the Ugo Sheath:

- Empty your urine bag before it reaches 2/3 full.
- It is important that your urine bag is positioned below the level of your bladder and to check regularly that your sheath and drainage tubing are free from obstruction and urine is draining freely.
- Ensure that the tap at the bottom of your urine bag does not touch the floor.
- Always support the weight of your sheath and urine drainage bag when fitting and securing to your leg or a urine bag stand.

STORAGE AND HANDLING

Keep dry and out of direct sunlight. Unopened packs have a 3-year shelf life. Do not use after expiry date.

DISPOSAL

The **Ugo Sheath** must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device. Each **Ugo Sheath** box and wrapping is fully recyclable.

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