REF 1156

REF 1156S

STERILE R

LATEX FREE

UK  
CA  
0086

MD

EC REP

MT Promedt Consulting GmbH  
Ernst-Heckel-Straße 7  
66386 St. Ingbert Germany


MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33- Suite 123  
2595 AM The Hague  
The Netherlands

Sponsor:  
Optimum Medical Australasia Pty Ltd  
Level 4 470 Church Street, North Parramatta,  
NSW 2151, Australia

1156-1156S-IFU-Apr23-V2.0

**vue.**<sup>TM</sup>  
ultrasound gel

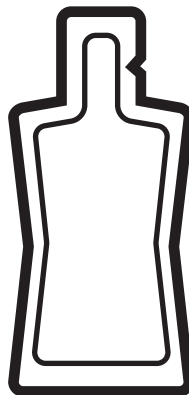# Instructions for use

optimummedical®

 Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom  
T: +44(0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

**PRODUCT REF / CONTENTS**

**1156** – 20ml sachet, double wrapped

**1156S** – 20ml sachet

This is a product information leaflet for the user.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable, please seek medical advice.

**PRODUCT DESCRIPTION:**

**Vue Ultrasound Gel** is a sterile, water-based gel.

**INGREDIENTS/COMPOSITION**

- Purified Water
- Polyethylene Glycol
- Glycerine
- Carbomer
- 4-Hydroxybenzoic acid esters

**INTENDED PURPOSE:**

**Vue Ultrasound Gel** is a sterile medical device intended to be used for the transmission of ultrasonic waves during general examinations on intact skin.

**INDICATIONS FOR USE / INTENDED USER:**

**Vue Ultrasound Gel** is a medical device intended to be used by Healthcare Professionals when performing ultrasound.

**CONTRAINDICATIONS:**

- Do not use if sensitive or allergic to any of the ingredients.
- Do not use on broken or damaged skin.
- Not suitable for use with a defibrillator, ECG or laser treatment.

**WARNINGS AND PRECAUTIONS:**

- It is recommended that **Vue Ultrasound Gel** is used in line with local healthcare policies and procedures.
- Ensure that **Vue Ultrasound Gel** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.
- Single use only. Do not reuse the sachet once opened.
- For external use only. Not for invasive use.
- Not to be ingested.
- Avoid contact with the eyes.
- Keep out of reach of children.
- **Vue Ultrasound Gel** is not made from natural rubber latex.

**UNDESIRABLE SIDE EFFECTS:**

There are no known side effects associated with using **Vue Ultrasound Gel**, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using **Vue Ultrasound Gel** and ensure patient records are updated.

Any serious incident or malfunction that has occurred in relation to the **Vue Ultrasound Gel** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

**HOW TO USE:**

Always wash and dry your hands before and after using **Vue ultrasound Gel** and apply a clean pair of gloves on every occasion to avoid cross infection. Do not use if packaging is damaged or unintentionally opened prior to use.

1. Clean the patient's skin prior to applying **Vue Ultrasound Gel**.
2. Follow the instructions supplied by the manufacturer of the equipment being used to perform the medical ultrasound procedure.
3. Apply the required amount of **Vue Ultrasound Gel** to the area of skin/ equipment.
4. Following the procedure, remove any remaining ultrasound gel from the patient's skin using personal hygiene wipes or wash off using patient dry wipes and soap and water.
5. Dry the patient's skin thoroughly afterwards with paper towels.
6. Discard any remaining gel.

**STERILE DEVICES**

**Vue Ultrasound Gel** is supplied sterile.

**Vue Ultrasound Gel** is sterilised by gamma irradiation after the packaging process. Do not re-sterilise.

**STORAGE AND HANDLING:**

Keep dry.

Keep out of direct sunlight.

Unopened packs have a 5-year shelf life.

Do not use after expiry date.

**DISPOSAL:**

**Vue Ultrasound Gel** must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

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