ugo.TM
leg bag

ugo.TM
2L drainage bag

Symbol Index

All Products	        22 PAP  LATEX FREE
Non-sterile products	 NON STERILE  
Sterile products	 STERILE EO  STERILIZE   2797 0086

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33-Suite 123
2595 AM The Hague, The Netherlands

EC REP

MT Promedt Consulting GmbH,
Ernst-Heckel-Straße 7,
66386 St. Ingbert, Germany
Tel: +49 6894 581020,
Email: ear@mt-procons.com

Sponsor:
Optimum Medical Australasia Pty Limited,
Level 4, 470 Church Street, North Parramatta,
New South Wales, 2151, Australia

optimummedical[®]



Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, UK
T: +44 (0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

Ugo-Leg-Bags-IFU-Apr23-V7.0

ugo.TM
leg bag

ugo.TM
2L drainage bag

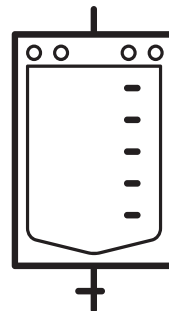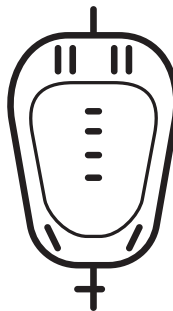Instructions for use

optimummedical[®]

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, UK
T: +44 (0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

Ugo Leg Bags and Ugo 2 Litre Drainage Bags

PRODUCT REF / CONTENTS

COMMUNITY LEG BAGS (Sterile)

Box Of 10 - non-woven fabric backing - 1 pair of **Ugo Fix Leg Bag Straps** (non-sterile) provided in every box.

CODE	PRODUCT NAME	SIZE (ML)	LENGTH (CM)	TAP TYPE
1C/500/ST/L	Ugo 1C	500ml	Short tube (5cm)	lever tap
2C/500/LT/L	Ugo 2C	500ml	Long tube (25cm)	lever tap
3C/500/ST/T	Ugo 3C	500ml	Short tube (5cm)	T tap
4C/500/LT/T	Ugo 4C	500ml	Long tube (25cm)	T tap
5C/350/DI/L	Ugo 5C	350ml	Direct inlet	lever tap
6C/350/ST/L	Ugo 6C	350ml	Short tube (5cm)	lever tap
7C/750/ST/L	Ugo 7C	750ml	Short tube (5cm)	lever tap
8C/750/LT/L	Ugo 8C	750ml	Long tube (25cm)	lever tap

COMMUNITY LEG BAGS (Sterile)

Box Of 10 - non-woven fabric backing - 1 pair of **Ugo Fix Leg Bag Straps** (non-sterile) provided in every box.

CODE	PRODUCT NAME	SIZE (ML)	LENGTH (CM)	TAP TYPE	KIT CONTENTS
3410	Ugo Pro 1C	500ml	Short tube (5cm)	lever tap	Apron and 1 pair of nitrile gloves
3411	Ugo Pro 2C	500ml	Long tube (25cm)	lever tap	Apron and 1 pair of nitrile gloves
3412	Ugo Pro 3C	500ml	Short tube (5cm)	T tap	Apron and 1 pair of nitrile gloves
3413	Ugo Pro 4C	500ml	Long tube (25cm)	T tap	Apron and 1 pair of nitrile gloves

HOSPITAL LEG BAGS (Sterile)

Box Of 10 - no fabric backing – 1 pair of **Ugo Fix Leg Bag Straps** provided in every pouch.

CODE	PRODUCT NAME	SIZE (ML)	LENGTH (CM)	TAP TYPE
1H/500/ST/L	Ugo 1H	500ml	Short tube (5cm)	lever tap
2H/500/LT/L	Ugo 2H	500ml	Long tube (25cm)	lever tap
3H/500/ST/T	Ugo 3H	500ml	Short tube (5cm)	T tap
4H/500/LT/T	Ugo 4H	500ml	Long tube (25cm)	T tap

2 LITRE DRAINAGE BAGS - 90cm Inlet Tubing.

CODE	PRODUCT NAME	USE	STERILITY	PER BOX/BAG
9/NS/SU/DO	Ugo 9	non drainable/tear strip	Non-Sterile	resealable bag of 10
10/NS/SU/DO	Ugo 10	Single-use T tap	Non-Sterile	resealable bag of 10
11/S/SU/DO	Ugo 11	Single-use T tap	Sterile	box of 10
12/S/D/L	Ugo 12	Lever tap	Sterile	box of 10
13/S/D/T	Ugo 13	T tap	Sterile	box of 10

Xtra 2 LITRE DRAINAGE BAGS - 120cm Inlet Tubing.

CODE	PRODUCT NAME	USE	STERILITY	PER BOX/BAG
3300	Ugo 10 Xtra	Single-use T tap	Non-Sterile	Resealable bag of 28

This is a product information leaflet for the user.

The **Ugo Leg Bags** and/or **Ugo 2L Drainage Bags** have been recommended/prescribed for you by a Healthcare Professional. Do not share them with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

PRODUCT DESCRIPTION

Each of the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** have different features to suit a range of clinical environments and individual patient needs (please see contents).

The **Ugo Leg Bags** are a range of sterile, single use urinary drainage bags designed to be worn on the leg for up to 7 days.

All the **Ugo Leg Bags** have a needle-free sample port, a universal stepped connector which is connected directly to the catheter or male urinary sheath, a soft silicone outlet tubing for direct drainage or connection to a 2 Litre urinary drainage bag (for link drainage) and a lever tap or T-tap for controlled manual drainage.

The **Ugo 2L Drainage Bags** are a range of sterile and non-sterile, single use urinary drainage bags.

All the **Ugo 2L Drainage Bags** have a universal stepped connector and a range of drainage options.

Ugo 9, Ugo 10, Ugo 10 Xtra and Ugo 11 can only be drained once and are suitable for link drainage only.

Ugo 12 and Ugo 13 have a needle-free sample port and are suitable for attaching directly to a catheter for up to 7 days.

INGREDIENTS/COMPOSITION

- PVC – bag, inlet tube, outlet tube, non-return valve, sample port
- Polypropylene – lever tap body
- Polyethylene – lever and sample port cap
- Silicone – tubing
- Non-woven fabric backing (**Ugo 1C to Ugo 8C, Ugo Pro 1C to Ugo Pro 4C**)
- Nitrile gloves (**Ugo Pro 1C to Ugo Pro 4C**)

- Polyethylene apron (**Ugo Pro 1C to Ugo Pro 4C**)
- **Ugo Fix Leg Bag Straps** – see separate device IFU

INTENDED PURPOSE

The **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are for the collection, storage and drainage of urine.

They are suitable for use with indwelling urethral and suprapubic catheters and male urinary sheaths.

INTENDED USER

The intended users of the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are patients with an indwelling urethral catheter, suprapubic catheter or urinary sheath system, their carers and Healthcare Professionals.

CONTRAINDICATIONS

Do not use the **Ugo Leg Bags** and/or **Ugo 2L Drainage Bags** unless you have been assessed as suitable by a Healthcare Professional and they have been recommended for you;

Do not use the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** if you are sensitive or allergic to any of the device materials.

WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Leg Bags** and/or **Ugo 2L Drainage Bags** are suitable for the intended use and are compatible with other medical devices to be used in conjunction with them.
- Only use the **Ugo Leg Bags** and/or **Ugo 2L Drainage Bags** if you (or your carer) have the ability to operate them at regular intervals and can remember to do so.
- You should only disconnect your **Ugo Leg Bag** or **Ugo 2L Drainage Bag** from your catheter when you need to change it. Disconnecting the system unnecessarily increases your risk of introducing infection.
- Volume markings on the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are for guidance only and not for accurate measurement of urine.
- It is important that your urine bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are not twisted, and urine is draining freely.
- Always ensure the drainage tap of the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are not touching the floor.
- When emptying and changing your **Ugo Leg Bags** and **Ugo 2L Drainage Bags**, it is important that you do not touch the ends of the catheter or stepped connector, as this increases your risk of introducing infection.
- Always support the weight of your catheter when fitting the **Ugo Leg Bag** and/or **Ugo 2L Drainage Bag**.
- Never washout and reuse a **Ugo Leg Bag** or **Ugo 2L Drainage Bag** under any circumstances.
- The **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are not made with natural rubber latex.
- Do not use the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** if they are damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.

The **Ugo Leg Bags** and **Ugo 2L Drainage Bags** are single-use devices. Re-use of these devices may result in patient infection / cross-contamination. Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.

UNDESIRABLE SIDE EFFECTS

There may be an increased risk of complications from using the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** if you have decreased sensation or if they are not suitable for you (see warnings and precautions).

Seek help from your Healthcare Professional immediately if you experience any of the following when using

the **Ugo Leg Bags** and/or **Ugo 2L Drainage Bags**:

- Feeling unwell, high temperature, nausea/vomiting;
- Feeling of any pain or discomfort;
- Urine leaking around your catheter;
- Urine not draining into the urine bag;
- Cloudy or foul-smelling urine;
- Blood seen in urine;
- Changes to urine colour and volume.

In rare cases, local irritation/hypersensitivity reactions may occur such as redness, itching or blistering. If this occurs, stop using the **Ugo Leg Bags** and/or **Ugo 2L Drainage Bags** and consult your Healthcare Professional.

Any serious incident or malfunction that has occurred in relation to the **Ugo Leg Bags** and **Ugo 2L Drainage Bags** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare Professional for advice on where to wear your **Ugo Leg Bag** and/or **Ugo 2L Drainage Bag**.

Always wash and dry your hands before any after handling your **Ugo Leg Bag** or **Ugo 2L Drainage Bag**.

If someone else is handling your urine drainage bags, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

Warning! - Make sure your catheter and drainage tubing are not twisted and urine is draining freely after each episode of use.

How to change your Ugo Leg Bag

Where on your leg you choose to wear your **Ugo Leg Bag** will depend on many factors personal to you. These may include whether you have a urethral or suprapubic catheter, the length of your catheter and leg bag tubing, your leg bag volume and your level of mobility. In addition to this you may need to consider which clothes you prefer to wear and the normal daily activities which you take part in.


- Remove any straps or sleeve which are supporting your current leg bag to your leg;
- Remove the protective cap from the new **Ugo Leg Bag** stepped connector;
- Pinch off the funnel end of the catheter using your thumb and forefinger just below where the end of the old leg bag is inside the catheter;

- Detach the old leg bag from the catheter and immediately insert the stepped connector from the new **Ugo Leg Bag** ensuring an adequate seal (the leg bag is supplied with the tap in the closed position);
- The discarded protective cap from the new **Ugo Leg Bag** can be attached to the stepped connector of the old leg bag for disposal;
- Attach to your leg using the **Ugo Fix Leg Bag Straps** and adjust to ensure optimum drainage and comfort;

How to empty your Ugo Leg Bag

Empty your **Ugo Leg Bag** as frequently as recommended by your Healthcare Professional. This is usually when it is 2/3 full.

- Hold the outlet tap over a clean toilet or suitable clean receptacle (used only for your urine drainage);
- Open the tap by sliding the T tap or pushing the lever tap down. Allow the bag to drain completely (avoid contact between the drainage tap and the receptacle) then close the tap;

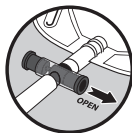T-tap

Lever tap

- Dry the end of the soft silicone outlet tubing with a clean wipe.

How to attach and remove your Ugo 2L Drainage Bag (link drainage)

If you need additional drainage, the **Ugo 2L Drainage Bag** can be connected to the soft silicone outlet tube on the end of your **Ugo Leg Bag** (this is known as link drainage). It is advised that you remove your **Ugo 2L Drainage Bag** when you are mobile and no longer in bed.

ATTACHING YOUR UGO 2L DRAINAGE BAG

- Remove any straps or sleeve which are supporting your **Ugo Leg Bag** to your leg;
- Remove the protective cap from the **Ugo 2L Drainage Bag** stepped connector;
- Insert the stepped connector from the **Ugo 2L Drainage Bag** into the soft silicone tubing on the **Ugo Leg Bag** (ensure an adequate seal);
- Attach the **Ugo 2L Drainage Bag** to the **Ugo Stand** or **Ugo Hanger**;
- Open the lever tap or T tap valve on your **Ugo Leg Bag** to allow immediate and continuous drainage into the **Ugo 2L Drainage Bag**.

REMOVING YOUR UGO 2L DRAINAGE BAG

- Close the lever tap or T tap on your **Ugo Leg Bag**;
- Disconnect the **Ugo 2L Drainage Bag** from the soft silicone tubing on your **Ugo Leg Bag** (make sure the silicone tubing is not disconnected with the bag and remains attached to the **Ugo Leg Bag**);

- Apply your leg bag fixation and support devices and adjust to ensure optimum drainage and comfort;
- Empty the urine from the used **Ugo 2L Drainage Bag** into the toilet or clean receptacle and dispose of the bag.

Obtaining a urine sample

Urine samples must only be taken using the needle-free sample port and in line with local healthcare policies and procedures.

- Disinfect sample port;
- Press the tip of a sterile luer slip syringe into sample port and it will hold in place;
- Obtain the necessary amount of urine;
- Twist and remove syringe in one movement once the sample is collected.

STERILE DEVICES

All the **Ugo Leg Bags** are supplied sterile. Sterilised by ethylene oxide after the packing process. **Ugo 11**, **Ugo 12** and **Ugo 13** are supplied sterile. Sterilised by ethylene oxide after the packing process.

Ugo 9, **Ugo 10** and **Ugo 10 Xtra** are supplied non-sterile. Do not re-sterilise.

STORAGE AND HANDLING

Keep dry and out of direct sunlight. Unopened packs have a 5-year shelf life. Do not use after expiry date.

DISPOSAL

The **Ugo Leg Bags** and **Ugo 2L Drainage Bags** must be disposed of according to local policies and waste disposal procedures, including any consumables and/or accessories used with the device.

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