ugofix™
gentle

Instructions for use

optimummedical®

 Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
LS2 9ET, Leeds, UK
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

PRODUCT REF / CONTENTS

Product Code	Description
3004	Ugo Fix Gentle


This is a product information leaflet for the user.

The **Ugo Fix Gentle** catheter clip has been prescribed/recommended for you, by a Healthcare Professional. Do not share it with anyone else. Read all the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek medical advice.

PRODUCT DESCRIPTION

The **Ugo Fix Gentle** catheter clip is a non-sterile, silicone-based securement medical device, used to provide fixation of a 2-way indwelling urinary catheter at the bifurcation of the catheter. The **Ugo Fix Gentle** features a rotating clip mounted on a re-adhesive silicone gel pad. The rotating catheter clip holds the catheter away from the skin and rotates, allowing the catheter to move naturally with the motion of the body. The re-adhesive silicone gel pad is applied to the skin, providing gentle adhesion to all skin types.

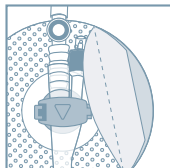COMPOSITION

Description	Material Composition
Adhesive silicone	Silicone
Catheter Clip	Polyvinyl Chloride
Release liner	Polyurethane film

INTENDED PURPOSE

The **Ugo Fix Gentle** catheter clip is a medical device intended to provide securement and fixation of 2-way indwelling urinary catheters, size 8CH to 20CH. The fixation of the catheter is intended to provide comfort and help reduce the risk of trauma caused by inadvertent movement of the catheter.

INTENDED USERS

The intended users of the **Ugo Fix Gentle** catheter clip are patients with a 2-way indwelling urinary catheter, in size 8CH to 20CH, and their carers and/or Healthcare Professionals.

CONTRAINDICATIONS

- Do not use the **Ugo Fix Gentle** catheter clip unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.
- Do not use if you are sensitive or allergic to any of the device materials.
- Do not use on skin which is broken or inflamed

WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Fix Gentle** catheter clip is used in line with local healthcare policies and procedures following a patient continence assessment by a Healthcare Professional.
- Ensure that the **Ugo Fix Gentle** catheter clip is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it. It may not be compatible with all 2-way indwelling urinary catheters. Check the operating instructions of the catheter in use.
- The **Ugo Fix Gentle** catheter clip is a non-sterile medical device.
- The **Ugo Fix Gentle** catheter clip is not designed to support the weight of a urine drainage bag. Always support the weight of your urine bag with an appropriate device.
- Do not disconnect your urine drainage bag during fitting of the **Ugo Fix Gentle** catheter clip. It is designed to allow fitting without disconnecting the system.
- Do not use moisturisers, moisturising soap, shower gel, wet wipes, baby wipes or talcum powder on the area of skin where the **Ugo Fix Gentle** catheter clip will be positioned, as they may affect the adhesion of the **Ugo Fix Gentle** catheter clip to the skin. It is important that you check your skin beneath the **Ugo Fix Gentle** catheter clip regularly, to ensure it remains intact.
- Do not shave where the **Ugo Fix Gentle** catheter clip is to be applied to the skin, as it may cause skin irritation. If necessary, any excess hair can be trimmed.
- It is important to check that your catheter is not twisted prior to insertion into the **Ugo Fix Gentle** catheter clip (see how to use) and

to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely (see undesirable side effects)

- Do not use the **Ugo Fix Gentle** catheter clip if it is damaged or soiled
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Fix Gentle** catheter clip not made with natural rubber latex.

This is a single-use medical device. Re-use of this medical device may result in patient infection and/or cross-contamination. Sterilisation, reprocessing, cleaning and/or disinfection, may also compromise the product characteristics, resulting in trauma or infection to the patient.

UNDESIRABLE SIDE EFFECTS

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Fix Gentle** catheter clip and consult your Healthcare Professional and ensure patient records are updated. There may be an increased risk of complications from wearing the **Ugo Fix Gentle** catheter clip if you have decreased sensation and if the **Ugo Fix Gentle** catheter clip has not been fitted correctly.

Remove the **Ugo Fix Gentle** catheter clip immediately if you experience any of the following during or after use, and seek medical advice:

- Swelling, inflammation or discomfort of the skin.
- Lesions or broken skin.
- Urine not draining into urine bag.
- Urine leaking outside the catheter.
- Blood in urine.
- Changes to urine colour or volume.
- Cloudy or foul-smelling urine.
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.

Any serious incident or malfunction that has occurred in relation to the **Ugo Fix Gentle** catheter clip should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare professional for advice before using the **Ugo Fix Gentle** catheter clip.

Always wash and dry your hands before and after handling your catheter or drainage tubing. If someone else is handling your catheter or drainage tubing, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

The **Ugo Fix Gentle** catheter clip can be used for up to 7 days depending on your lifestyle. We recommend that you change your **Ugo Fix Gentle** catheter clip to suit your personal hygiene and physical needs. Change it sooner if it becomes damaged or soiled or there are signs of degradation.

The **Ugo Fix Gentle** catheter clip can be worn in the shower to support your catheter. Ensure the edges are fully adhered to the skin prior to showering and pat dry the **Ugo Fix Gentle** catheter clip with a towel.

Always support the weight of your catheter and/or leg bag when fitting your **Ugo Fix Gentle** catheter clip.

How to Apply the Ugo Fix Gentle catheter clip:

Before Applying the Ugo Fix Gentle catheter clip to your skin:

It is important that you fix your catheter into the **Ugo Fix Gentle** catheter clip prior to applying to your skin.

Open clip (see illustrations on last page).

1. Push and lift the catch to open the **Ugo Fix Gentle** catheter clip lid.

Insert catheter (see illustrations on last page).

2. Place the bifurcation (the part of the catheter where it splits into two) part of the catheter into the **Ugo Fix Gentle** catheter clip. It will only fit one way.

WARNING - Make sure the catheter is not twisted before placing into the Ugo Fix Gentle catheter clip.

- Once in place, fix the **Ugo Fix Gentle** catheter clip lid closed by gently pressing the lid down.
- You will sense a click when the clip is closed correctly. Ensure the arrow on the clip is pointing towards your bladder.

Application of the Ugo Fix Gentle catheter clip to your skin

Now your catheter is fixed into the clip, you are ready to apply the **Ugo Fix Gentle** catheter clip to your skin. Your Healthcare Professional can advise on the most suitable position to place the **Ugo Fix Gentle** catheter clip.

- Remove backing (A) and apply to skin in desired position.
- Remove backing (B) and smooth in place.

IMPORTANT – Ensure the arrow on the clip is pointing towards your bladder.

We recommend an allowance of 2-4cm slack to enable natural movement of the catheter. If your catheter is pulling when you stand up or move around, you can reposition and reapply as necessary.

Removal of Ugo Fix Gentle from your skin and repositioning

You may want to reposition the **Ugo Fix Gentle** catheter clip between daytime and bedtime. You can wear your **Ugo Fix Gentle** catheter clip in bed and follow your usual overnight regime.

Your Healthcare Professional can advise on regular repositioning of the **Ugo Fix Gentle** catheter clip.

7. Gently lift the edges of the **Ugo Fix Gentle** catheter clip adhesive until it is fully removed taking care not to allow it to stick together.

8. Reapply to skin in new position and smooth down in place.

STORAGE AND HANDLING

Keep dry and out of direct sunlight.

Unopened packs have a 5-year shelf life.

Do not use after expiry date.

DISPOSAL

Ugo Fix Gentle catheter clip must be disposed of according to local policies and procedures, including any accessories or consumables used with the device.

MANUFACTURER INFORMATION

Optimum Medical Solutions Ltd, Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK.

Tel: +44 (0) 845 643 5479

Email: enquires@optimummedical.co.uk

Website: www.optimummedical.co.uk

AUTHORISED REPRESENTATIVE

MT Promedix Consulting GmbH,
Ernst-Heckel-Straße 7, 66386 St. Ingbert, Germany.

Tel: +49 6894 581020

Email: ear@mt-procoons.com

EU IMPORTER INFORMATION

MedEnvy Global B.V.
Prinses Margrietplantsoen - 33Suite 123
2595 AM The Hague
The Netherlands

SWITZERLAND AUTHORISED REPRESENTATIVE

MedEnvy Switzerland
Gotthardstrasse 2
6302 Zug
Switzerland

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MedEnvoy Global B.V.
MT Promedt Consulting GmbH
Optimum Medical Australasia Pty Ltd