

Washing Instructions: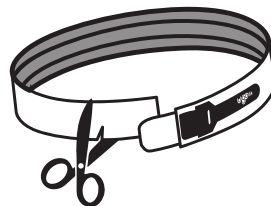

# ugofix.™

catheter strap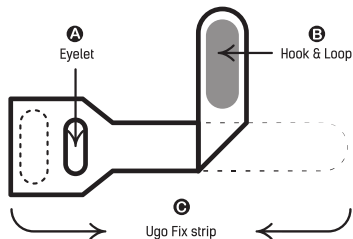

Sponsor:  
Optimum Medical Australasia  
Pty Limited, Level 4, 470 Church Street,  
North Parramatta, New South Wales, 2151, Australia

MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33-Suite 123  
2595 AM The Hague, The Netherlands

EC REP

MT Promedt Consulting GmbH,  
Ernst-Heckel-Straße 7,  
66386 St. Ingbert, Germany

Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, UK  
T: +44 (0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

3002-3003-3009-3010-IFU-JUL24-V8.0

## Instructions for use

## PRODUCT REF / CONTENTS

3002	Ugo Fix Catheter Strap - Short –Up to 45 cm
3003	Ugo Fix Catheter Strap - Medium – Up to 80 cm
3009	Ugo Fix Catheter Strap - Long –Up to 130 cm
3010	Ugo Fix Catheter Strap - Extra Long – Up to 180 cm

This is a product information leaflet for the user.

The **Ugo Fix Catheter Strap** has been prescribed/recommended for you by a Healthcare Professional. Do not share it with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

## DESCRIPTION

The **Ugo Fix Catheter Strap** is a single patient, multiple use, non-sterile, medical device for securing urinary catheters or urinary drainage bag tubing that is attached to indwelling urinary catheters or male urinary sheaths.

The **Ugo Fix Catheter Strap** consists of an adjustable elasticated strap, which is worn around the user's leg or lower abdomen and secured with hook and loop fastenings. The strap features silicone strips on the inner surface to help prevent movement on the skin. A PVC fixing strip is mounted on the strap which is used to secure the catheter or urinary drainage bag tubing in place with a hook and loop tab.

There are 4 lengths of the **Ugo Fix Catheter Strap**, and all lengths are adjustable (cut down to size) and suitable for securing catheters or urinary drainage bag tubing up to 10mm in diameter.

The **Ugo Fix Catheter Strap** is washable and reusable.

## INGREDIENTS/COMPOSITION

Silicone strip

Nylon hook

Elastic loop (polyester, nylon, spandex)

PVC Tie

## INTENDED PURPOSE

The **Ugo Fix Leg Catheter Strap** is intended to provide securement for indwelling urinary catheters or urinary drainage bag tubing.

This support is intended to provide comfort and reduce the risk of injury from the weight of your urinary leg bag pulling on your catheter or urinary sheath.

## INTENDED USER

The intended users of the **Ugo Fix Catheter Strap** are patients with indwelling urinary catheters or male urinary sheaths attached to urinary drainage bags, and their Carers and/or Healthcare Professionals.

## CONTRAINDICATIONS

- Do not use the **Ugo Fix Catheter Strap** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.
- Do not use it if you are sensitive or allergic to any of the device materials.
- Do not use on skin which is broken or inflamed.

## WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Fix Catheter Strap** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Fix Catheter Strap** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with them.
- Do not disconnect your urinary leg bag when fitting your **Ugo Fix Catheter Strap**. They are designed specifically to allow fitting without disconnecting the system.
- Always support the weight of your catheter and urinary leg bag when fitting your **Ugo Fix Catheter Strap**.
- The **Ugo Fix Catheter Strap** is not designed to support the weight of a urinary leg bag. Always support the weight of your urinary leg bag with an appropriate device.
- It is important to measure for the correct size. Allow for shrinkage after washing and drying. If the strap is too short, it can restrict circulation. (see how to use).
- Only apply the **Ugo Fix Catheter Strap** to skin which is clean and dry with no wounds or sores.
- The **Ugo Fix Catheter Strap** is elasticated. It is important that you visually check your skin regularly to ensure it remains healthy and intact and your strap is not too tight and marking your skin.
- It is important to check that your catheter or drainage bag tubing are not twisted prior to securing onto the **Ugo Fix Catheter Strap** (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely (see undesirable side effects).
- Do not use the **Ugo Fix Catheter Strap** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Fix Catheter Strap** is not made with natural rubber latex.
- The **Ugo Fix Catheter Strap** is a non-sterile medical device.
- The **Ugo Fix Catheter Strap** is a reusable device. Please refer to 'How to care for your device' to ensure the product performance is maintained.

## UNDESIRABLE SIDE EFFECTS

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Fix Catheter Strap** and consult your Healthcare Professional and ensure patient records are updated.

There may be an increased risk of complications from wearing the **Ugo Fix Catheter Strap** if you have decreased sensation or changes to your leg circumference (oedema) and if the **Ugo Fix Catheter Strap** has not been fitted correctly.

Remove the **Ugo Fix Catheter Strap** immediately if you experience any of the following during use and seek medical advice.

- Swelling, inflammation or discomfort of the skin.
- Pressure markings to the skin, lesions or broken skin.
- Discolouration to the skin or oedema.
- Urine not draining into urine bag.
- Urine leaking outside the catheter.
- Cloudy or foul smelling urine.
- Blood in urine.
- Changes to urine colour or volume.
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.

Any serious incident or malfunction that has occurred in relation to the **Ugo Fix Catheter Strap** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

## HOW TO USE

Consult your Healthcare Professional for advice before using the **Ugo Fix Catheter Strap**.

Always wash and dry your hands before and after handling your catheter or drainage bag tubing and **Ugo Fix Catheter Strap**. If someone else is handling your catheter or drainage bag tubing and catheter strap, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

We recommend that you change your **Ugo Fix Catheter Strap** daily along with your catheter hygiene regime. Only change them sooner if they are damaged or soiled.

The **Ugo Fix Catheter strap** is not waterproof; however you can still bathe and shower when wearing it to support your catheter. Replace the wet **Ugo Fix Catheter Strap** with a clean dry one afterwards.

### Identify the best place to wear the Ugo Fix Catheter Strap.

The **Ugo Fix Catheter Strap** can be worn on the lower abdomen or the leg. The position you wear your catheter strap will depend on whether you have a urethral or suprapubic catheter, the length of your catheter and leg bag tubing, your leg bag volume and your level of mobility. (see placement options)

### Ugo Fix Catheter Strap Placement Options

1. Using the **Ugo Fix Catheter Strap** at the catheter bifurcation.
2. Using the **Ugo fix Catheter Strap** on the lower abdomen to secure a superpubic catheter attached to a valve.
3. Using the **Ugo Fix Catheter Strap** on the leg bag tubing.

### Measuring and selecting the Correct Length of the Ugo Fix Catheter Strap

Using a measuring tape, measure around the lower abdomen or leg at the position you intend to wear your **Ugo Fix Catheter Strap** and select the correct length strap.

Then using the tape measure, measure the length of the **Ugo Fix Catheter Strap** and if

necessary, cut it down to size. (ensuring there is sufficient strap for hook and loop overlap).

**! Warning** – Do not cut the **Ugo Fix Catheter Strap** whilst it is being worn and stretched.

### How to apply your Ugo Fix Catheter Strap

#### Before securing the catheter or drainage bag tubing:

1. Place the **Ugo Fix Catheter Strap** around either the leg or lower abdomen in the desired position with the two silicone lines facing down on the skin. Ensure the strap is not too tight and the **Ugo Fix** strip is positioned where easily accessible.

#### Securing the catheter or drainage bag tubing

2. Lift the **Ugo Fix** strip and **(2a)** wrap it over and under the tubing and then **(2b)** through the eyelet.
3. Ensure the **Ugo Fix** strip is in the desired position and has a firm hold of the tubing **(3)** then secure in place onto the catheter strap using the hook and loop tab.

**! Warning** - Make sure your catheter or drainage bag tubing are not twisted before securing into the **Ugo Fix** strip. Ensure the catheter is not pulling when stood up and that you are able to move around comfortably with the **Ugo Fix Catheter Strap** in place.

#### Removal and repositioning

It is recommended that you remove the **Ugo Fix Catheter Strap** at bedtime. Your Healthcare Professional can advise on regular repositioning of the **Ugo Fix Catheter Strap**.

#### Using your urine drainage bag in conjunction with the Ugo Fix Catheter Strap

- Empty your urinary leg bag before it reaches 2/3 full.
- It is important that your urinary leg bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are free from obstruction and urine is draining freely.
- You do not need to remove your urinary leg bag from the **Ugo Fix Catheter Strap** to drain it.
- Ensure that the tap at the bottom of your urinary leg bag does not touch the floor.

#### CARING FOR YOUR DEVICE

- To maintain optimum performance of the **Ugo Fix Catheter Strap** we recommend that you wash the strap at 60°C. Tumble dry on a low heat. Do not iron. Do not bleach. Do not dry clean.
- Replace the **Ugo Fix Catheter Strap** when the elasticity and support is reduced from washing and stretching.
- The **Ugo Fix Catheter Strap** should no longer be used if there are any signs of damage or degradation.

#### STORAGE AND HANDLING

Keep dry and out of direct sunlight.

#### DISPOSAL

The **Ugo Fix Catheter Strap** must be disposed of according to local policies and procedures, including any consumables used with the device.

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