ugo.<sup>TM</sup>  
leg bag

ugo.<sup>TM</sup>  
drainage bags

Symbol Index

|                      |  |
|----------------------|--|
| All Products         |  |
|                      |  |
| Non-sterile products |  |
| Sterile products     |  |

MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33-Suite 123  
2595 AM The Hague, The Netherlands

MT Promedt Consulting GmbH,  
Ernst-Heckel-Straße 7,  
66386 St. Ingbert, Germany  
Tel: +49 6894 581020,  
Email: ear@mt-procons.com

Sponsor:  
Optimum Medical Australasia Pty Limited,  
Level 4, 470 Church Street, North Parramatta,  
New South Wales, 2151, Australia

optimummedical®



Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, UK  
T: +44 (0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

Ugo-Leg-Bags-IFU-Sept24-V8.0

ugo.<sup>TM</sup>  
leg bag

ugo.<sup>TM</sup>  
drainage bags

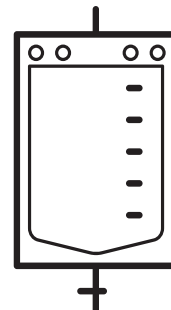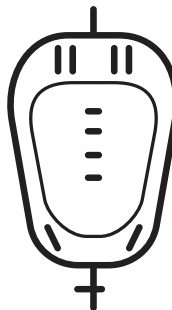## Instructions for use

optimummedical®

Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, UK  
T: +44 (0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

## PRODUCT REF / CONTENTS

## LEG BAGS

## COMMUNITY LEG BAGS (Sterile)

Box of 10 - non-woven fabric backing - 1 pair of **Ugo Fix Leg Bag Straps** (non-sterile) provided in every box.

| CODE        | PRODUCT NAME  | SIZE (ML) | LENGTH (CM)      | TAP TYPE  |
|-------------|---------------|-----------|------------------|-----------|
| 1C/500/ST/L | <b>Ugo 1C</b> | 500ml     | Short tube (5cm) | Lever tap |
| 2C/500/LT/L | <b>Ugo 2C</b> | 500ml     | Long tube (25cm) | Lever tap |
| 3C/500/ST/T | <b>Ugo 3C</b> | 500ml     | Short tube (5cm) | T tap     |
| 4C/500/LT/T | <b>Ugo 4C</b> | 500ml     | Long tube (25cm) | T tap     |
| 5C/350/DI/L | <b>Ugo 5C</b> | 350ml     | Direct inlet     | Lever tap |
| 6C/350/ST/L | <b>Ugo 6C</b> | 350ml     | Short tube (5cm) | Lever tap |
| 7C/750/ST/L | <b>Ugo 7C</b> | 750ml     | Short tube (5cm) | Lever tap |
| 8C/750/LT/L | <b>Ugo 8C</b> | 750ml     | Long tube (25cm) | Lever tap |

## COMMUNITY LEG BAGS (Sterile)

Box of 10 - non-woven fabric backing - 1 pair of **Ugo Fix Leg Bag Straps** (non-sterile) provided in every box.

| CODE | PRODUCT NAME      | SIZE (ML) | LENGTH (CM)      | TAP TYPE  | KIT CONTENTS                       |
|------|-------------------|-----------|------------------|-----------|------------------------------------|
| 3410 | <b>Ugo Pro 1C</b> | 500ml     | Short tube (5cm) | Lever tap | Apron and 1 pair of nitrile gloves |
| 3411 | <b>Ugo Pro 2C</b> | 500ml     | Long tube (25cm) | Lever tap | Apron and 1 pair of nitrile gloves |
| 3412 | <b>Ugo Pro 3C</b> | 500ml     | Short tube (5cm) | T tap     | Apron and 1 pair of nitrile gloves |
| 3413 | <b>Ugo Pro 4C</b> | 500ml     | Long tube (25cm) | T tap     | Apron and 1 pair of nitrile gloves |

## HOSPITAL LEG BAGS (Sterile)

Box of 10 - no fabric backing - 1 pair of **Ugo Fix Leg Bag Straps** provided in every pouch.

| CODE        | PRODUCT NAME  | SIZE (ML) | LENGTH (CM)      | TAP TYPE  |
|-------------|---------------|-----------|------------------|-----------|
| 1H/500/ST/L | <b>Ugo 1H</b> | 500ml     | Short tube (5cm) | Lever tap |
| 2H/500/LT/L | <b>Ugo 2H</b> | 500ml     | Long tube (25cm) | Lever tap |
| 3H/500/ST/T | <b>Ugo 3H</b> | 500ml     | Short tube (5cm) | T tap     |
| 4H/500/LT/T | <b>Ugo 4H</b> | 500ml     | Long tube (25cm) | T tap     |

## DRAINAGE BAGS

2 LITRE DRAINAGE BAGS - 90cm Inlet Tubing.

| CODE        | PRODUCT NAME  | USE                      | STERILITY   | PER BOX/BAG          |
|-------------|---------------|--------------------------|-------------|----------------------|
| 9/NS/SU/DO  | <b>Ugo 9</b>  | Non drainable/tear strip | Non-Sterile | Resealable bag of 10 |
| 10/NS/SU/DO | <b>Ugo 10</b> | Single-use T tap         | Non-Sterile | Resealable bag of 10 |
| 11/S/SU/DO  | <b>Ugo 11</b> | Single-use T tap         | Sterile     | Box of 10            |
| 12/S/D/L    | <b>Ugo 12</b> | Lever tap                | Sterile     | Box of 10            |
| 13/S/D/T    | <b>Ugo 13</b> | T tap                    | Sterile     | Box of 10            |

Xtra 2 LITRE DRAINAGE BAGS - 120cm Inlet Tubing.

| CODE | PRODUCT NAME       | USE              | STERILITY   | PER BOX/BAG          |
|------|--------------------|------------------|-------------|----------------------|
| 3300 | <b>Ugo 10 Xtra</b> | Single-use T tap | Non-Sterile | Resealable bag of 28 |

3 LITRE DRAINAGE BAGS - 120cm Inlet Tubing.

| CODE        | PRODUCT NAME  | USE              | STERILITY   | PER BOX/BAG          |
|-------------|---------------|------------------|-------------|----------------------|
| 14/NS/SU/DO | <b>Ugo 14</b> | Single use T tap | Non-Sterile | Resealable bag of 10 |
| 15/S/D/L    | <b>Ugo 15</b> | Lever tap        | Sterile     | Box of 10            |

## This is a product information leaflet for the user.

The urine bags have been recommended/prescribed for you by a Healthcare Professional. Do not share them with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

## PRODUCT DESCRIPTION

Each of the urine bags have different features to suit a range of clinical environments and individual patient needs (please see contents).

The **Ugo Leg Bags** are a range of sterile, single use urinary drainage bags designed to be worn on the leg for up to 7 days.

All the **Ugo Leg Bags** have a needle-free sample port, a universal stepped connector which is connected directly to the catheter or male urinary sheath, a soft silicone outlet tubing for direct drainage or connection to a **Ugo Drainage Bag** (for link drainage) and a lever tap or T tap for controlled manual drainage.

The **Ugo Drainage Bags** are a range of sterile and non-sterile, single use urinary drainage bags. All the **Ugo Drainage Bags** have a universal stepped connector and a range of drainage options.

**Ugo 9, Ugo 10, Ugo 10 Xtra and Ugo 14** can only be drained once and are suitable for link drainage only.

**Ugo 11** is suitable for attaching directly to a catheter or for link drainage; it can only be drained once.

**Ugo 12, Ugo 13 and Ugo 15** have a needle-free sample port and are suitable for attaching directly to a catheter for up to 7 days.

#### INGREDIENTS/COMPOSITION

- PVC – bag, inlet tube, outlet tube, non-return valve, sample port
- Polypropylene – lever tap body
- Polyethylene – lever and sample port cap
- Silicone – tubing
- Non-woven fabric backing (**Ugo 1C to Ugo 8C, Ugo Pro 1C to Ugo Pro 4C**)
- Nitrile gloves (**Ugo Pro 1C to Ugo Pro 4C**)
- Polyethylene apron (**Ugo Pro 1C to Ugo Pro 4C**)
- **Ugo Fix Leg Bag Straps** – see separate device IFU

#### INTENDED PURPOSE

The urine bags are for the collection, storage and drainage of urine. They are suitable for use with indwelling urethral and suprapubic catheters and male urinary sheaths.

#### INTENDED USER

The intended users of the urine bags are patients with an indwelling urethral catheter, suprapubic catheter or urinary sheath system, their carers and Healthcare Professionals.

#### CONTRAINDICATIONS

Do not use the urine bags unless you have been assessed as suitable by a Healthcare Professional and they have been recommended for you;

Do not use the urine bags if you are sensitive or allergic to any of the device materials.

#### WARNINGS AND PRECAUTIONS

- It is recommended that the urine bags are used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Leg Bags** and/or **Ugo Drainage Bags** are suitable for the intended use and are compatible with other medical devices to be used in conjunction with them.
- Only use the urine bags if you (or your carer) have the ability to operate them at regular intervals and can remember to do so.
- You should only disconnect your urine bag from your catheter when you need to change it. Disconnecting the system unnecessarily increases your risk of introducing infection.
- Volume markings on the urine bags are for guidance only and not for accurate measurement of urine.
- It is important that your urine bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are not twisted, and urine is draining freely.
- Always ensure the drainage tap of the urine bag is not touching the floor.
- The **Ugo 3L Drainage Bag** is not suitable to be used with a **Ugo Hanger**.
- When emptying and changing your urine bags, it is important that you do not touch the ends of the catheter or stepped connector, as this increases your risk of introducing infection.
- Always support the weight of your catheter when fitting the urine bag.
- Never washout and reuse a urine bag under any circumstances.
- The urine bags are not made with natural rubber latex.

- Do not use the urine bags if they are damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.

The urine bags are single-use devices. Re-use of these devices may result in patient infection / cross-contamination. Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.

#### UNDESIRABLE SIDE EFFECTS

There may be an increased risk of complications from using the urine bags if you have decreased sensation or if they are not suitable for you (see warnings and precautions).

Seek help from your Healthcare Professional immediately if you experience any of the following when using the urine bags:

- Feeling unwell, high temperature, nausea/vomiting;
- Feeling of any pain or discomfort;
- Urine leaking around your catheter;
- Urine not draining into the urine bag;
- Cloudy or foul-smelling urine;
- Blood seen in urine;
- Changes to urine colour and volume.

In rare cases, local irritation/hypersensitivity reactions may occur such as redness, itching or blistering. If this occurs, stop using the urine bags and consult your Healthcare Professional.

Any serious incident or malfunction that has occurred in relation to the urine bags should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

#### HOW TO USE

Consult your Healthcare Professional for advice on where to wear your urine bag.

Always wash and dry your hands before any after handling your urine bag.

If someone else is handling your urine drainage bags, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

**Warning!** - Make sure your catheter and drainage tubing are not twisted and urine is draining freely after each episode of use.

#### How to change your Ugo Leg Bag

Where on your leg you choose to wear your **Ugo Leg Bag** will depend on many factors personal to you. These may include whether you have a urethral or suprapubic catheter, the length of your catheter and leg bag tubing, your leg bag volume and your level of mobility. In addition to this you may need to consider which clothes you prefer to wear and the normal daily activities which you take part in.


- Remove any straps or sleeve which are supporting your current leg bag to your leg;
- Remove the protective cap from the new **Ugo Leg Bag** stepped connector;
- Pinch off the funnel end of the catheter using your thumb and forefinger just below where the end of the old leg bag is inside the catheter;
- Detach the old leg bag from the catheter and immediately insert the stepped connector from the new **Ugo Leg Bag** ensuring an adequate seal (the leg bag is supplied with the tap in the closed position);

- The discarded protective cap from the new **Ugo Leg Bag** can be attached to the stepped connector of the old leg bag for disposal;
- Attach to your leg using the **Ugo Fix Leg Bag Straps** and adjust to ensure optimum drainage and comfort;

### How to empty your Ugo Leg Bag

Empty your **Ugo Leg Bag** as frequently as recommended by your Healthcare Professional. This is usually when it is 2/3 full.

- Hold the outlet tap over a clean toilet or suitable clean receptacle (used only for your urine drainage);
- Open the tap by sliding the T tap or pushing the lever tap down. Allow the bag to drain completely (avoid contact between the drainage tap and the receptacle) then close the tap;

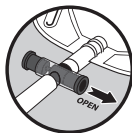T tap

Lever tap

- Dry the end of the soft silicone outlet tubing with a clean wipe.

### How to attach and remove your Ugo Drainage Bag (link drainage)

If you need additional drainage, the **Ugo Drainage Bag** can be connected to the soft silicone outlet tube on the end of your **Ugo Leg Bag** (this is known as link drainage).

It is advised that you remove your **Ugo Drainage Bag** when you are mobile and no longer in bed.

### ATTACHING YOUR UGO DRAINAGE BAG

- Remove any straps or sleeve which are supporting your **Ugo Leg Bag** to your leg;
- Remove the protective cap from the **Ugo Drainage Bag** stepped connector;
- Insert the stepped connector from the **Ugo Drainage Bag** into the soft silicone tubing on the **Ugo Leg Bag** (ensure an adequate seal);
- Attach the **Ugo 2L Drainage Bag** to the **Ugo Stand** or **Ugo Hanger**;
- Attach the **Ugo 3L Drainage Bag** to the **Ugo Stand** without the **Dignity Cover** [it is not suitable to be used with a **Ugo Hanger**];
- Open the lever tap or T tap valve on your **Ugo Leg Bag** to allow immediate and continuous drainage into the **Ugo Drainage Bag**.

### REMOVING YOUR UGO DRAINAGE BAG

- Close the lever tap or T tap on your **Ugo Leg Bag**;
- Disconnect the **Ugo Drainage Bag** from the soft silicone tubing on your **Ugo Leg Bag** (make sure the silicone tubing is not disconnected with the bag and remains attached to the **Ugo Leg Bag**);
- Apply your leg bag fixation and support devices and adjust to ensure optimum drainage and comfort;

- Empty the urine from the used **Ugo Drainage Bag** into the toilet or clean receptacle and dispose of the bag.

### Obtaining a urine sample

Urine samples must only be taken using the needle-free sample port and in line with local healthcare policies and procedures.

- Disinfect sample port;
- Press the tip of a sterile luer slip syringe into sample port and it will hold in place;
- Obtain the necessary amount of urine;
- Twist and remove syringe in one movement once the sample is collected.

### STERILE DEVICES

All the **Ugo Leg Bags** are supplied sterile. Sterilised by ethylene oxide after the packing process. Do not re-sterilise.

**Ugo 11, Ugo 12, Ugo 13 and Ugo 15** are supplied sterile. Sterilised by ethylene oxide after the packing process. Do not re-sterilise.

**Ugo 9, Ugo 10, Ugo 10 Xtra and Ugo 14** are supplied non-sterile. Do not sterilise.

### STORAGE AND HANDLING

Keep dry. Keep out of direct sunlight. Unopened packs have a 5-year shelf life. Do not use after expiry date.

### DISPOSAL

The urine bags must be disposed of according to local policies and waste disposal procedures, including any consumables and/or accessories used with the device.

## Ugo Leg Bags (Poches de jambe Ugo) et Ugo Drainage Bags (Poches de drainage Ugo)

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