UK  
CA

REF

1157

REF

1159

LATEX FREE

MD

PAPER ONLY

EC REP

MT Promedt Consulting GmbH,  
Ernst-Heckel-Straße 7,  
66386 St. Ingbert, Germany

MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33 - Suite 123  
2595 AM The Hague  
The Netherlands

Sponsor:  
Optimum Medical Australasia Pty Ltd  
Level 4 470 Church Street,  
North Parramatta, NSW 2151, Australia

1157-1159-IFU-Mar25-V6.0

optimummedical®

**Optimum Medical Solutions Ltd**


Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom

T: +44(0) 845 643 5479

enquiries@optimummedical.co.uk

www.optimummedical.co.uk

**vue.**<sup>TM</sup>  
ultrasound gel

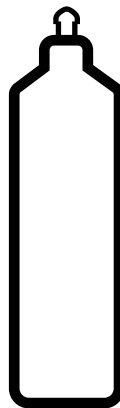Instructions for use

This is a product information leaflet for the user.

Read all of the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek medical advice.

#### PRODUCT DESCRIPTION:

**Vue Ultrasound Gel** is a non-sterile, water-based gel.

#### INGREDIENTS/COMPOSITION

- Deionised water
- Carbomer
- Propylene glycol
- Triethanolamine
- Benzyl alcohol
- Methylchloroisothiazolinone
- Methylisothiazolinone

#### INTENDED PURPOSE:

**Vue Ultrasound Gel** is a medical device intended to be used for the transmission of ultrasonic waves during low-risk examinations in areas involving intact skin; in examinations that do not involve invasive procedures; or more than 24 hours prior to a probable invasive procedure at or near the same site.

#### INDICATIONS FOR USE / INTENDED USER:

**Vue Ultrasound Gel** is a medical device intended to be used by Healthcare Professionals when performing ultrasound.

#### CONTRAINDICATIONS:

- Do not use for invasive procedures, that is any ultrasound-guided procedure that involves passing a device through skin into sterile tissue, such as intravenous line insertion or fine needle aspirate;
- Do not use if an invasive procedure is likely to be undertaken in the following 24 hours – this includes 'viewing or initial assessment' of a site by ultrasound prior to undertaking an (aseptic) invasive procedure;
- Do not use in labour where there is high likelihood of C-section or invasive instrumentation during delivery;
- Do not use where there is contact with or near to non-intact skin (any alteration in skin integrity such as a rash or surgical wound, including umbilicus in neonates);
- Do not use where the ultrasound examination is near to an indwelling invasive device, such as an intravenous line or suprapubic catheter;
- Do not use where there is contact with a mucous membrane (for example for transrectal, transvaginal or ophthalmic procedures);
- Do not use for examinations on severely immunocompromised individuals, in an intensive-care setting, high-dependency, or equivalent unit/s, including neonatal intensive care units;
- Do not use if the patient is sensitive or allergic to any of the ingredients;

- Not suitable for use with a defibrillator, ECG or laser treatment.

#### WARNINGS AND PRECAUTIONS:

- It is recommended that **Vue Ultrasound Gel** is used in line with local healthcare policies and procedures.
- Ensure that **Vue Ultrasound Gel** is suitable for the intended use and compatible with other medical devices to be used in conjunction with it.
- Do not use if the bottle is out of date, damaged, soiled or the seal is broken prior to first use.
- Do not refill. Reuse or refilling of the bottle may result in infection and/or cross-contamination.
- Do not decant into smaller bottles as there is a risk of contamination.
- Clean the whole bottle, including the tip, with a disinfectant wipe between uses.
- The tip/nozzle of bottle should not come into contact with anything – if it does, clean immediately with a disinfectant wipe.
- If an invasive procedure is subsequently undertaken within 24 hours of the use of non-sterile gel at or near to the site, then ensure all residual gel is removed, and the skin is thoroughly cleaned using antiseptic skin preparation in line with local policy for the procedure.
- For external use only.
- Not to be ingested.
- Avoid contact with the eyes.
- Keep out of reach of children.
- **Vue Ultrasound Gel** is not made with natural rubber latex.

#### UNDESIRABLE SIDE EFFECTS:

There are no known side effects associated with using **Vue Ultrasound Gel**, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using

**Vue Ultrasound Gel** and ensure patient records are updated.

Any serious incident or malfunction that has occurred in relation to the **Vue Ultrasound Gel** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

#### HOW TO USE:

Prior to use, label the bottle with the date first opened on the space provided. Once opened, dispose of the bottle when either empty, after one month or on expiry date, whichever comes first (see warnings and precautions). Close cap between uses.

Always wash and dry your hands before and after using **Vue Ultrasound Gel** and apply a clean pair of gloves on every occasion to avoid cross infection.

1. Clean the patient's skin prior to applying **Vue Ultrasound Gel**.
2. Follow the instructions supplied by the manufacturer of the equipment being used to perform the medical ultrasound procedure.
3. Apply the required amount of **Vue Ultrasound Gel** to the area of skin/equipment.
4. Following the procedure, remove any remaining ultrasound gel from the patient's

skin using personal hygiene wipes or wash off using patient dry wipes and soap and water.

5. Dry the patient's skin thoroughly afterwards with paper towels.

#### **STORAGE AND HANDLING:**

Store between 5°C and 30°C. Keep dry. Keep out of direct sunlight. Unopened packs have a 5-year shelf life. Do not use after expiry date.

#### **DISPOSAL:**

**Vue Ultrasound Gel** must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

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