**GBR****INSTRUCTIONS FOR USE**

JACKET[®]

optilube[™]

GEL LUBRIFIANT
LUBRICATING GEL

Instructions for use
Instructions d'utilisation

PRODUCT REF / CONTENTS COMPOSITION:**REF: 1180** – 6ml sachet.

This is a product information leaflet for the user. Read all of the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

PRODUCT DESCRIPTION

OptiLube Jacket is a water-soluble, sterile lubricating gel compatible for use with condoms.

INGREDIENTS/COMPOSITION

Purified water, polyethylene glycol, glycerine, carbomer, sodium hydroxide, 4-Hydroxybenzoic acid esters.

INTENDED PURPOSE / USE

OptiLube Jacket is intended to provide additional lubrication during sexual activity. The gel complies with ISO 19671:2018 for compatibility with natural rubber condoms.

CONTRAINDICATIONS

Do not use if you are sensitive or allergic to any of the device ingredients.

WARNINGS AND PRECAUTIONS

OptiLube Jacket is not a contraceptive. It will not prevent sexually transmitted infections (STIs) or reduce the risk of pregnancy. Do not use if packaging is damaged/soiled or unintentionally opened prior to use. Keep out of reach of children. This is a single-use device. Re-use of this device may result in patient infection / cross-contamination. Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics. **OptiLube Jacket** is not made with natural rubber latex.

UNDESIRABLE SIDE EFFECTS

There are no known side effects associated with using **OptiLube Jacket**, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using **OptiLube Jacket** and consult your Healthcare Professional. If symptoms suddenly progress or become worse, which may include feeling lightheaded or faint, breathing difficulties, a fast heartbeat, confusion and anxiety, seek medical help immediately.

Any serious incident or malfunction that has occurred in relation to **OptiLube Jacket** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Before penetration, place the gel at the entrance of the orifice or on the condom.

STERILE DEVICES

OptiLube Jacket is supplied sterile. **OptiLube Jacket** is sterilised by gamma irradiation after the packaging process. Do not re-sterilise.

STORAGE AND HANDLING

Keep dry and out of direct sunlight. Unopened packs have a 5-year shelf life. Do not use after expiry date. Store between 5°C and 40°C.

DISPOSAL

OptiLube Jacket should be disposed of according to local policies and waste disposal procedures.

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Optimum Medical Solutions Ltd,
MT Promedt Consulting GmbH,
Medenvoy Global B.V.