optipro.TM
pre-injection isopropyl wipes

REF 1102

REF 1102HK

CH REP

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug, Switzerland

EC REP

MT Promedt Consulting GmbH
Ernst-Heckel-Straße 7
66386 St. Ingbert Germany

MedEnvoy Global B.V.
Prinses Margrietplantsoen -33Suite 123
2595 AM The Hague
The Netherlands


optimummedical®

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

1102-HFU-Apr25-V02.0

optipro.TM
pre-injection isopropyl wipes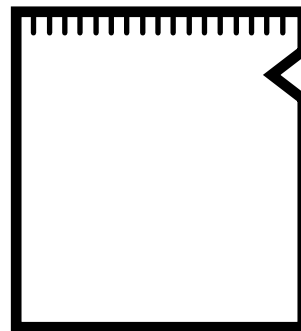

Instructions for use

PRODUCT REF / CONTENTS

1102	OptiPro 70% isopropyl alcohol pre-injection wipes.	100 Sachets
1102HK	OptiPro 70% isopropyl alcohol pre-injection wipes.	200 Sachets

This is a product information leaflet for the user. Read all of the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

INGREDIENTS/COMPOSITION

Non-woven fabric (polyester and viscose) impregnated with 70% isopropyl alcohol and purified water.

INTENDED PURPOSE

OptiPro 70% isopropyl alcohol pre-injection wipes are for skin cleansing prior to injection.

INTENDED USER

The intended users of the **OptiPro** 70% isopropyl alcohol pre-injection wipes are Healthcare Professionals on patients or the general public.

CONTRAINDICATIONS

- Do not use **OptiPro** 70% isopropyl alcohol pre-injection wipes if the patient is sensitive or allergic to the ingredients.
- Do not use on skin which is broken or inflamed.

WARNINGS AND PRECAUTIONS

- It is recommended that **OptiPro** 70% isopropyl alcohol pre-injection wipes in line with local healthcare policies and procedures.
- Ensure that **OptiPro** 70% isopropyl alcohol pre-injection wipes are suitable for the intended use and compatible with other medical devices to be used in conjunction with it.
- The **OptiPro** 70% isopropyl alcohol pre-injection wipes are a non-sterile medical device.
- For external use only.
- Keep out of reach of children.
- **OptiPro** 70% isopropyl alcohol pre-injection wipes are not made with natural rubber latex.
- Do not use **OptiPro** 70% isopropyl alcohol pre-injection wipes if the packaging is damaged or soiled.
- Highly flammable; keep away from fire.
- Avoid contact with the eyes or broken skin.
- In cases of irritation discontinue use.
- Do not swallow; if swallowed seek medical advice.
- This is a single-use medical device. Re-use of this device may result in patient infection / cross-contamination. Re/sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in the product being ineffective.

UNDESIRABLE SIDE EFFECTS

In rare cases, local irritation/hypersensitivity reactions may occur such as redness, itching or blistering. If this occurs, stop using the **OptiPro** 70% isopropyl alcohol pre-injection wipes and consult your Healthcare Professional.

Any serious incident or malfunction that has occurred in relation to the **OptiPro** 70% isopropyl alcohol pre-injection wipes should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare Professional for advice before using the **OptiPro** 70% isopropyl alcohol pre-injection wipes. Do not use if packaging is damaged or unintentionally opened prior to use. Remove wipes from sachet. Apply the wipes to the intended area to cleanse the skin prior to injection.

- Wash hands and open **OptiPro** 70% isopropyl alcohol pre-injection wipes sachet.
- Apply the wipes to the intended area of skin for 30 seconds to cleanse.
- Wait for 30 seconds for skin to dry before injection.

STORAGE AND HANDLING

- Keep dry and out of direct sunlight;
- Unopened packs have a 3-year shelf life;
- Do not use after expiry date.

DISPOSAL

OptiPro 70% isopropyl alcohol pre-injection wipes must be disposed of according to local policies and waste disposal procedures, including any accessories or consumables used with the device.

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