REF **3000**

CE 2797 **STERILE EO**

**UK
CA**
0086

LATEX FREE

3000-IFU-Apr25-V8.0


MedEnvoy Global B.V.
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague, The Netherlands

EC REP

MT Promedt Consulting GmbH
Ernst-Heckel-Straße 7
66386 St. Ingbert, Germany

Sponsor:
Optimum Medical Australasia
Pty Limited, Level 4, 470 Church
Street, North Parramatta
New South Wales 2151, Australia

ugo.TM
catheter valve

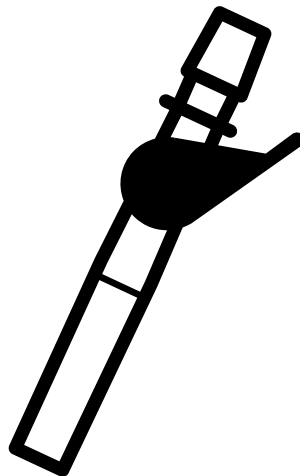Instructions for use

optimummedical®

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

PRODUCT REF / CONTENTS

Product Code	Description
3000	Ugo Catheter Valve

This is a product information leaflet for the user.

The **Ugo Catheter Valve** has been prescribed/recommended for you, by a Healthcare Professional. Do not share it with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice.

PRODUCT DESCRIPTION

The **Ugo Catheter Valve** is a small, sterile, single use drainage valve used to provide intermittent urinary drainage from an indwelling urethral or suprapubic catheter. The **Ugo Catheter Valve** has a universal stepped connector which is connected directly to the catheter, a soft silicone outlet tube for direct drainage or connection to a urinary drainage bag, and a lever for controlled manual drainage.

COMPOSITION

Description	Material Composition
Catheter Valve	Polypropylene/Polyethylene
Outlet tube	Silicone

INTENDED PURPOSE

The **Ugo Catheter Valve** enables drainage of urine from the bladder to be intermittent and controlled. The **Ugo Catheter Valve** should only be used by someone who has been assessed as suitable by a Healthcare Professional (see contraindications and warnings). Intermittent drainage of urine allows the bladder to fill and empty, helping to maintain bladder muscle tone and capacity.

The **Ugo Catheter Valve** is a discreet alternative drainage option to urine drainage bags or can be used as part of a urinary link drainage.

INTENDED USER/S

The intended users of the **Ugo Catheter Valve** are patients with an indwelling urinary catheter (who have been assessed as suitable by a Healthcare Professional), their Carers and/or Healthcare Professionals.

CONTRAINDICATIONS

- Do not use the **Ugo Catheter Valve** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.
- Do not use the **Ugo Catheter Valve** if you are sensitive or allergic to any of the device ingredients.

The **Ugo Catheter Valve** may be contraindicated or not be suitable for individuals who:

- Are unable to operate the valve (unless a carer is able to perform this).
- Are unable to remember to operate the valve.
- Have a small bladder capacity.
- Have reduced bladder sensation.
- Have poor renal function or ureteric reflux.
- Have uncontrolled bladder overactivity.
- Have had recent bladder or prostate surgery.

WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Catheter Valve** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Catheter Valve** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it.
- Only use the **Ugo Catheter Valve** if you (or your carer) have the ability to operate it at regular intervals and can remember to do so.
- You should only disconnect your **Ugo Catheter Valve** from your catheter when you need to change it. Disconnecting the system unnecessarily increases your risk of introducing infection.
- Do not use the **Ugo Catheter Valve** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- When emptying and changing your **Ugo Catheter Valve**, it is important that you do not touch the ends of the catheter or the valve, as this increases your risk of introducing infection.
- Always support the weight of your catheter when fitting the **Ugo Catheter Valve** and secure your catheter between draining to prevent discomfort.
- Keep out of reach of children.
- The **Ugo Catheter Valve** is not made with natural rubber latex.
- This is a single-use medical device. Re-use of this medical device may result in patient infection and/or cross-contamination. Sterilisation, reprocessing, cleaning and/or disinfection, may also compromise the product characteristics, resulting in trauma or infection to the patient.

UNDESIRABLE SIDE EFFECTS

There may be an increased risk of complications from using the **Ugo Catheter Valve** if you have decreased sensation or you are not suitable to use the **Ugo Catheter Valve** (see contraindications).

Drain your **Ugo Catheter Valve** immediately if you experience any of the following during

or after use, and seek medical advice:

- Feeling unwell, high temperature, nausea/vomiting.
- Feeling of any pain or discomfort.
- Urine leaking around your valve or catheter.
- Urine not draining from the valve or into urine bag.
- Cloudy or foul-smelling urine.
- Blood in urine.
- Changes to urine colour or volume.

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Catheter Valve** and consult your Healthcare Professional and ensure patient records are updated.

Any serious incident or malfunction that has occurred in relation to the **Ugo Catheter Valve** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare Professional for advice before using the **Ugo Catheter Valve**.

Always wash and dry your hands before and after handling your **Ugo Catheter Valve**.

If someone else is handling your catheter valve, they must apply a clean pair of gloves after drying their hands.

How to change your Ugo Catheter Valve:

Change your **Ugo Catheter Valve** every 7 days. Only change it sooner if it is damaged or soiled.

1. Drain your bladder fully using the old valve which is in place.
2. Pinch off the funnel end of the catheter using your thumb and forefinger just below where the end of the old valve is inside the catheter.
3. Remove the old valve and immediately attach the new sterile valve
(Note: **Ugo Catheter Valves** are supplied in a closed position).

How to drain your bladder using your Ugo Catheter Valve:

Drain your bladder using the **Ugo Catheter Valve** as frequently as recommended by your Healthcare Professional. This is usually every 2-4 hours.

1. Hold the **Ugo Catheter Valve** over a toilet or suitable receptacle.
2. Open the valve by pushing the lever tap fully into the downward position (you may feel some mild tension in the lever at 45 degrees, this is normal - it is the safety feature working to prevent accidental opening).
3. Allow the valve to drain the bladder completely (the flow may be slow – this is due to the narrow lumen inside the catheter).
4. Once urine has stopped draining, close the valve by lifting the lever tap fully into the upright position.
5. Dry the end of the valve with a clean wipe.

How to attach a drainage bag to your Ugo Catheter Valve:

If you use a drainage bag, then this can be connected directly to the soft silicone outlet tube on the end of your **Ugo Catheter Valve**.

1. Insert the stepped connector end of the drainage bag into the soft silicone outlet tube on the **Ugo Catheter Valve**.
2. Check the connection seal is adequate.
3. Open the catheter valve (See “How to drain your bladder using your catheter valve” for guidance on opening your catheter valve) and this will allow immediate continuous drainage of urine into the night bag.

WARNING: Make sure your catheter and drainage tubing are not twisted and urine is draining freely from your catheter into the drainage bag.

How to remove a drainage bag from your Ugo Catheter Valve:

1. Close the **Ugo Catheter Valve** using the lever.
2. Disconnect the drainage bag from the soft silicone tubing on the **Ugo Catheter Valve** (Make sure the silicone tubing is not disconnected with the bag).
3. Empty the urine from the drainage bag into the toilet or suitable receptacle.

STERILE DEVICES

The **Ugo Catheter Valve** is supplied sterile.

The **Ugo Catheter Valve** is sterilised by ethylene oxide after the packaging process.

Do not re-sterilise.

STORAGE AND HANDLING

Keep dry.

Keep away from sunlight.

Unopened packs have a 5-year shelf life.

Do not use after expiry date.

DISPOSAL

The **Ugo Catheter Valve** must be disposed of according to local policies and procedures, including any accessories or consumables used with the device.

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NOTES

This image shows a full page of white paper with horizontal dashed lines, typical of primary-ruled notebook paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

NOTES

[illegible]

NOTES

[illegible]

NOTES

This image shows a full page of white paper with horizontal dashed lines, typical of primary school writing paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.