ugofix™  
leg bag straps

REF

3011

MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33 - Suite 123  
2595 AM The Hague  
The Netherlands

EU REP

MT Promedt Consulting GmbH,  
Ernst-Heckel-Straße 7,  
66386 St. Ingbert, Germany  
Tel: +49 6894 581020  
Email: ear@mt-procons.com

Sponsor:  
Optimum Medical Australasia Pty Limited,  
Level 4, 470 Church Street, North Parramatta, New South Wales,  
2151, Australia


optimummedical®

 Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, UK  
T: +44 (0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

ugofix™  
leg bag straps

3011-IFU-Ju25-V7.0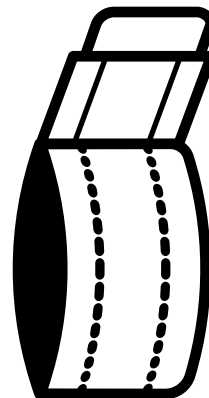

## Instructions for use

optimummedical®

 Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
LS2 9ET, Leeds, UK  
T: +44(0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

**PRODUCT REF / CONTENTS****3011 Ugo Fix Leg Bag Straps**

This is a product information leaflet for the user.

The **Ugo Fix Leg Bag Straps** have been prescribed/recommended for you by a Healthcare Professional. Do not share them with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

**DESCRIPTION**

The **Ugo Fix Leg Bag Straps** are a single patient, multiple use, non-sterile, medical device for supporting the weight of a urinary leg bag that is attached to an indwelling urinary catheter or male urinary sheath.

The **Ugo Fix Leg Bag Straps** consist of a pair of elasticated straps, which are used to support urinary leg bags to the user's thigh, upper knee or shin, and are secured with a hook and loop fastening. The straps feature silicone strips on the inner surface to help prevent movement on the skin.

The **Ugo Fix Leg Bag Straps** measure 75cm and 45cm. They can be used as a pair or joined to form one strap and are adjustable and can be cut down to fit if necessary. They are suitable for supporting urinary leg bags from 350ml to 750ml volumes, through the eyelet holes in the top and bottom band of the urinary leg bag.

The **Ugo Fix Leg Bag Straps** are washable and reusable.

**INGREDIENTS/COMPOSITION**

Silicone strip

Nylon hook

Elastic loop (polyester, nylon, spandex)

**INTENDED PURPOSE**

The **Ugo Fix Leg Bag Straps** provide support for the weight of your urinary leg bag, allowing it to fill whilst worn on the leg, enabling mobility.

This support is intended to provide comfort and reduce the risk of injury from the weight of your urinary leg bag pulling on your catheter or male urinary sheath.

**INTENDED USER**

The intended users of the **Ugo Fix Leg Bag Straps** are patients with urinary leg bags attached to indwelling urinary catheters or male urinary sheaths, and their Carers and/or Healthcare Professionals.

**CONTRAINDICATIONS**

- Do not use the **Ugo Fix Leg Bag Straps** unless you have been assessed as suitable by a Healthcare Professional and they have been recommended for you.
- Do not use the if you are sensitive or allergic to any of the device materials.

- Do not use on skin which is broken or inflamed.

**WARNINGS AND PRECAUTIONS**

- It is recommended that the **Ugo Fix Leg Bag Straps** are used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Fix Leg Bag Straps** are suitable for the intended use and are compatible with other medical devices to be used in conjunction with them.
- Do not disconnect your urinary leg bag when fitting your **Ugo Fix Leg Bag Straps**. They are designed specifically to allow fitting without disconnecting the system.
- Always support the weight of your catheter and urinary leg bag when fitting your **Ugo Fix Leg Bag Straps**.
- It is important that your catheter is also secured to prevent inadvertent movement, in addition to using the **Ugo Fix Leg Bag Straps**.
- Ensure you measure around your thigh, upper knee or lower leg at the position you intend to wear your **Ugo Fix Leg Bag Straps** before cutting to length. Allow for shrinkage after washing and drying. If the straps are too short, they can restrict circulation (see How to use).
- Only apply the **Ugo Fix Leg Bag Straps** to skin which is clean and dry with no wounds or sores.
- The **Ugo Fix Leg Bag Straps** are elasticated. It is important that you visually check your skin regularly to ensure it remains healthy and intact and your straps are not too tight and marking your skin.
- It is important to check that your catheter or drainage bag tubing are not twisted prior to applying the **Ugo Fix Leg Bag Straps** to your leg (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely (see undesirable side effects).
- Do not use the **Ugo Fix Leg Bag Straps** if they are damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Fix Leg Bag Straps** are not made with natural rubber latex.
- The **Ugo Fix Leg Bag Straps** are a non-sterile medical device.
- The **Ugo Fix Leg Bag Straps** are a reusable device. Please refer to 'How to care for your device' to ensure the product performance is maintained.

**UNDESIRABLE SIDE EFFECTS**

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Fix Leg Bag Straps** and consult your Healthcare Professional and ensure patient records are updated.

There may be an increased risk of complications from wearing the **Ugo Fix Leg Bag Straps** if you have decreased sensation or changes to your leg circumference (oedema) and if the **Ugo Fix Leg Bag Straps** have not been fitted correctly.

Remove the **Ugo Fix Leg Bag Straps** immediately if you experience any of the following during use and seek medical advice:

- Swelling, inflammation or discomfort of the skin;
- Pressure markings to the skin, lesions or broken skin;
- Discolouration to the skin or oedema;
- Urine not draining into urine bag;
- Urine leaking outside the catheter;

- Cloudy or foul smelling urine;
- Blood in urine;
- Changes to urine colour or volume;
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.

Any serious incident or malfunction that has occurred in relation to the **Ugo Fix Leg Bag Straps** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

## HOW TO USE

Consult your Healthcare Professional for advice before using the **Ugo Fix Leg Bag Straps**.

Always wash and dry your hands before and after handling your catheter or drainage bag tubing and straps. If someone else is handling your catheter or drainage bag tubing and straps, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

We recommend that you change your **Ugo Fix Leg Bag Straps** daily along with your catheter hygiene regime. Only change them sooner if they are damaged or soiled.

The **Ugo Fix Leg Bag Straps** are not waterproof; however you can still bathe and shower when wearing them to support your leg bag. Replace the wet **Ugo Fix Leg Bag Straps** with a clean dry pair afterwards.

### Identify the best place to wear the Ugo Fix Leg Bag Straps

The **Ugo Fix Leg Bag Straps** can be used to support your urinary leg bag on your thigh, upper knee or lower leg. The position you wear your leg bag will depend on whether you have a urethral or suprapubic catheter, the length of your catheter and leg bag tubing, your leg bag volume and your level of mobility.

### Measuring and adjusting the Ugo Fix Leg Bag Straps to the correct length

1. Using a tape measure, measure around your leg at the position you intend to wear your **Ugo Fix Leg Bag Straps**.
2. Then using the tape measure, measure the length of the **Ugo Fix Leg Bag Straps**, add at least 5cm for hook and loop overlap and cut to length.

**! Warning** – Do not cut the **Ugo Fix Leg Bag Straps** whilst being worn and stretched.

### How to apply your Ugo Fix Leg Bag Straps

Fitting options:

**Method A** – Using both straps to attach your urinary leg bag to your thigh or shin.

1. Feed the long **Ugo Fix Leg Bag Strap** (75cm) through the 4 holes in the top of your urinary leg bag and the short **Ugo Fix Leg Bag Strap** (45cm) through the bottom 2 holes.
2. Ensure the silicone line will be facing down onto your skin;
3. Attach the straps around your thigh or shin in the desired position and use the hook and loop tabs to secure in place (it is usually easiest to fit the top strap first).

**Method B** – Using the long strap to attach your urinary leg bag to your upper knee.

1. Feed the long **Ugo Fix Leg Bag Strap** (75cm) through the 4 holes in the top of your urinary leg bag;

2. Ensure the silicone line will be facing down onto your skin;
3. Take both ends of the **Ugo Fix Leg Bag Strap** and pass around your thigh just above your knee. Cross the strap at the back of the knee;
4. Bring the two ends of the strap to the front of your leg and feed through the bottom 2 holes of your urinary leg bag, attaching with the hook and loop tab behind the bag.

If the long **Ugo Fix Leg Bag Strap** (75cm) is not long enough for Method B, you can attach it to the short **Ugo Fix Leg Bag Strap** (45cm) using the hook and loop tab for extra length before attaching to your leg.

**! Warning** - Make sure your catheter or drainage bag tubing is not twisted before supporting your urinary leg bag to your leg using the **Ugo Fix Leg Bag Straps**.

Ensure your urinary leg bag is supported, the **Ugo Fix Leg Bag Straps** are not too tight, your catheter is not pulling and that you are able to move around comfortably with it in place.

### Removal and repositioning

It is recommended that you remove the **Ugo Fix Leg Bag Straps** at bedtime. Your Healthcare Professional can advise on regular repositioning of the **Ugo Fix Leg Bag Straps**.

### Using your urine drainage bag in conjunction with the Ugo Fix Leg Bag Straps

- Empty your urinary leg bag before it reaches 2/3 full.
- It is important that your urinary leg bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are free from obstruction and urine is draining freely.
- To drain your urinary leg bag, it is not necessary to remove the straps, however you may find it easier to do this for optimal drainage.
- Ensure that the tap at the bottom of your urinary leg bag does not touch the floor.

### CARING FOR YOUR DEVICE

- To maintain optimum performance of the **Ugo Fix Leg Bag Straps** we recommend that you wash the strap at 60°C. Tumble dry. Do not iron. Do not bleach. Do not dry clean.
- Replace your **Ugo Fix Leg Bag Straps** when the elasticity and support is reduced from washing and stretching;
- The **Ugo Fix Leg Bag Straps** should no longer be used if there are any signs of damage or degradation.

### STORAGE AND HANDLING

Keep dry. Keep away from sunlight.

### DISPOSAL

The **Ugo Fix Leg Bag Straps** must be disposed of according to local policies and waste disposal procedures, including any accessories or consumables used with the device.

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