optimummedical®

Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
LS2 9ET, Leeds, UK
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague, The Netherlands

EU REP

MT Promedt Consulting GmbH,
Ernst-Heckel-Straße 7,
66386 St. Ingbert, Germany

CH REP


MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug, Switzerland

Sponsor:
Optimum Medical Australasia Pty Ltd
Level 4 470 Church Street,
North Parramatta, NSW 2151, Australia

multifix™
gentle

MFG-IFU-Dec25-V12.0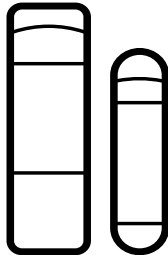

Instructions for use

PRODUCT REF / CONTENTS

Product Code	Description
3030	Multi Fix Gentle Large (140mm x 40mm).
3031	Multi Fix Gentle Small (110mm x 25mm).

This is a product information leaflet for the user. The **Multi Fix Gentle** has been prescribed for you by a Healthcare Professional. Do not share it with anyone else. Read all the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

PRODUCT DESCRIPTION

The **Multi Fix Gentle** is a non-sterile, silicone-based securement medical device used to provide securement of indwelling urinary catheters and medical tubing. The **Multi Fix Gentle** features a tubing securement tab mounted on a re-adhesive silicone gel strip. The tubing securement tab holds the catheter or tubing in place and the re-adhesive silicone gel strip is applied to the skin. There are two sizes of **Multi Fix Gentle**: Large (140mm x 40mm) and Small (110mm x 25mm). Both sizes are suitable for securing urinary drainage tubing and medical tubing which are up to 10mm in diameter.

INGREDIENTS/COMPOSITION

Description	Material Composition
Adhesive gel strip	Silicone
Tube fixation adhesive	Acrylic
Hook (Tube Fixation tab)	Polypropylene
Loop (to attach to hook)	Polypropylene/ Polyester
Release Liner, rear, and front peel off	Polyethylene film

INTENDED PURPOSE

The **Multi Fix Gentle** is a medical device intended to provide securement of indwelling urinary catheters and medical tubes which are up to 10mm in diameter. This securement is intended to provide comfort and help reduce the risk of trauma caused by inadvertent movement of the catheter or medical tubing.

INTENDED USER/S

The intended users of the **Multi Fix Gentle** are patients with indwelling urinary catheters or medical tubes, their carers and/or Healthcare Professionals.

CONTRAINDICATIONS

- Do not use the **Multi Fix Gentle** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.
- Do not use if you are sensitive or allergic to any of the device materials.
- Do not use on skin which is broken or inflamed.

WARNINGS AND PRECAUTIONS

- It is recommended that the **Multi Fix Gentle** is used in line with local policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Multi Fix Gentle** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it. Check the operating instructions of the medical tubing in use.
- The **Multi Fix Gentle** is a non-sterile medical device.
- Do not use **Multi Fix Gentle** for securement at the point where the catheter or medical tubing enters the body.
- The **Multi Fix Gentle** is not designed to support the weight of a urine or fluid collection bag. Always support the weight of your bag with an appropriate device.
- Do not disconnect your urine drainage bag during fitting of the **Multi Fix Gentle**. It is designed to allow fitting without disconnecting the system.
- Do not use barrier creams, moisturisers, moisturising soap, shower gel, wet wipes, baby wipes or talcum powder on the area of skin where the **Multi Fix Gentle** will be positioned as they may affect the adhesion of the **Multi Fix Gentle** to the skin.

- It is important that you check your skin beneath the **Multi Fix Gentle** regularly, to ensure it remains intact.
- Do not shave where the **Multi Fix Gentle** is to be applied to the skin as it may cause skin irritation.
- It is important to check that your catheter is not twisted prior to insertion into the **Multi Fix Gentle** (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely (see undesirable side effects).
- Do not use the **Multi Fix Gentle** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Multi Fix Gentle** is not made with natural rubber latex.
- This is a single-use device. Re-use of this device may result in patient infection / cross-contamination. Re/sterilisation, reprocessing, cleaning, and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.
- The device may not be suitable for all skin types.

UNDESIRABLE SIDE EFFECTS

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Multi Fix Gentle** and consult your Healthcare Professional and ensure patient records are updated. There may be an increased risk of complications from wearing the **Multi Fix Gentle** if you have decreased sensation and if the **Multi Fix Gentle** has not been fitted correctly.

Remove the **Multi Fix Gentle** immediately if you experience any of the following during or after use and seek medical advice:

- Swelling, inflammation or discomfort of the skin;
- Pressure markings to the skin, lesions or broken skin;
- Discolouration to the skin or oedema;
- Changes to fluid drainage;
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.

In addition to the above, if **Multi Fix Gentle** is used to secure a urinary catheter or drainage tube, remove the

Multi Fix Gentle immediately if you experience any of the following whilst using and seek medical help:

- Urine not draining into urine bag;
- Urine leaking outside the catheter;
- Cloudy or foul-smelling urine;
- Blood in urine;
- Changes to urine colour or volume.

Any serious incident or malfunction that has occurred in relation to the **Multi Fix Gentle** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare professional for advice before using the **Multi Fix Gentle**.

Always wash and dry your hands before and after handling your catheter or drainage tubing. If someone else is handling your catheter or drainage tubing, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection. The **Multi Fix Gentle** can be used for up to 7 days depending on your lifestyle. We recommend that you change your **Multi Fix Gentle** to suit your personal hygiene and physical needs. Change it sooner if it becomes damaged or soiled or there are signs of degradation. The **Multi Fix Gentle** can be worn in the shower to support your catheter. Ensure the edges are fully adhered to the skin prior to showering and pat dry the **Multi Fix Gentle** with a towel. Always support the weight of your catheter or drainage/medical tubing and any fluid collection bag when fitting your **Multi Fix Gentle**.

How to Apply the Multi Fix Gentle;

Before Applying the **Multi Fix Gentle** to your skin Your Healthcare Professional can advise on the most suitable position to place the **Multi Fix Gentle**.

- Choose the **Multi Fix Gentle** size most suitable for where it will be worn on the body.
- Identify where to place the **Multi Fix Gentle** and trim any excess hair if necessary (do not shave).

Application of the Multi Fix Gentle to your skin

For ease of application, it is recommended to apply the **Multi Fix Gentle** to the skin prior to securing the tubing.

- Remove the backing from the **Multi Fix Gentle**.

- Place the **Multi Fix Gentle** onto the skin in the desired spot and smooth in place.

Securement of Catheter or Medical Tubing into the MultiFix Gentle

- Lift the front tab on the **Multi Fix Gentle** strip and remove the release paper to reveal the adhesive base pad and adhesive tab.
- Place the tubing on the adhesive base pad and replace the adhesive tab over the tubing and smooth to secure in place.

Warning! Make sure your catheter or medical tubing is not twisted before securing in the **Multi Fix Gentle**.

- Ensure your catheter or tubing is not pulling and you are able to move around comfortably with it in place.

Removal and repositioning of the Multi Fix Gentle

You may want to reposition the **MultiFix Gentle** between daytime and bedtime. You can wear your

Multi Fix Gentle in bed and follow your usual overnight regime.

Your Healthcare Professional can advise on regular repositioning of the Multi Fix Gentle.

- Remove the catheter or medical tubing from the **Multi Fix Gentle** before repositioning the **Multi Fix Gentle** on the skin.
- Gently lift the edges of the **Multi Fix Gentle** adhesive and peel off taking care not to allow it to stick together.
- Reapply the **Multi Fix Gentle** to skin in the new position and smooth down in place.
- Additionally, the tubing length can also be repositioned within the **Multi Fix Gentle** as necessary

STORAGE AND HANDLING

- Keep dry and out of direct sunlight.
- Unopened packs have a 5-year shelf life.
- Do not use after expiry date.
- Store below 40°C.

DISPOSAL

The **Multi Fix Gentle** must be disposed of according to local policies and procedures, including any consumables used with the device.

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