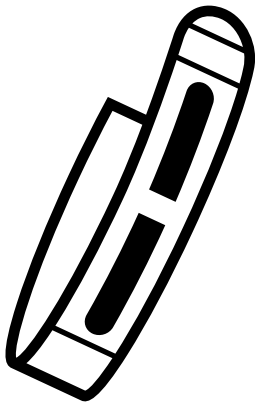
ugofix.™

bag belt

Instructions for use

optimummedical®

 Optimum Medical Solutions Ltd
Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom
T: +44(0) 845 643 5479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk

PRODUCT REF / CONTENTS

3012 Ugo Fix Bag Belt

This is a product information leaflet for the user.

The **Ugo Fix Bag Belt** has been prescribed/recommended for you by a Healthcare Professional. Do not share it with anyone else.

Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

DESCRIPTION

The **Ugo Fix Bag Belt** is a single patient, multiple use, non-sterile medical device for supporting the weight of a urinary leg bag that is attached to an indwelling urinary catheter or male urinary sheath.

The **Ugo Fix Bag Belt** consists of an elasticated strap, which is worn around the user's leg and is secured with a hook and loop fastening. The strap features silicone strips on the inner surface to help prevent movement on the skin. Two hook and loop tabs are mounted on the front of the strap, to which the top of the urinary leg bag is secured.

The **Ugo Fix Bag Belt** is 70cm long and is suitable for use on a leg with a circumference up to 65cm. It is adjustable and can be cut down to fit if necessary and supports urinary leg bags from 350ml to 750ml volumes, through the eyelet holes in the top band of the leg bag.

The **Ugo Fix Bag Belt** is washable and reusable.

INGREDIENTS/COMPOSITION

Silicone strip

Nylon hook

Elastic loop (polyester, nylon, spandex)

INTENDED PURPOSE

The **Ugo Fix Bag Belt** provides enhanced support for the weight of your urinary leg bag, allowing it to fill discreetly whilst worn on the leg, enabling mobility.

This support is intended to provide comfort and reduce the risk of injury from the weight of your leg bag pulling on your catheter or urinary sheath.

INTENDED USER

The intended users of the **Ugo Fix Bag Belt** are patients with urinary leg bags attached to indwelling urinary catheters or male urinary sheaths, and their Carers and/or Healthcare Professionals.

CONTRAINDICATIONS

- Do not use the **Ugo Fix Bag Belt** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.
- Do not use if you are sensitive or allergic to any of the device materials.

- Do not use on skin which is broken or inflamed.

WARNINGS AND PRECAUTIONS

- It is recommended that the **Ugo Fix Bag Belt** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.
- Ensure that the **Ugo Fix Bag Belt** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it.
- Do not disconnect your urinary leg bag when fitting your **Ugo Fix Bag Belt**. It is designed specifically to allow fitting without disconnecting the system.
- Always support the weight of your catheter and urinary leg bag when fitting your **Ugo Fix Bag Belt**.
- It is important that your catheter is also secured to prevent inadvertent movement, in addition to using the **Ugo Fix Bag Belt**.
- Ensure you measure around your thigh, upper knee or lower leg at the position you intend to wear your **Ugo Fix Bag Belt** before cutting to length. Allow for shrinkage after washing. If the bag belt is too short it can restrict circulation (see how to use).
- Only wear the **Ugo Fix Bag Belt** on skin which is clean and dry with no wounds or sores.
- The **Ugo Fix Bag Belt** is elasticated. It is important that you visually check your skin regularly to ensure it remains healthy and intact and your **Ugo Fix Bag Belt** is not too tight and marking your skin.
- It is important to check that your catheter or urinary leg bag tubing are not twisted prior to attaching onto the **Ugo Fix Bag Belt** (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely (see undesirable side effects).
- Do not use the **Ugo Fix Bag Belt** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Fix Bag Belt** is not made with natural rubber latex.
- The **Ugo Fix Bag Belt** is a non-sterile medical device.
- The **Ugo Fix Bag Belt** is a reusable device. Please refer to 'How to care for your device' to ensure the product performance is maintained.

UNDESIRABLE SIDE EFFECTS

In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Fix Bag Belt** and consult your Healthcare Professional and ensure patient records are updated.

There may be an increased risk of complications from wearing the **Ugo Fix Bag Belt** if you have decreased sensation or changes to your leg circumference (oedema) and if the **Ugo Fix Bag Belt** has not been fitted correctly.

Remove the **Ugo Fix Bag Belt** immediately if you experience any of the following during use and seek medical advice:

- Swelling, inflammation or discomfort of the skin.
- Pressure markings to the skin, lesions or broken skin.
- Discolouration to the skin or oedema.
- Urine not draining into urine bag.
- Urine leaking outside the catheter.

- Cloudy or foul-smelling urine.
- Blood in urine.
- Changes to urine colour or volume.
- Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.

Any serious incident or malfunction that has occurred in relation to the **Ugo Fix Bag Belt** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Consult your Healthcare Professional for advice before using the **Ugo Fix Bag Belt**.

Always wash and dry your hands before and after handling your catheter or drainage tubing and **Ugo Fix Bag Belt**. If someone else is handling your catheter or drainage tubing and bag belt, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection.

We recommend that you change your **Ugo Fix Bag Belt** daily along with your catheter hygiene regime. Only change it sooner if it is damaged or soiled.

The **Ugo Fix Bag Belt** is not waterproof; however you can still bathe and shower when wearing it to support your leg bag. Replace the wet **Ugo Fix Bag Belt** with a clean dry one afterwards.

Identify the best place to wear the Ugo Fix Bag Belt

The **Ugo Fix Bag Belt** can be worn to support your urinary leg bag on your thigh, upper knee or lower leg. The position you wear your bag belt and urinary leg bag will depend on whether you have a urethral or suprapubic catheter, the length of your catheter and leg bag tubing, your leg bag volume and your level of mobility.

Measuring and adjusting the Ugo Fix Bag Belt to the correct length

1. Using a tape measure, measure around your leg at the position you intend to wear your **Ugo Fix Bag Belt**.
2. Then using the tape measure, measure the length of the **Ugo Fix Bag Belt**, add at least 5cm for hook and loop overlap and cut to length.

! Warning - Do not cut the **Ugo Fix Bag Belt** whilst it is being worn and stretched.

How to apply your Ugo Fix Bag Belt

Fitting options:

You may choose to attach your leg bag to the **Ugo Fix Bag Belt** first and then to your leg afterwards (Method A) or you may find it easier to fit the **Ugo Fix Bag Belt** to your leg first and then attach your leg bag afterwards (Method B).

Method A:

Feed the grey hook and loop tabs through the four eyelet holes at the top of your urinary leg bag and secure.

Place the **Ugo Fix Bag Belt** around the leg in the desired position with the silicone line facing down on the skin and fix in place with hook and loop.

Method B:

Place the **Ugo Fix Bag Belt** around the leg in the desired position with the silicone line facing down on the skin and fix in place with hook and loop.

Feed the grey hook and loop tabs through the four eyelet holes at the top of your urinary leg bag and secure.

! Warning - Make sure your catheter or drainage tubing are not twisted before securing onto the **Ugo Fix Bag Belt**.

Ensure the **Ugo Fix Bag Belt** is not too tight, your catheter is not pulling and that you are able to move around comfortably with it in place.

Removal and repositioning

It is recommended that you remove the **Ugo Fix Bag Belt** at bedtime.

Your Healthcare Professional can advise on regular repositioning of the **Ugo Fix Bag Belt**.

Using your urinary leg bag in conjunction with the Ugo Fix Bag Belt

- Empty your urinary leg bag before it reaches 2/3 full.
- It is important that your urinary leg bag is positioned below the level of your bladder and to check regularly that your catheter and drainage tubing are free from obstruction and urine is draining freely.
- You do not need to remove your urinary leg bag from the bag belt to drain it.
- Ensure that the tap at the bottom of your urinary leg bag does not touch the floor.

CARING FOR YOUR DEVICE

- To maintain optimum performance of the **Ugo Fix Bag Belt** we recommend that you hand wash the strap in temperatures up 30°C. Do not tumble dry. Do not iron. Do not bleach. Do not dry clean.
- Replace the **Ugo Fix Bag Belt** when the elasticity and support is reduced from washing and stretching.
- The **Ugo Fix Bag Belt** should no longer be used if there are any signs of damage or degradation.

STORAGE AND HANDLING

Keep dry. Keep away from sunlight.

DISPOSAL

The **Ugo Fix Bag Belt** must be disposed of according to local policies and waste disposal procedures, including any accessories or consumables used with the device.

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MedEnvoy Global B.V.
MT Promedt Consulting GmbH,
Optimum Medical Australasia Pty Limited,