# ugofix.™

sleeve

**EU REP**

MT Promedt Consulting GmbH  
Ernst-Heckel-Straße 7  
66386 St. Ingbert, Germany

MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33 - Suite 123  
2595 AM The Hague, The Netherlands

Sponsor:  
Optimum Medical Australasia Pty Limited,  
Level 4, 470 Church Street, North Parramatta  
New South Wales, 2151, Australia


**REF 3005/3006/3007/3008/3027**

Ugo Fix Sleeve: IFU-Mar26-V1.0.0

# ugofix.™

sleeve

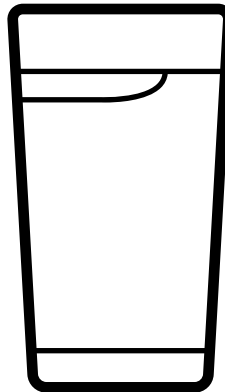## Instructions for use

**optimummedical®**

Optimum Medical Solutions Ltd  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom  
T: +44(0) 845 643 5479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

## PRODUCT REF / CONTENTS

3005	Small	24cm – 39cm	Yellow
3006	Medium	36cm – 55cm	Blue
3007	Large	40cm – 70cm	Brown
3008	Extra-large	65cm – 90cm	Green
3027	XXL	75cm – 105cm	White

This is a product information leaflet for the user.

The **Ugo Fix Sleeve** has been prescribed/recommended for you by a Healthcare Professional. Do not share it with anyone else. Read all of the product information leaflet prior to use and retain for your information. If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from your Healthcare Professional.

**DESCRIPTION** | The **Ugo Fix Sleeve** is a single patient, multiple use, non-sterile, medical device for supporting the weight of a urinary leg bag that is attached to an indwelling urinary catheter or male urinary sheaths. The **Ugo Fix Sleeve** is made from elasticated fabric and is worn on the user's leg. The sleeve features an integral front pocket which holds the urinary leg bag in place as urine is drained into it. There are 5 sizes of the **Ugo Fix Sleeve**, and all sizes are suitable for holding urine drainage bags from 350ml to 750ml volumes. The **Ugo Fix Sleeve** has an external seam and is washable and reusable.

**INGREDIENTS/COMPOSITION** | Polyester and elastane sleeve.

**INTENDED PURPOSE** | The **Ugo Fix Sleeve** provides support for the weight of your urinary leg bag, allowing it to fill discreetly whilst worn on the leg, enabling mobility. This support is intended to provide comfort and reduce the risk of injury from the weight of your urinary leg bag pulling on your catheter or male urinary sheath. The **Ugo Fix Sleeve** is elasticated and supports the controlled expansion of the urinary leg bag, ensuring even distribution of urine and reduced noise when filling.

**INTENDED USER** | The intended users of the **Ugo Fix Sleeve** are patients with urinary leg bags attached to indwelling urinary catheters or male urinary sheaths, their Carers and/or Healthcare Professionals.

**CONTRAINDICATIONS**

- Do not use the **Ugo Fix Sleeve** unless you have been assessed as suitable by a Healthcare Professional and it has been recommended for you.
- Do not use the if you are sensitive or allergic to any of the device materials.
- Do not use on skin which is broken or inflamed.

**WARNINGS AND PRECAUTIONS**

- It is recommended that the **Ugo Fix Sleeve** is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional;
- Ensure that the **Ugo Fix Sleeve** is suitable for the intended use and is compatible with other medical devices to be used in conjunction with it;

- Do not disconnect your urinary leg bag when fitting your **Ugo Fix Sleeve**. It is designed specifically to allow fitting without disconnecting the system.
- Always support the weight of your catheter and urinary leg bag when fitting your **Ugo Fix Sleeve**.
- It is important that your catheter is also secured to prevent inadvertent movement, in addition to using the **Ugo Fix Sleeve**.
- It is important to measure for the correct size. Allow for shrinkage after washing and drying. If the sleeve is too small, it can restrict circulation and if too large will not give adequate support (see how to use).
- Only wear the **Ugo Fix Sleeve** on skin which is clean and dry with no wounds or sores.
- The **Ugo Fix Sleeve** is elasticated. It is important that you visually check your skin regularly to ensure it remains healthy and intact and your sleeve is not too tight and marking your skin.
- It is important to check that your catheter or urine drainage bag are not twisted prior to inserting your leg bag into the **Ugo Fix Sleeve** (see how to use) and to regularly check that your catheter and all drainage tubing are free from obstruction and urine is draining freely (see undesirable side effects).
- Do not use the **Ugo Fix Sleeve** if it is damaged or soiled.
- Do not use if the packaging is damaged or unintentionally opened prior to use.
- Keep out of reach of children.
- The **Ugo Fix Sleeve** is not made with natural rubber latex.
- The **Ugo Fix Sleeve** is a non-sterile medical device.
- The **Ugo Fix Sleeve** is a reusable device. Please refer to 'How to care for your device' to ensure the product performance is maintained.

**UNDESIRABLE SIDE EFFECTS** | In rare cases local irritation or hypersensitivity may occur. Symptoms of irritation and hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using the **Ugo Fix Sleeve** and consult your Healthcare Professional and ensure patient records are updated. There may be an increased risk of complications from wearing the **Ugo Fix Sleeve** if you have decreased sensation or changes to your leg circumference (oedema) and if the **Ugo Fix Sleeve** has not been fitted correctly.

Remove the **Ugo Fix Sleeve** immediately if you experience any of the following during use and seek medical advice:

- Swelling, inflammation or discomfort of the skin;
  - Pressure markings to the skin, lesions or broken skin;
  - Discolouration to the skin or oedema;
  - Urine not draining into urine bag;
  - Urine leaking outside the catheter;
  - Cloudy or foul-smelling urine;
  - Blood in urine;
  - Changes to urine colour or volume;
  - Feeling unwell, high temperature, nausea/vomiting or in any pain/discomfort.
- Any serious incident or malfunction that has occurred in relation to the **Ugo Fix Sleeve** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

**HOW TO USE** | Consult your Healthcare Professional for advice before using the **Ugo Fix Sleeve**. Always wash and dry your hands before and after handling your catheter or drainage bag tubing and sleeve. If someone else is handling your catheter or drainage bag tubing and sleeve, they must wash and dry their hands and apply a clean pair of gloves on every occasion to avoid cross infection. We recommend that you change your **Ugo Fix Sleeve** daily along with your catheter hygiene regime. Only change it sooner if it is damaged or soiled. The **Ugo Fix Sleeve** is not waterproof; however, you can still bathe and shower when wearing it to support your leg bag. Replace the wet **Ugo Fix Sleeve** with a clean dry one afterwards.

**Identify the best place to wear the Ugo Fix Sleeve** | The **Ugo Fix Sleeve** can be worn on your thigh or calf.

The position you wear your sleeve and leg bag will depend on the length of your catheter and leg bag tubing, your leg bag volume and your level of mobility.

**Selecting the correct size of your Ugo Fix Sleeve** | Using a measuring tape, measure around your thigh or calf (at the widest part) at the position you intend to wear your **Ugo Fix Sleeve**. Select the correct size.

**How to apply your Ugo Fix Sleeve** (see illustration on the back)

1. Pull the **Ugo Fix Sleeve** over your foot and position on your thigh or calf. Ensure the coloured strip is at the top and on the outside of the sleeve, and that the urinary leg bag pouch is facing forwards.
2. Slide your urinary leg bag into the pouch, through the opening at the top. Do not disconnect your urinary leg bag from your catheter or male urinary sheath to fit your **Ugo Fix Sleeve**.

**Warning!** - Make sure your catheter or drainage bag tubing are not twisted before putting the leg bag into the leg bag pouch.

3. Push the drainage tap from your urinary leg bag through the hole at the bottom of the sleeve. Adjust your **Ugo Fix Sleeve** to ensure optimum drainage.
4. Ensure your catheter and drainage bag tubing is not pulling and you are able to move around comfortably with it in place.

**Removal and repositioning** | It is recommended that you remove the **Ugo Fix Sleeve** at bedtime. Your Healthcare Professional can advise on regular repositioning of the **Ugo Fix Sleeve**.

**Using your urine drainage bag in conjunction with the Ugo Fix Sleeve**

- Empty your urinary leg bag before it reaches 2/3 full.
- It is important that your urinary leg bag is positioned below the level of your bladder and to check regularly that your catheter and drainage bag tubing are free from obstruction and urine is draining freely.
- You do not need to remove your urinary leg bag from the sleeve to drain it.
- Ensure that the tap at the bottom of your urinary leg bag does not touch the floor.

**CARING FOR YOUR DEVICE**

- To maintain optimum performance of the **Ugo Fix Sleeve** we recommend that you wash the sleeve at 60°C.
- Tumble dry on a low heat. Do not iron. Do not bleach. Do not dry clean.

- Replace the **Ugo Fix Sleeve** when the elasticity and support is reduced from washing and stretching.
- The **Ugo Fix Sleeve** should no longer be used if there are any signs of damage or degradation.

**STORAGE AND HANDLING** | Keep dry and out of direct sunlight.

**DISPOSAL** | The **Ugo Fix Sleeve** must be disposed of according to local policies and waste disposal procedures, including any consumables used with the device.

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